

Myoline Single Fiber *Instruction For Use*

EMG DISPOSABLE SINGLE FIBER EMG BIPOLAR NEEDLE

A technologically advanced disposable bipolar needle electrode designed for patient comfort along with top reliability and safety for the user.

CLINICAL APPLICATION


To be used for Single Fiber Electromyography tests (SFEMG)

PACKAGING

Each pack contains one ready for use sterile needle, to be used for one patient only, not resterilizable. Each box contains 5 needles.

TECHNICAL FEATURES

- 1 - Moulded needle with ergonomic handgrip
 - 2 - Special tip sharpening for patient and user friendly introduction
 - 3 - Recording area marker
 - 4 - Press-on tongues to automatically disconnect needle after introduction into hospital disposal container
 - 5 - Easy to handle protective sheath in rigid material
 - 6 - ETO sterilizable light and flexible leadwire, diameter 1,2 mm, with 5-pole DIN 45322 connector. (Provided separately)
 - 7 - Moulded connections on both ends.
- Entirely shielded handgrip connected with the grounding of the device

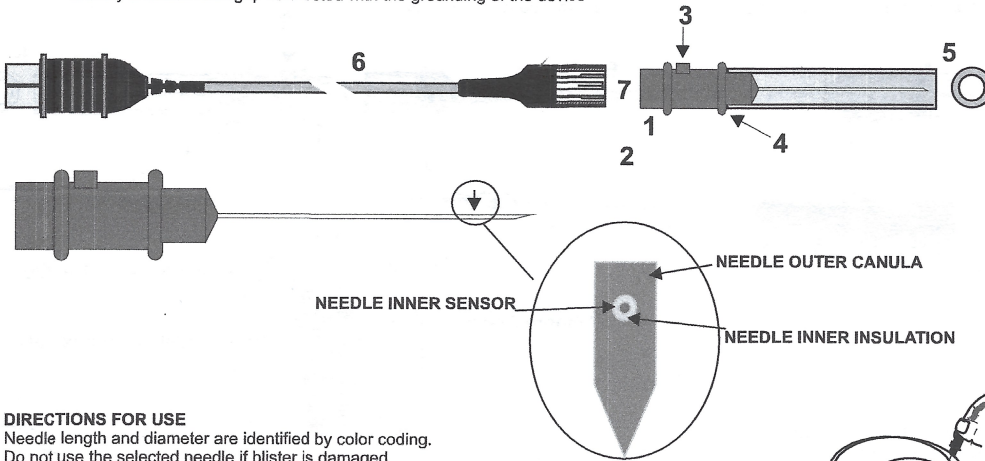


DIRECTIONS FOR USE

Needle length and diameter are identified by color coding.

Do not use the selected needle if blister is damaged.

Open the pack following the instructions on the sterile pouch:

- 1) Remove needle from sterile pouch
- 2) Connect needle to connector
- 3) Remove the needle sheath
- 4) Introduce needle into area to be tested taking into account position of recording area marker.
- 5) After test dispose of needle using the same procedure as, and a specific container for subdermal needle, or alternatively enclose needle back into sheath before disposal.

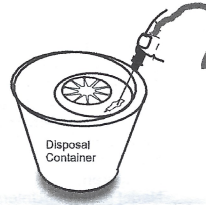

Use Only



Latex free



Do not resterilize



Do not use if package is damaged



Spes Medica S.r.l.
Via Europa-Zona Industriale
84091 Battipaglia (SA) Italy
Tel: +39 (0)0828 614191
Fax: +39 (0)0828 341788

www.spesmedica.com, info@spesmedica.com



Caution: This single-use product is not designed for re-use. Attempts to clean and re-use this single-use device expose patients to a risk of cross contamination.