

FAQ's FOR

HEADALOFT™

- **Does HEADALOFT attach to all kinds of wheelchairs and other seating and mobility devices?**

Yes, HEADALOFT easily attaches to all kinds of seating and mobility devices equipped with a headrest, whether a wheelchair, a stroller, a stander, a walker, a car seat, and more.

- **Does HEADALOFT fit every head size?**

With its three different size forehead straps, and its Velcro versatility, not only is there a perfect fit for everyone there are also options for adjustments.

- **Who needs anterior support?**

Anyone who suffers from lack of head control can benefit from professional anterior head support. Basic passive support holds the head upright enabling better breathing, feeding, communicating, etc. Additionally, in some ongoing case studies this support shows a positive proactive impact on the user's independent ability to hold their head up to various extents.

- **By using HEADALOFT does the user always have to have the anterior head support strap on?**

No, HEADALOFT is a versatile head support system which is easily adjustable. The forehead strap can be put in the up position (slightly above the user's head) when not in use, and then put easily back in place in the down position when wanted. For example, some user's use HEADALOFT only during certain activities such as eating, watching TV, using a monitor and for eye gaze control practice while others use it a vast majority of the time.

- **Can you shift the product from one device to another (i.e; wheelchair to car seat, stroller to stander, etc.)**

Yes, HEADALOFT is easily attachable and detachable which makes it easy to shift from one device equipped with a headrest to another.

- **Is it safe to use HEADALOFT on car seats?**

HEADALOFT has been crash tested at the acclaimed US Calspan lab. The US Federal Safety Standards tests showed that HEADALOFT does not alter the way the car seat or the special needs car seat preforms in a crash.

Who should I consult with before use?

- It is always advisable to consult with your physician, physiotherapist, occupational therapist or your ATP before using any new assistive product. Our HEADOVATIONS team is always available to answer any questions, feel free to contact us at info@headaloft.com.

Is HEADALOFT suitable for people with ALS?

- HEADALOFT generally is suitable for most people who lack head control, including ALS patients. Consult your doctor, PT or OT to see if it's right for you.

Does HEADALOFT meet medical regulations?

- Yes, HEADALOFT is an FDA registered for the US and CE registered for the EU.

For more information please visit us at Headaloft.com

Developed by:



In Partnership with:



13485:2016