

ALL ON X

STEP-BY-STEP CHAIRSIDE PROTOCOL

1st Appointment Implant Level Impression



- Take a preliminary impression.
- Duplicate existing prosthesis.
- Select tooth shade on RX.

2nd Appointment Verification Jig Wax Bite



- Insert multi-unit abutments.
- Seat and lute the verification jig.
- Take final Impression with custom tray.
- Try wax rim, mark the midline and smile line.
- Take a bite registration with wax rim.

3rd Appointment Teeth Try-In



- Teeth try in
- and verify esthetics and function.
- Select gingival shade on RX.

4th Appointment Delivery of PMMA



- Try in the provisional implant prosthesis (PMMA).
- If modification is made, send back PMMA to duplicate the final restoration.

5th Appointment Final Prosthesis



- Deliver the final Prudential zirconia restoration.