20211102





HOMIETEC SMART HEALTH

A Cool model of Smart Blood Pressure Monitor Promotion

The analogy between HOMIETEC smart blood pressure monitor and OMRON smart blood pressure monitor

(The information of Omron's smart blood pressure monitor comes from JD Mall jd.com)

(Note: The content of this document comes from machine (software) translation. If you have any improper or doubts, please write to homietec@outlook.com))

	OMRON MEE 118 MEE 78 ↔ START STOP	HOMIETEC®
Selling price	1098 yuan RMB (equal to 168.92 US Dollar)	438 yuan RMB (equal to 67.38 US Dollar + plus shipping cost)
Special	840 yuan RMB (7.65% off, equal to 129.23 US	288 yuan RMB (6.57 off, equal to 44.30 US
offer	Dollar), only special offer from 20:00 on October	Dollar + plus shipping cost, only 950 units in
	31st to 24:00 on November 3rd, 2021, the top 60	stock, while stocks last)
	will be given a 48-yuan temperature gun	

Second Special	without	218 yuan RMB (4.97% off, equal to 33.53 US Dollar + plus shipping cost, only 950 units in
opeen		stock, while stocks last)
Three specials	without	Registered member for one purchase of 4 units for 198 yuan RMB (4.52% off, equal to 30.46 US Dollar + plus shipping cost, only 950 units in stock, while stock lasts)
Special care 1	without	Especially enjoy a one-month experience period, calculated from the date of delivery + 7 days to + 30 days + 7 days (returning to the warehouse), provided that the product and packaging are not damaged when returning to the warehouse For our of Mainland China areas, please mail us to check with us again considering higher shipping cost.
Special care 2	without	 Within 2 years from the date of delivery: 1 The same or similar models can be upgraded for free, and the user pays the round-trip freight 2 Repurchase or renew other models, enjoy 30% off, users pay round-trip freight
Functio n	High pressure, low pressure, heart rate	High pressure, low pressure, heart rate
Precisio n	Medical grade, ±3mmHg (0.4kPa) / ±5%	Medical grade, ±3mmHg (0.4kPa) / ±5%
Certifica tion	American Association for the Advancement of Medical Devices (Testing Association) AAMI International Accuracy Standards, British Hypertension Association BHS, European Hypertension Association ESH, China Medical Devices NMPA (Source: JD.com Mall)	US medical device FDA (K172171, model B06T), EU medical device MDR (G2180190084005, model B06T), Chinese medical device NMPA (Guangdong Medical Equipment Injection Standard 20142200322, model B61T)
Warning	None, high-pressure warning of the fuselage	Yes, high-pressure warning on the fuselage, 1 red button warning 2 voice broadcast warning
voice	without	Yes, two voice broadcasts in Chinese and English, 3 levels of volume adjustable, silent mode can be set
Battery	4 AAA batteries	3.7V/600mAh rechargeable battery, fully charged for 3-4 hours, can be fully charged 150 measurements (standby about 7-21 days)
One key measure ment	Have	Have

Intellige	Have	Have	
nt			
pressure			
show	Yes, the body has a digital display	No, there is no digital display on the body	
Stored	Unknown	Yes, 120 sets of stored value, 3 sets of averages	
value on			
the			
fuselage			
Backgro	Unknown	Have	
und			
music			
Automa	Unknown	60 seconds	
tic shut-			
down			
Cuff	Have	None, but there is an error prompt (the body	
self-test		voice broadcasts an error code prompt, and the	
display		App displays an error code prompt)	
Irregular	Have	without	
pulse			
wave			
display			
APP	Yes, official version, Android or Apple	Yes, beta version, Android or Apple	
Carrying	Have	None, but the hardened color box can be used	
case		as a collection box (also can be carried on	
		travel)	
Origin	import	China	
Shelf	2 years	2 Years	
life			
Dimensi	Length 120*width 85*height 20mm	Length 121*width 67*height 28mm	
ons			
weight	240g (without battery)	237.3g (including rechargeable battery)	
Arm	220-420mm	230-360mm	
circumf			
erence			

The following comparison of measured parameters is for reference only

Compari	High pressure 117, low pressure 69, heart rate	High pressure 120 (tolerance +3mmHg /
son of 3	78	+2.56%), low pressure 64 (tolerance -5mmHg / -
sets of	(Omron Bluetooth Desktop blood pressure	7.81%), heart rate 82 (tolerance +4 / +5.12%)
mean	monitor HEM-8732T)	
values		
measure		

d by any	ตระก	
user		
usei		
Compari	High pressure 102.67, low pressure 73, heart	High pressure 103.17 (tolerance
son of 6	rate: none	+0.5mmHg/+0.49%), low pressure 66.33
sets of	(Mercury sphygmomanometer)	(tolerance -6.67mmHg/-10.06%), heart rate:
mean		83.67 (none)
values		(HOMIETEC random blood pressure monitor
measure		device 1#)
d by any		
user		
Compari	High pressure 102.67, low pressure 73, heart	High pressure 102.33 (tolerance -0.34mmHg/-
son of 6	rate: none	0.33%), low pressure 64.17 (tolerance -
sets of	(Mercury sphygmomanometer)	8.83mmHg/-13.76%), heart rate: 81.83 (none)
mean		(HOMIETEC random blood pressure monitor
values		device 2#)
measure		
d by any		
user		
Compari	High pressure 103.17, low pressure 66.33, heart	High pressure 102.33 (tolerance -0.84mmHg/-
son of 6	rate: 83.67	0.82%), low pressure 64.17 (tolerance -
sets of	(HOMIETEC random blood pressure monitor	2.16mmHg/-3.37%), heart rate: 81.83 (tolerance
mean	device 1#)	-1.84/-2.24%) (HOMIETEC random blood
values		pressure monitor device 2#)
measure		
d by any		
user		

Attachment 1:

In November 2018, the actual measurement value of any HOMIETEC blood pressure monitor device of the same user and the OMRON Bluetooth blood pressure monitor HEM-8732T device is compared with reference to the attached table (the measurement accuracy is also affected by the user's status, method, time, etc., which is a non-standard answer and is for reference only.):

2018.11.15测试	测试人	时间	高压	低压	心室
		16:44	120	64	82
	HOMIETEC	17:15	116	63	82
	血压计	17:39	123	64	83
	游文清	极差	т	1	1
		平均值	120	64	82
		17:20	114	65	78
	欧姆龙	17:27	118	69	80
	浙文清	17:33	120	72	75
		极差	6	7	5
		平均值	117	69	78

Tolerance: high pressure +3mmHg (+2.56%), low pressure -5mmHg (-7.81%), heart rate +4 (+5.12%)

Attachment 2:

In November 2018, the comparison between HOMIETEC random blood pressure monitor and mercury sphygmomanometer:

豪美特	水銀	豪美好
108 73 94	104 72	109 70 88
105 73 92	104 72	105 72 89
104 71 87	104 72	106 71 88
102+ 61 75	102 74	99 60 76
100 59 77	100 74	97 56 75
98 61 77	102 74	98 56 75

Device 1#, 6 groups average: high pressure 103.17, low pressure 66.33, heart rate: 83.67

Mercury meter: 6 groups average: high pressure 102.67, low pressure 73, heart rate: none

Tolerance 1#: high pressure +0.5mmHg (+0.49%), low pressure -6.67mmHg (-10.06%), heart rate: none

Device 2#, 6 groups average: high pressure 102.33, low pressure 64.17, heart rate: 81.83 Mercury meter: 6 groups average: high pressure 102.67, low pressure 73, heart rate: none Tolerance 2#: high pressure -0.34mmHg (-0.33%), low pressure -8.83mmHg (-13.76%), heart rate: none

Attachment 3:





Welcome to contact us via Email homietec@outlook.com for more information.



Homietec Enterprise (Xiamen) Co., Ltd. is a comprehensive enterprise integrating Research and Development, Manufacturing, Sales, Retail, Online Shopping, Online expert health clinic and so on. The company was established in 2009 with a registered capital of 20 million yuan (3.5M US Dollars). It has been more than 12 years of development history.



Main products:

- . Medical and health,
- . Massage and therapy,
- . Sports and fitness,
- . Beauty skin and body shape,
- . Maternal and baby care,
- . Healthy appliances,



Core products:

- . Smart blood pressure meter,
- . Smart blood glucose meter,
- . Smart blood lipid meter,
- . Smart uric acid meter,
- . Smart body fat management,

Business mode: There are two modes of "Homietec" brand and OEM / ODM customization. The channels include online shopping and off-line stores. The current markets cover more than 30 countries or regions such as China, Europe, and the United States; more partners are welcome to come to discuss cooperation.

Business philosophy: "Smart Health, Assist Healthier" as the development philosophy, "Innovation, Brand, Quality" as the corporate purpose.

We are looking for global brand distributors as well as global OEM/ODM partners.

Welcome to contact us or visit our website www.homietec.com for more details.

Yours Bob

Homietec Technology is a Smart Health Solution

Homietec Enterprise Co., Limited. (www.homietec.hk) Homietec Enterprise (Xiamen) Co., Ltd. (www.homietec.com) Homietec Health Technology (Xiamen) Co., Ltd. (www.homietectechnology.cn) Mobile and Message: +86 13616059395. Telephone Number: +86 5925601178. Email and Skype: <u>homietectechnology@outlook.com</u> Research and Manufacturing Center: Digital Economic Industrial Park, Xiamen, China (Postal Code 361011)

This email contains secret information. Please abide by the obligation of confidentiality and do not disclose it to third parties. If you are not the correct recipient, please delete this message immediately, thank you for your cooperation. Any complaint or suggestion, please kindly mail to <u>homietectechnology@outlook.com</u>

Homietec Technology Smart Health Solution Administration Center Research and Manufacturing Center Address: Digital Economic Industrial Park, Xiamen, China (Postal Code 361011)