


Needle Inserters

Premium - Single Needle Inserter - (Hand Needles Only)

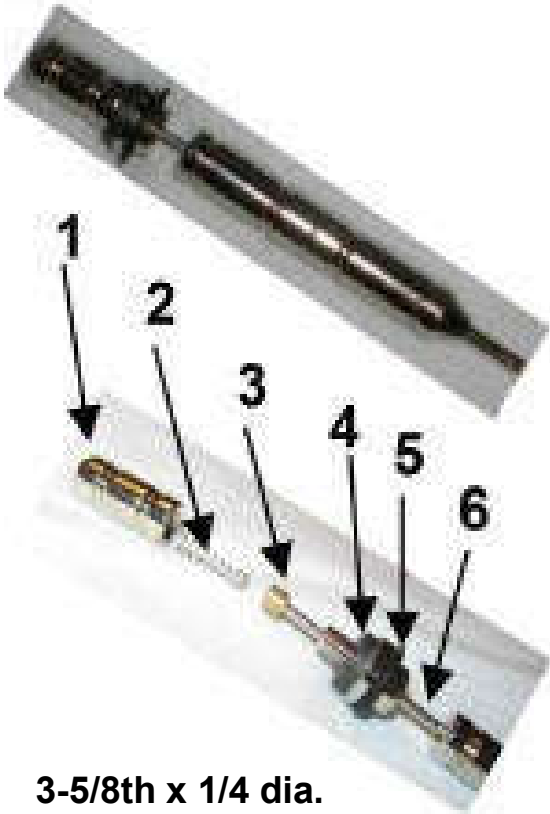

3-5/8th x 1/4 dia.

1. Plunger Head-Retains Spring
2. Shock Absorbing Spring
3. Needle retention magnet
4. Needle pickup magnet
5. High friction lifting washer
6. Non-stick, stainless drop shaft

Gravity Drop

- Invert the inserter
- Drop needle in handle first (# 3 magnet holds needle)
- Hold inserter between thumb & middle fingers.
- Raise the plunger with the index finger on washer
- Release the plunger so it drops by it's own weight.
- Lift inserter straight up off the needle vertically



Note: This will give a rapid, painless insertion.

Auto-1- Single Needle Inserter-(Hand Needles Only)



1. Insertion spring trigger
2. Body
3. Holding magnet
4. Adjustable Retainer
5. Depth Adjustment
6. Replaceable Plastic Needle Guide Tip

Spring Loaded

- Invert the inserter
- Drop a needle handle first into the tip
- Internal magnet holds the needle
- Position the inserter tip on the desired location
- Press down gently on the insertion trigger
- Spring loaded plunger instantly inserts the needle

Auto-35 Multi Needle Inserter-(Hand Needles only)



Quick & Convenient Inserter

- Adjustable insertion depth
- Single press spring loaded trigger for fast insertion
- Light weight , chrome plated plastic outer body
- Durable metal inner triggering mechanism



6.5" x 1 " dia

Pre Loaded Cartridge-Hand Needles Only

35 Stainless steel needles- sterilized-(gamma ray)

Unused needles in cartridge remain sterile


Ideal for:

- Hand
- Ear
- Face
- Shallow body pts.




Replaceable Transparent Tips

- Prevent contamination between patients
- Visual confirmation that a Needle is in the tip ready for insertion
- Easy snap-on & off
- Clean with alcohol
- Accurate needle guide

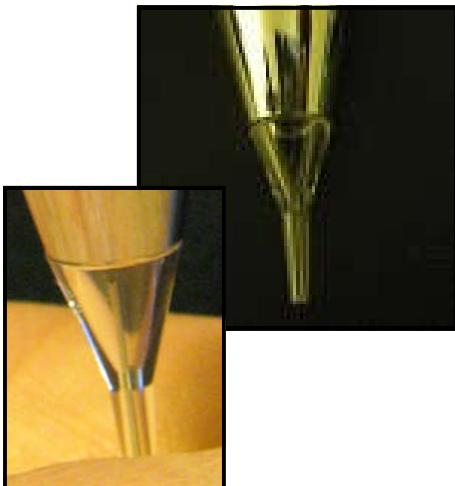

Auto 35 Inserter Diagram

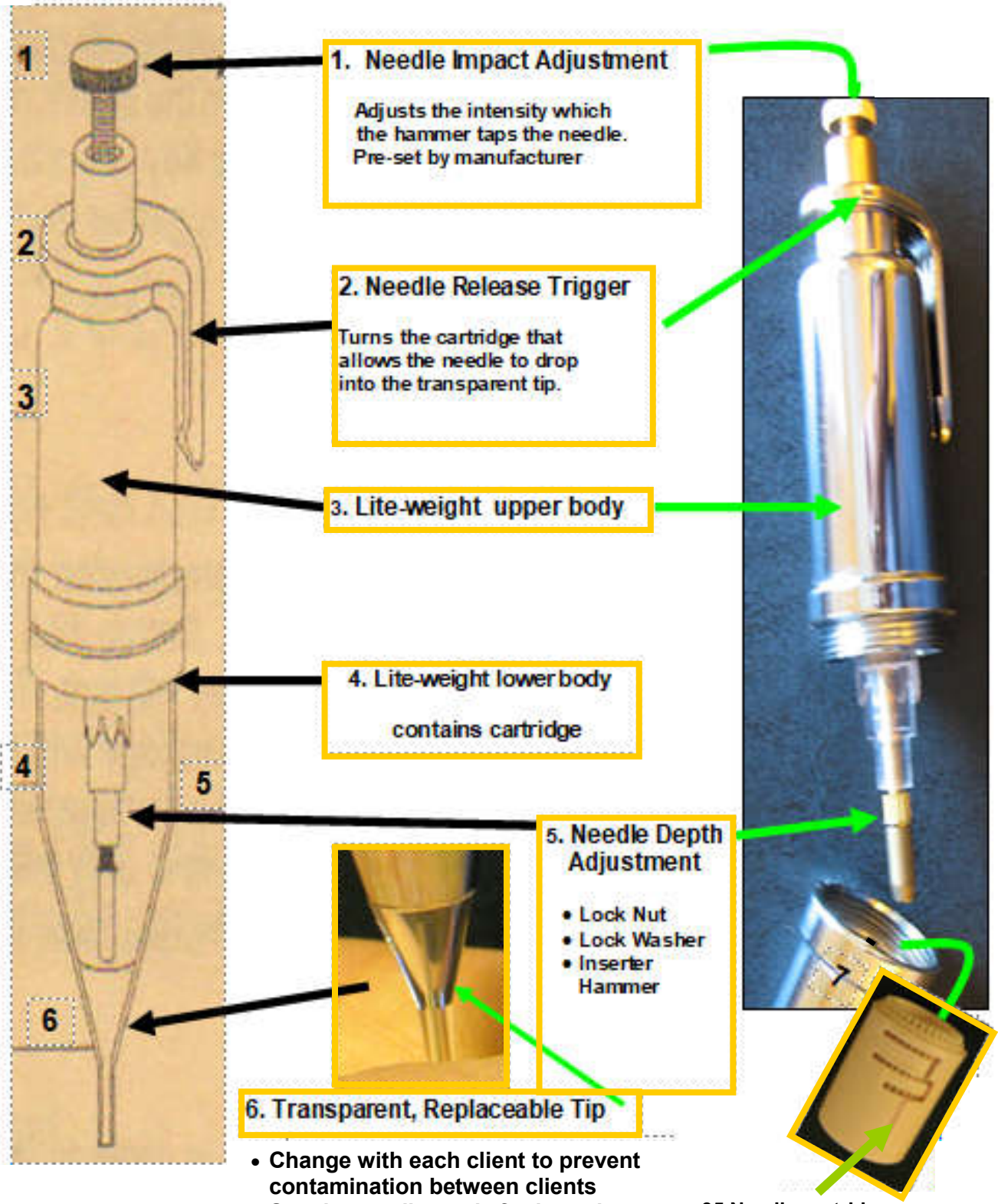


Auto-35 Instructions



1. Hold the inserter vertically on the acupoint
2. Drop the needle by depressing the Release Trigger (#2) to rotate the cartridge




3. Look into the transparent tip (#3) to make sure the needle has dropped into position.

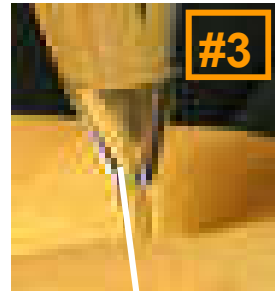

If Needle Has Not Dropped:

- Gently tap the side of the inserter body with your finger
- If no needle drops, remove the cartridge to insure that a needle did not jam part way out of the cartridge.
- If a needle is stuck, remove it with a forceps.
- Re-insert the cartridge.
- If the cleared cartridge still does not work, replace it.

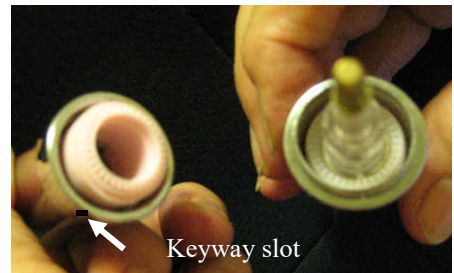

To insert cartridge, align the protrusion on the cartridge side with the keyway slot in the lower body



4. To Insert needle. slowly press the Needle Release Trigger (#4).
5. Quickly lift the inserter vertically.....sideways removal will pull the needle out of the hand.

Tonify and Sedate



After the needle is in the tip slant to sedate or tonify

- Loosen – grasp A & B between thumb & forefinger
- Simultaneously twist
 - A- Hammer- CCW
 - B- Lock Nut- CW
- Shallow A-clockwise
- Deep A – counter clockwise
- Re-tighten - A & B

