



# 5 Using your STATIM

## 5.2.2. Wrapped Cycle (STATIM 2000 G4)



The Wrapped Cycle is used to sterilize up to 1.0 kg (2.2 lbs) of solid and hollow metal instruments which have been sealed in paper / paper, or paper / plastic autoclave bags. Dental handpieces may be sterilized in this cycle.

To select the Wrapped Cycle, press the Wrapped Cycle button, then press the START button.

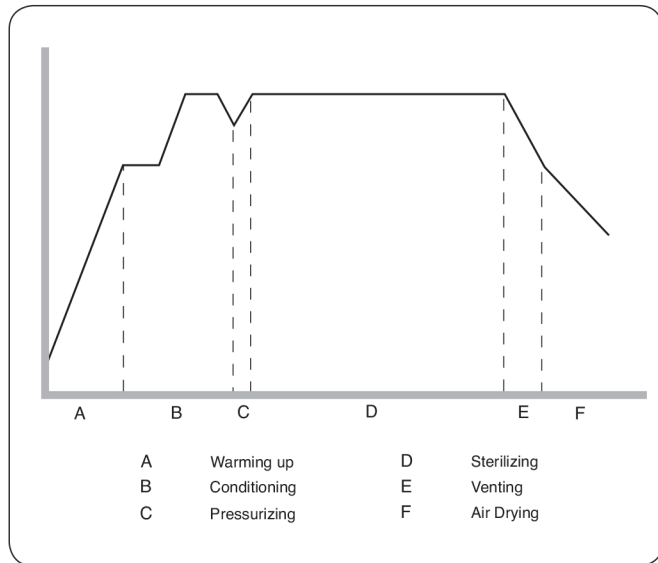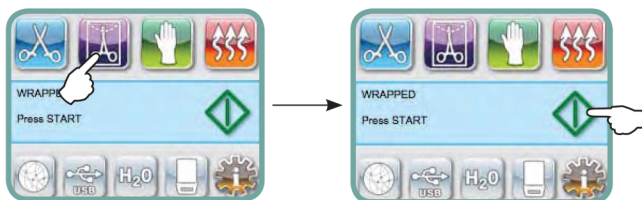

The sterilization temperature in the cassette is 134°C (273°F) and the holding time is 10 minutes. See Section STATIM 2000 G4— Cassette, and Section Preparing and Loading Instruments before running this cycle.





If wrapped instruments are intended for storage, the wraps must be dry when the cassette is removed from the unit and opened.

## 5.2.3. Rubber and Plastics Cycle (STATIM 2000 G4)



The Rubber and Plastics Cycle is used to sterilize up to 0.4 kg (0.9 lbs) of solid unwrapped instruments constructed of metal or the materials listed in Section Preparing and Loading Instruments.

To select the Rubber and Plastics Cycle, press the Rubber and Plastics Cycle button, then press the START button.

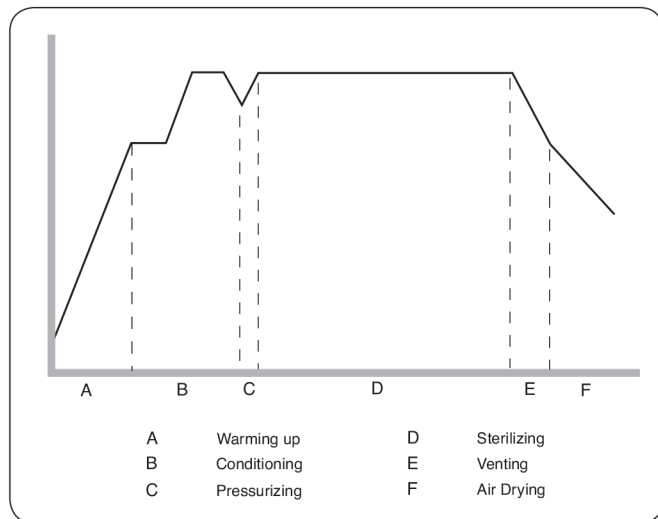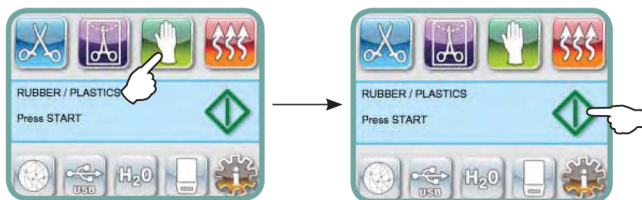

The sterilization temperature in the cassette is 121°C (250°F) and the holding time is 15 minutes.