


How to Use the EAZI-BREED™ CIDR® Cattle Insert

For best results, follow these simple steps:

1. Wear protective gloves whenever handling the EAZI-BREED CIDR Insert.
2. Prepare a container of clean water with disinfectant solution. Wash the applicator with the water between uses.
3. Fit the body of the insert into the applicator with the tail along the slot. The two wings will be pushed together, protruding about one inch above the top of the applicator.
4. Apply a generous amount of lubricant to the tip of the insert.
5. Shift the animal's tail to one side, and clean the vulva.
6. Make sure the tail of the EAZI-BREED CIDR Insert is on the underside of the applicator, curling down, to ensure that the tail will be hidden from curious penmates.
7. Open the lips of the vulva and insert the applicator at a slightly upward angle, moving forward over the pelvic bone until it meets resistance.
8. Dispense the insert from the applicator by depressing the plunger, then slowly withdraw the applicator.
9. To prevent removal by curious penmates, you may want to clip the tail of the insert so that 2-1/2" protrude from the vulva.
10. To withdraw the insert seven days later, simply give the tail a gentle but firm pull to release the insert.
11. Dispose of used inserts in a sealed, plastic container in accordance with applicable local, state and federal regulations.

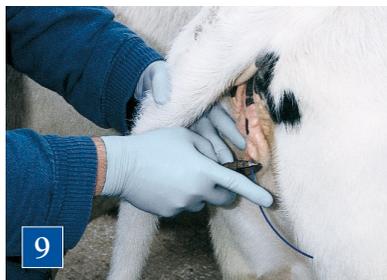

Breed More Cattle in Less Time

PHARMACIA Animal Health

www.cidr.com

