

VeinViewer Guide

- No need to alter your technique.
- No calibration is ever required.
- Batteries last for 2 hours and take 4 hours to charge. Battery must click into place.
- You may use the AC adapter when needed. Battery should be installed, but won't charge.
- Hold close to patient and move away until the high-resolution text/image is in focus. The focal distance/angle is 12"/perpendicular with a 1" window of focus.
- The f1 button is for resizing. Most remain at full-size. Smaller can be useful for pediatric patients or to avoid blood vessels.
- The f2 button has 2 brightness settings. Brighter can be helpful when in sunlight or near a window, etc.
- The bottom button is for fine detail and background. Most tend to prefer FINE with a green vein.
- Images may be captured and uploaded.
- Wipe down to clean.
- Clean lens area with a lint-free cloth as/if needed.
- Remains in the most recently used mode when powered down.

Cory Steiner (800) 366-8051
eortho@aol.com (215) 791-0674 cell
eortho.com (215) 689-1558 fax

CAREstreamAmerica.com