



Explore. Customize. Communicate Getting Started With Saltillo Chat Software

Not all of the features and functions listed on this handout are available on all device models

HARDWARE OVERVIEW



NOVA chat

- Built on Android Operating System
- 4 Sizes: 5, 8, 10, 12 (inches)
- Amplifier with a Bluetooth connection
- Built-in camera for editing
- 4 color boots included (green, blue, gray, purple)



Chat Fusion

- Built on Android Operating System
- 2 sizes: 8 and 10 inch
- Integrated speaker
- Built-in camera for editing
- ChatPoint available as an accessory

CHARGING: It is recommended to charge the device every night



NOVA chat

connect both cords, 1 into tablet, 1 into amp



Chat Fusion

includes both standard (1 cord) and wireless charger

WHAT'S INCLUDED?

Consultant Support	Technical Phone Support	Live Chat	Warranty	Accessories
	800-382-8622	www.saltillo.com/support	1-3 years	stylus, strap, bag, mounting plate

Models: Standard, Plus, Dedicated Plus. Models and devices vary by price, warranty, and accessories

ACCESS METHODS

Device	Direct Selection	Keyguards (standard & custom)	Scanning*	Headpointing with Chat Point
NOVA chat	X	X	X	
Chat Fusion	X	X	X	X

Scanning software: switches (1, 2, screen), auditory prompts, visual effects

CLEANING THE DEVICE

- Turn the device off; wipe with a soft, lint-free cloth slightly dampened with water, dry with another soft cloth
- To **disinfect** the case, moisten cloth in a solution of 1/4 cup vinegar and 1 cup water. Do not use on screen!
Do not spray or splash liquid directly on the device

SOFTWARE OVERVIEW

- Symbols: Symbol Stix (SS-default) & PCS (\$)
- Speech Synthesizers: Ivona (default) & Acapela (\$)
- Writing Enhancement Features
- Word Finder

CHAT EDITOR: FREE companion software for PC computer

How to get it

- Install the included CD or
- Download from <https://saltillo.com/products#chat-editor>

What to do with it

- Backup customized files
- Customize without taking the device away
- Create materials using Capture tool
- Learn vocabulary



THE DEVICE: NOVA CHAT & CHAT FUSION

Location of power button, volume control, and operational icons (e.g., back, menu) varies by device.

Power Button

- **Turn on:** press and hold until starts up
- **Sleep Mode:** quickly press to sleep/wake
- **Turn off:** press and hold – select power off – OK

Volume Control: adjust as needed;

Note: loudest when bluetooth speaker is charged

Chat Menu Bar

- **Edit Mode:** edit and hide mode
- **Settings:** choose voice, set lock, etc.
- **Library:** list of all vocabulary files
- **Help:** software version & help index
- **Word Finder** (if enabled in settings)
- **Lock** (if enabled in settings)



- **X:** Tap – delete last word
Hold – clear display
Double tap – stop speech

Back – go back a screen

Home – wake up screen

Menu – bring up Menu Bar



SOFTWARE UPDATES: required to provide the best device possible. These fix bugs and add new features.

- **Menu** ⇒ **Help** ⇒ **About**
- Learn how to update your device with a video tutorial <http://saltillo.com/training/>
- Wi-Fi updates available for unlocked/non-dedicated devices only.

VOCABULARY OPTIONS

Exploring Vocabulary Files

- **Menu** ⇒ **Library**
- Select vocabulary file ⇒ **Open** ⇒ Explore file

SS= Symbol Stix
PCS= Picture Communication Symbols

Type of Message Formulation


Vocabulary File Name	Type of Message Formulation				
	Spelling	Sentence	Phrase	Single Words	
				Nouns	Core
Communication Journey: Aphasia	Includes features & vocabulary to support people with aphasia, customization necessary				
Basic Scan		★	✓	✓	
Intermediate Scan		★	✓	✓	✓
Essence	★	✓	✓		
Spelling	★				
Teclado	★				
MyQuickChat	✓	★	✓	✓	
4 Basic			★	✓	
Vocab PC			★	✓	
Vocab PC Spanish			★		
MultiChat 15- all configurations	✓	✓	★	★	✓
• Student					
• Adolescent					
• Adult					
• Bilingual					
• Spanish					
WordPower: all configurations	✓	✓	✓	★	★
• 20 Simply			★		
• 25 Touch & Scan			★		
• 42 Basic					
• 42					
• 48					
• 48 Español Basico					
• 48 Español					
• 60 Basic					
• 60 Español Basico					
• 80					
• 108	★				
• 108 with keyboard	★				
• myCore	✓	✓	✓	★	★
Sin sintaxis 4 X4 English & Spanish		✓	✓	★	✓
Sintaxis 4 X5 English & Spanish	✓	✓	★	★	★

★: primary method of message formulation typically used

✓: additional methods of message formulation available

– All vocabulary files can be modified, and an AAC user may utilize a vocabulary file in a different manner –

NOTES ABOUT VOCABULARY:

CUSTOMIZING SETTINGS	
- Additional instructions are found in the User's Manual, Getting Started Guide, and Help Index on Device -	
Adjust Timing	Menu ⇒ Settings ⇒ Input ⇒ Timing ⇒ Select Desired Button Timing (described below) ⇒ Enable ⇒ Set Time <ul style="list-style-type: none"> ○ Acceptance Time: length of time to hold a button down before it activates ○ Release Time: length of time required to wait between button activations ○ Activate on release: if checked, button will not activate until finger is lifted off button. There is no timing adjusting required for this setting.
Stop Screen Rotation	Menu ⇒ Settings ⇒ Input ⇒ Tilt ⇒ Action ⇒ No Action
Change Voice	Menu ⇒ Settings ⇒ Speech Output ⇒ Voice ⇒ Voice Settings <ul style="list-style-type: none"> ○ Select the Voice dropdown ⇒ Select a voice ○ Adjust as needed (rate for Ivona and rate & pitch for Acapela) ⇒ Select Save
Change Pronunciations	Menu ⇒ Settings ⇒ Speech Output ⇒ Voice ⇒ Pronunciations <ul style="list-style-type: none"> ○ A list of default words appears. To add a word select Menu ⇒ New ○ Type the word as it should be spelled - then in the pronounce as box, spell phonetically- select Pronounce It to hear it and then Save when satisfied
Add Pause Between Words	Menu ⇒ Settings ⇒ Speech Output ⇒ Voice ⇒ Pause Between Words <ul style="list-style-type: none"> ○ Enter time (in seconds) you want between each spoken word. ○ Select OK.
Change Font Size in SDB	Menu ⇒ Settings ⇒ Style ⇒ Font ⇒ Font Size
Align Keypad	Menu ⇒ Settings ⇒ Style ⇒ Font ⇒ Font Size ⇒ Adjust as needed Refer to instructions included with keypad for recommended font size and settings. These may vary per device and vocabulary file.
Add Word Finder to Menu Bar	Menu ⇒ Settings ⇒ Word Finder ⇒ Show Menu Item
Use Word Finder in Vocabulary	There are two ways to use Word Finder. <ol style="list-style-type: none"> 1) Menu ⇒ Word Finder ⇒ Type the word. Choose Done. Pathway(s) will appear. 2) Go to Keyboard in the vocabulary. Type the word. Choose Find Word on the keyboard Pathways(s) will appear in SDB.
Set Menu Lock Password	Menu ⇒ Settings ⇒ Menu Lock ⇒ Enable Menu Lock. <ul style="list-style-type: none"> ○ Enter and confirm the password. It is case sensitive ○ Choose Save.
Lock/Unlock Menu Bar	Menu ⇒ Settings ⇒ Lock. Access to menu bar is now "locked". Menu ⇒ Settings ⇒Unlock ⇒ Enter the password ⇒ OK. If you forget the password, enter BOSCO to override it, then re-set (above).
Check Battery Status	Menu ⇒ Settings ⇒System > About Device  Vocabulary file may also have a button to indicate battery status.
Adjust Screen Timeout	Menu ⇒ Settings ⇒ System ⇒ Display Settings ⇒ Screen Timeout ⇒ select time (e.g., 10)
Enable Touch Wake-up (Chat Fusion only)	Menu ⇒ Settings ⇒System ⇒ Display Settings ⇒ Touch Wake-up
Turn Data Logging On/Off	Menu ⇒ Settings ⇒Data Logging ⇒ Enable Data Logging See Device User Guide or video tutorials http://saltillo.com/training for more info
Turn Kiosk On/Off (unlocked devices only)	Menu ⇒ Settings ⇒ System ⇒ Kiosk ⇒ Enable Kiosk Mode <ul style="list-style-type: none"> ○ If you want to set a password, do so first before enabling Kiosk. The password for Kiosk, can be the same or different from the Menu Lock Password. ○ If you forget the password, enter bazinga to overwrite it.
NOTES ABOUT SETTINGS	

CUSTOMIZING VOCABULARY

Make a Plan - Where to Start?
 o Suggested places to start: social page, people, about me, and favorites (food, drinks, toys, books)
 o See what's working/not working, then go back and customize

Copy a Default Vocabulary File for Customizing
 In order to edit a vocabulary file, you must make a copy.
Menu ⇒ Library
 o Tap on the file you want to copy, select **Duplicate**.
 o Type a name for the file. It is recommended to leave the vocabulary file name (e.g., MultiChat 15_Dave). **Save**
 o Tap on the newly created file. **Open**.

Change Button Label or Message
Menu ⇒ Edit Mode
 o Press and hold on the button you want to edit
 o Choose **Edit Button**. The button properties appear
 o Edit the label and/or message by tapping in the box and typing.
 o **Label**: the words on the button
 o **Message**: the words spoken by the device
 o When finished, choose **Save**.

Change Image from Picture Library
Menu ⇒ Edit Mode
 o Press and hold on the button you want to edit
 o Select **Edit Button**. The button properties appear
 o Select **Find**. Spell item to find. Select **Search**
 o Drag the list up and down. Tap on the desired image. Select **Save**

Take a Picture for Button Image
Menu ⇒ Edit Mode
 o Press and hold on the button you want to edit
 o Select **Edit Button**. The button properties appear
 o Select **Camera**. Take the picture in horizontal view. Select **Save**.
 o Name the image. Select **Save**.
 o If needed, add a label and message to the button. Select **Save**.

Change a Button's Style
Menu ⇒ Edit Mode
 o Press and hold on the button you want to edit
 o Select **Edit Button**. The button properties appear
 o Select the **Style** tab on top. The style properties appear
 o Make the changes you want.
 o **Highlighting** the button: Select **Border**. Select the **color**. Increase **Border Width**.
 o When finished, select **Save**.



Copy and Paste a Button
Menu ⇒ Edit Mode
 o Press and hold on the button you want to copy. Select **Copy Button**.
 o Press and hold on the new location. Select **Paste Button**.

Copy and Paste a Button's Style
Menu ⇒ Edit Mode
 o Press and hold on the button, whose style (e.g., border, background color) you want to copy. Select **Copy Button**.
 o Press and hold on the desired button. Select **Paste Button Style**.

Hide/Show Buttons
Menu ⇒ Edit Mode ⇒ Hide Mode (may have to select **more ⇒ Hide Mode**)
 You must hide each page at a time. Once in hide mode, you have three options:
 o Select buttons one by one to either hide or show them.
 o Select **Menu ⇒ Hide All** to hide all buttons on the page.
 o Select **Menu ⇒ Show All** to show all buttons on the page.
 To exit Hide Mode, select **Menu**, uncheck **Hide Mode** or select **Edit Mode**.

NOTES ABOUT CUSTOMIZING:

TRANSFERRING VOCABULARY FILES TO/FROM CHAT EDITOR

- o Start **Chat Editor** on the computer (must already be downloaded)
- o Make sure device is not in Edit Mode (Menu ⇒ Uncheck Edit Mode)
- o Attach device to the computer via included USB cable. Device will enter Transfer Mode.

Transfer files **from Device** to Chat Editor

- o Within **Chat Editor**, choose **Library** ⇒ **Transfer** ⇒ **To/From Device**
- o Check the vocabulary file(s) (on the right side of the screen) you wish to transfer to the Editor.
- o Choose the **To Editor** button
- o The file name(s) will be listed in the **Editor Files** once complete. Choose **Done**.
- o Safely remove device from computer. Unplug USB cord from Device.

To view the file(s) in **Chat Editor**, choose **Library** ⇒ **My Resources** ⇒ Choose the Vocabulary ⇒ **Open**

Transfer files **from Chat Editor** to Device

- o Within **Chat Editor**, choose **Library** ⇒ **Transfer** ⇒ **To/From Device**
- o Check the vocabulary file(s) (on the left side of the screen) you wish to transfer to the Device.
- o Choose the **To Device** button
- o The file name(s) will be listed in the **Device Files** once complete. Choose **Done**.
- o Safely remove device from computer. Unplug USB cord from Device.

To view the file(s) on the **device**, choose **Menu** ⇒ **Library** ⇒ Choose the Vocabulary ⇒ **Open**

New Option for **Backup/ Transfer** via **USB**

You will need a flash drive with a micro-USB connector. The USB flash drive provided by Saltillo has a micro-USB connector on one end that plugs into your device and a USB 3.0 connector on the other end that plugs into a computer.

1. Connect the USB drive to the USB port on your device.

Note: If the device's file manager opens, use the device's **Back** arrow ↶ to close it.

2. On the device, select **Menu** > **Library** > **Menu** > **Import/Export**.

Note: A different file management screen may open. Depending on your device, select or navigate to the USB drive, choose the folder you want, and choose **Select** or **OK**.

3. A window will list the customized files under Device on the left. Any files already on the USB drive are listed under Backup Folder on the right.

To transfer files from your device to the USB drive:

- Select each file on the device that you want to copy to the USB drive.
- Select **To Backup**, which is now highlighted. The files will be copied to the USB drive. When the files finish copying, they will be listed under Backup Folder.

To transfer files from the USB drive to your device:


- Select each file on the USB drive that you want to copy to the device.
- Select **To Device**, which is now highlighted. The file(s) will be copied to the device. When the files finish copying, they will be listed under Device.

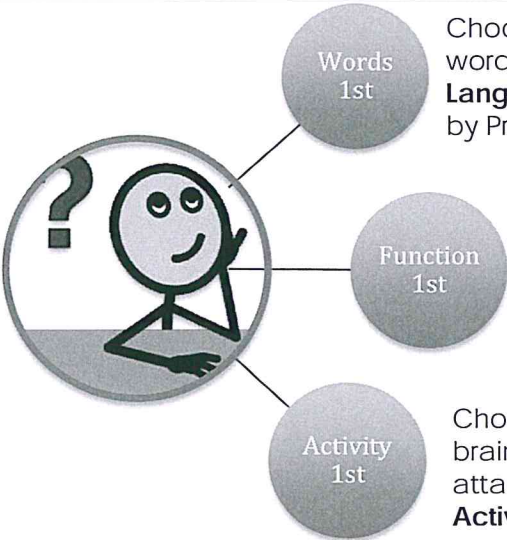
4. When you finish transferring files, disconnect the USB drive from your device and store it in a safe location and/or copy the files from the USB drive to a computer.

ADDITIONAL NOTES




GETTING STARTED-COMMUNICATE: Bolded resources available at <http://saltillo.com/implementation/chat/slp>

Communication Partners	NOTES
<ul style="list-style-type: none"> Communication partners are part of everyone's environment and social network (Blackstone, 2003) "8 of 10 most frequently reported reasons for device abandonment were related to partner training and support issues" (Johnson, Inglebret, Jones, & Ray, 2006). 	

Modeling: Aided Language Stimulation	NOTES
 <ul style="list-style-type: none"> "<u>Speak AAC to Teach AAC</u>" (Rachael Langley) Communicate on device as you talk with your voice. "Aided Language Stimulation Explained" by Chris Bugaj, available on YouTube Learn client's vocabulary <ul style="list-style-type: none"> Practice sentences Guided video review of vocabulary files: https://saltillo.com/videos/#category-2 What can I use to model? Client's device, TouchChat, low tech version, Chat Editor 	

Where to begin?	NOTES
 <p>Choose words from core or development word list (e.g., 15 Early Generative Language Words or A Year of Core Words by PrAACtical AAC).</p> <p>Think about WHY client may want to communicate (e.g., asking, directing, commenting, requesting). Brainstorm vocabulary based on those reasons.</p> <p>Choose a motivating activity/item, and then brainstorm vocabulary around that. Use the attached worksheet Choosing vocabulary Activity First.</p>	

SALTILLO RESOURCES

<p>Make a Low-Tech Communication Book https://saltillo.com/chatcorner/content/29</p> <p>Join our Newsletter https://saltillo.com/newsletter</p> <p>Attend a Webinar http://saltillo.com/training</p> <p>Watch Client and How-To Videos http://youtube.com/saltillocorp</p> <p>Learn about Realize Language & Data Logging www.realizelanguage.com</p>	<p>Learn about TouchChat http://touchchatapp.com/</p> <p>Find More Implementation Ideas www.practicalaac.org www.aaclanguagelab.com www.inmaninnovations.com</p> <p>Get Support 1-800-382-8622 http://saltillo.com/support</p> <p>Connect with Us   </p>
---	---

Choosing CORE Vocabulary: "Activity-First" Approach

