


## POWER TO RESTORE WITHOUT LEAVING A TRACE


### PREPARATION GUIDE

Preparation of **genex** takes only a few careful moments. Use only the mixing solution provided. Do not add any additional substances to the paste. All times stated are measured from the start of the timer.


### genex

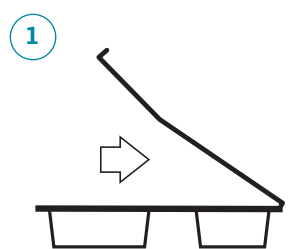


1 Open pack and remove all components using an aseptic, non-touch technique

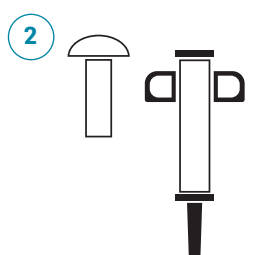


2 Remove plunger from syringe and place to side

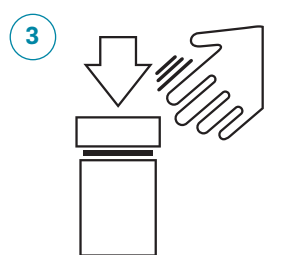


3 Gently tap pot, remove lid and empty powder into the sterile mixing bowl

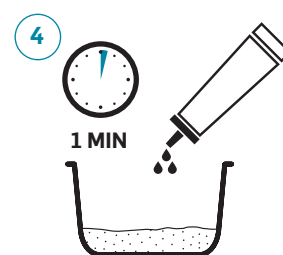


4 Add pre-measured sterile solution to powder. Allow to stand (soak out) for 1 minute

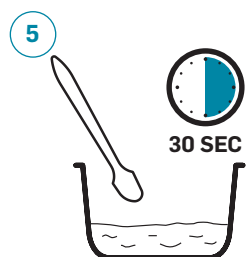


5 Mix thoroughly until paste is formed (approx. 30 seconds). Do not over mix

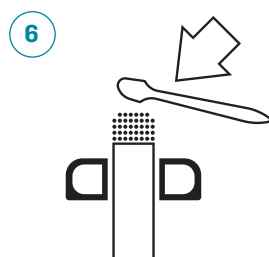

6 Use spatula to press paste into the syringe as quickly as possible

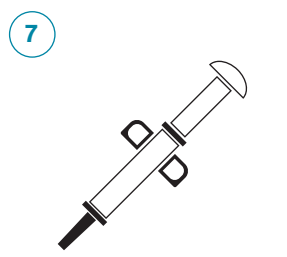

7 **genex** is now ready to use

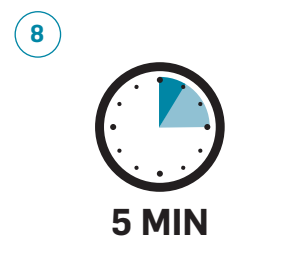

8 **genex** bone graft is injectable up to 5 minutes from the addition of the mixing solution. Allow to harden for at least 15 minutes and prior to closure

### SURGEON'S TIPS

1. Prepare site prior to commencing **genex** paste preparation
2. Use timer during preparation. All times stated are measured from the start of the timer
3. Use only the mixing solution provided. Do not add any additional substances to the paste

For indications, contraindications, warnings and precautions see Instructions for Use.

©2022, Biocomposites, **genex** and Power to Restore are trademarks/registered trademarks of Biocomposites Ltd. All rights reserved. No unauthorized copying, reproduction, distributing or re-publication is allowed unless prior written permission is granted by the owner, Biocomposites Ltd. Patents granted: EP I390086 B1, US 8632796, CN ZL02809194.9, US 8496955