



# Quick Profile™ Syphilis Antibody Test

(Chromatographic Lateral Flow)

Quick Profile™ Syphilis Antibody Test is a rapid immuno-chromatographic assay for the qualitative determination of *Treponema pallidum* antibodies.

Specimen:  
Human Whole Blood  
Serum or Plasma.



**Quick Profile™ Syphilis Antibody**  
One Step Test available as Test Card or Test Strip

2946 Scott Blvd.  
Santa Clara, CA 95054  
USA

Tel.: 1-408-855-0061  
info@LumiQuick.com

[www.lumiquick.com](http://www.lumiquick.com)

# Quick Profile™ Syphilis Antibody Test


## Procedure Test Card

Bring the kit components to room temperature.

Transfer 150µL of sample using the pipette provided or a micropipette.

Hold the pipette in a vertical position over the sample well and deliver 3 drops (120-150µL).

Read the results at 15 min.

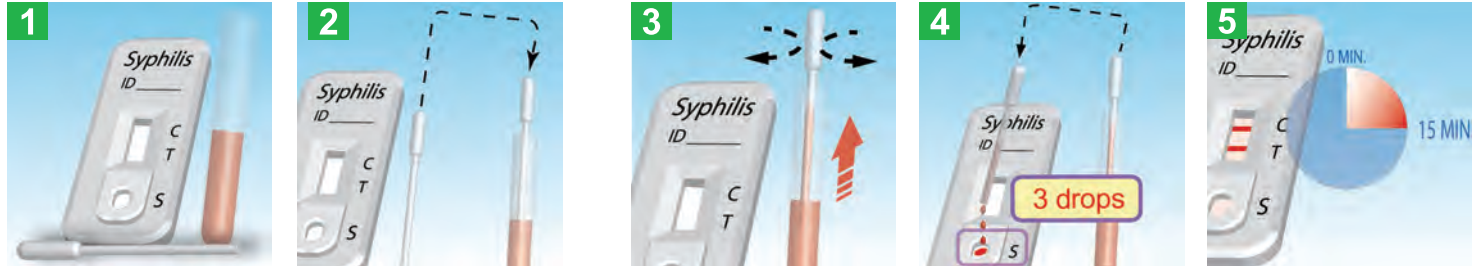

## Procedure Test Strip

Bring the kit components to room temperature.

Dip the strip into the sample with the arrow marked end down.

Keep the strip in touch with the sample until it reaches the membrane. Remove the strip and place it on a flat surface.

Read the results at 15 min.



## Test Result Appearance and Interpretation



## Product Performance

REFERENCE	QuickProfile™ Syphilis Antibody Test	
TEST RESULTS	POSITIVE	NEGATIVE
POSITIVE SAMPLE	349	7
NEGATIVE SAMPLE	5	709
TOTAL SAMPLES	354	716
ACCURACY	98.6%	99%

## Ordering Information

Catalog No	Description	Qty of test per Kit
71015	Quick Profile™ Syphilis Antibody Test Strip	50 tests
71016	Quick Profile™ Syphilis Antibody Test Card	25 tests



2946 Scott Blvd, Santa Clara CA 95054 USA  
 Tel.: +1(408)855-0061 - info@LumiQuick.com  
 www.LumiQuick.com