

# Getting Started with Evaclean Surface Wipes & PurOne Cleaner / Disinfection Tablets

## ▶ STEP 1

Fill 32 oz. bottle with warm water to fill line, or just below bottle neck.

## ▶ STEP 2

Add **one 13.1g tablet** to create a **4306 ppm** solution strength.

## ▶ STEP 3

Allow tablets to completely dissolve in water.

## ▶ STEP 4

Pour solution over the dry wipes in the container.

## ▶ STEP 5

Repeat steps 1-4 for a **total of 64 oz.** of **4306 ppm** solution in the wipes container.

## ▶ STEP 6

Replace lid and allow 5 minutes for solution to saturate wipes.

*Use PurOne 13.1 gram tablets to prepare a 4306 ppm solution.*

**x2**

TOTAL = 64oz.

**4306PPM**

