PHARMACOGENOMICS TESTING

Why Medication Management Matters

Phi Life Sciences



About Phi Life Sciences

Phi Life Sciences is a Molecular Diagnostic and Laboratory Service company.

Our goal is to commercialize novel technologies and develop products that empower physicians to deliver precision treatments and diagnoses by incorporating molecular tests that will improve patients' health.









Charleston, SC Location

645 Meeting Street, Suite 3

Charleston, SC 29403

CLIA#: 42D2114196

CAP#: 8079143





Rockville, MD Location

9430 Key West Ave #130 Rockville, MD 20850 CLIA #: 21D2158103




WHAT IS PHARMACOGENENETICS?

Pharmacogenetics is the clinical testing of the genetic variations (i.e. SNPs) that give rise to differing responses to drugs.

Genetic differences in metabolic pathways can affect an individual's response to drugs in terms of therapeutic effect as well as adverse events.





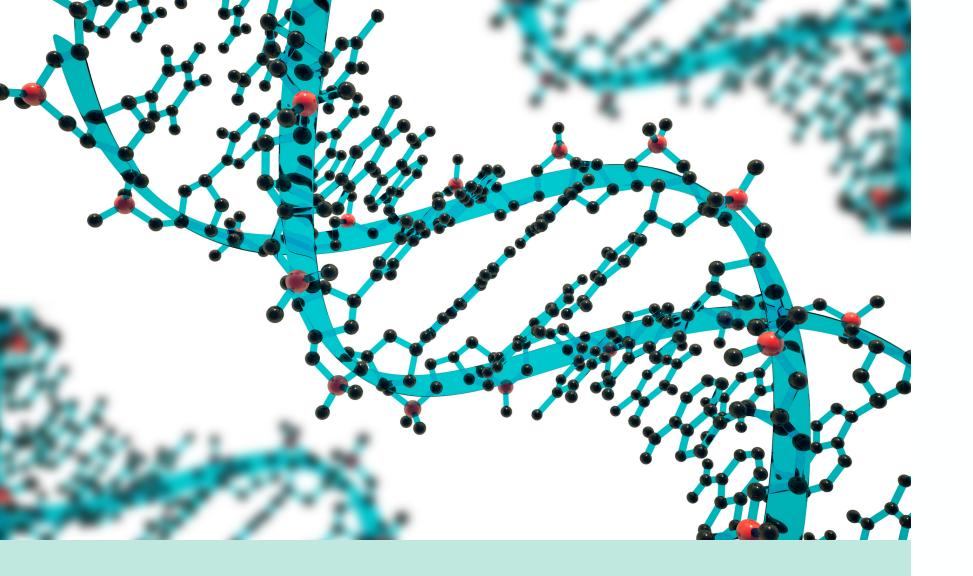

Genes and SNPs

All enzymes (like the CYP enzymes) are constructed by DNA "blueprints" called genes. Genes commonly have DNA code variations called "Single Nucleotide Polymorphism" or "SNPs" (pronounced "snips").

SNPs can be neutral, meaning they
have no effect on the enzyme
"blueprint" or they can have small to
profound effects on the enzyme
"blueprint" depending on the change.

Types of Metabolizers

POOR METABOLIZER

There are two variants
present. Typically one is on
the maternal-derived
chromosome and a second
variant on the paired
paternal-derived
chromosome. The alleles on
both chromosomes produce
"loss of function" enzymes.

INTERMEDIATE METABOLIZER

Having one gene variant which produces an enzyme with a significant loss of enzyme function.

or

Having one or two gene variants which produce only a modest loss of enzyme function.

Oľ

Having one significant loss of function variant and one modest loss of function variant.

NORMAL METABOLIZER

No variant alleles present on either of the patient's paired chromosomes.


Enzyme activity is normal.

ULTRA RAPID OR RAPID METABOLIZER

Having one or two
enzymes with significant
gain of enzyme function. If
there are two variants
present, this will often
cause more rapid
metabolism.



Personalized Medicine

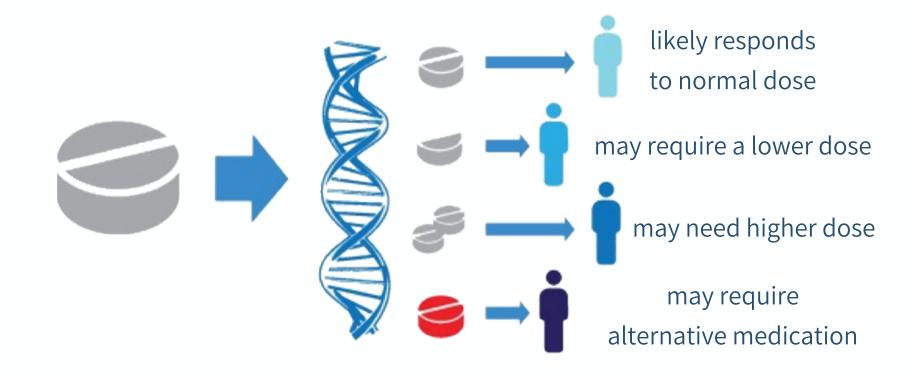

The one-size-fits-all approach to prescribing may not work for all patients.


Personalized prescribing helps you select the right drug and the right dose.



How is Pharmacogenetics Testing Used?

- To minimize adverse drug reactions (ADRs)
- To select the best medication the first time around
- To adjust the dose based on the genetic information, versus trial and error dosing



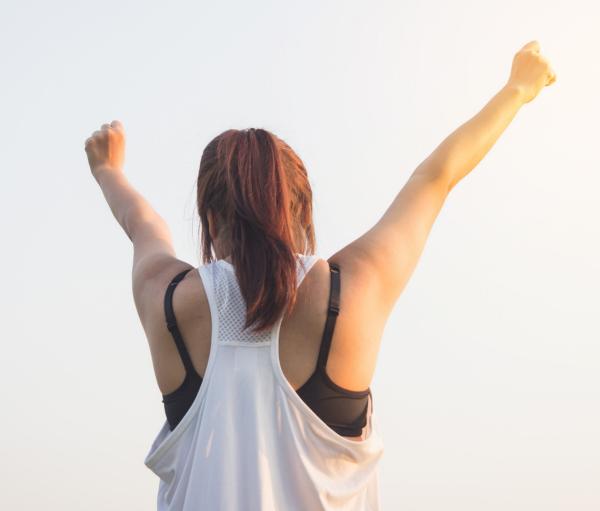



WHO NEEDS PHARMACOGENETIC TESTING?

Anyone...

- Taking medications
- Long Term Medication usage
- Not getting full benefit from medication(s)
- Taking warfarin (Coumadin),
 clopidogrel (Plavix) or blood
 thinners or beta-blockers

- SSRI or Psychotropic medication(s)
- Statins
- With cancer, psychiatric symptoms or other conditions where "trial and error" prescribing carries high risk
- With unexplained symptoms

Phi Life Sciences' PGx Tests



General Panel



Cardiology Panel




Psychiatry Panel



Pain Management

- ABCG2 421C>A
- ANKK1/DRD2 DRD2:Taq1A
- Apolipoprotein E ε2, ε4, (ε3 is reference)
- COMT Val158Met
- CYP1A2 *1F
- CYP2B6 *6, *9
- CYP2C19 *2, *3, *4, *4B, *5, *6, *7, *8, *9, *10, *17
- CYP2C9 *2, *3
- CYP2D6 *2, *3, *4, *4M, *6, *7, *8, *9, *10,
 *12, *17, *29, *35, *41, *5 (gene deletion),
 XN (gene duplication)
- CYP3A4 *1B, *22
- CYP3A5 *3, *3C, *6, *7
- DPYD rs1801265, rs1801267, rs3918290, rs55886062, rs67376798

- DRD2 -241A>G
- Factor II 20210G>A
- Factor V Leiden 1691G>A
- HTR2A -1438G>A, rs7997012
- HTR2C -759C>T, 114138144C>G
- ITGB3 176T>C
- LPA rs3798220, rs10455872
- MTHFR 1298A>C, 677C>T
- OPRM1 A118G
- SLCO1B1 521T>C
- TPMT *3A, *3B, *3C
- VKORC1 -1639G>A

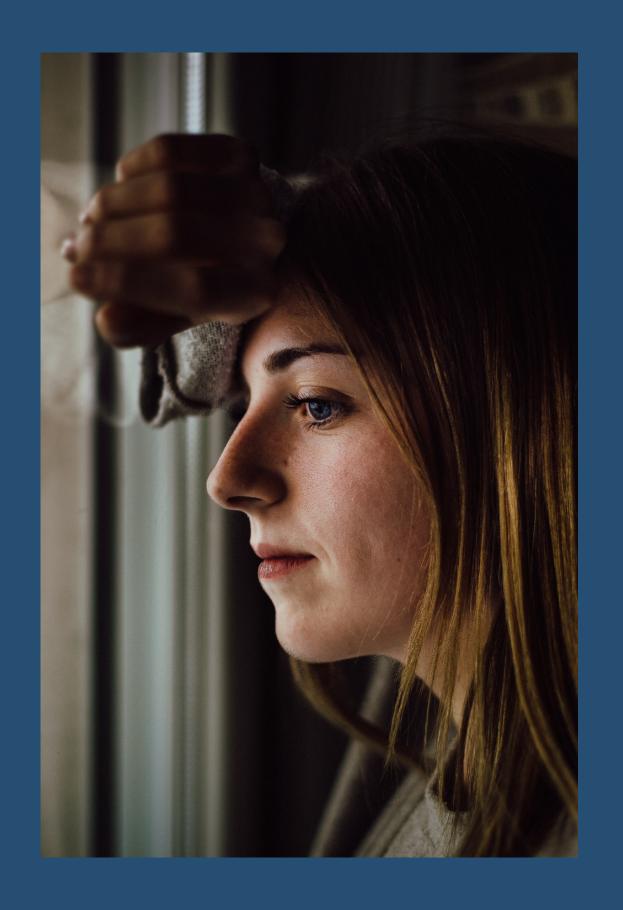

MEDICATION THERAPY MANAGEMENT

Genetic testing will identify the individual's ability to metabolize medication via the P-450 enzyme for medication efficacy.





How Often are Prescriptions Written?

Two-thirds of patient visits result in a prescription.

Specialist Providers write 2.3 prescriptions per visit.

Three **BILLION** outpatient prescriptions are written a year.

89.2% of Medicare patients take a prescription medications every day.

REDUCING AND PREVENTING ADVERSE DRUG EVENTS TO DECREASE HOSPITAL COSTS

Adverse drug events (ADEs) result in more than 770,000 injuries and deaths each year and cost up to \$5.6 million per hospital, depending on size. Many ADE injuries and resulting hospital costs can be reduced if hospitals make changes to their systems for preventing and detecting ADEs.





Why Learn About Adverse Drug Reactions?



- There are over **2,000,000** serious adverse drug reactions every year. (2,3,4)
- There are **100,000** deaths due to adverse drug reactions yearly. (2,3,4)
- Approximately 10% of hospital admissions are due to adverse drug reactions. (2,3,4)
- There are about **350,000** nursing home patients that have adverse drug reactions every year. (2,3,4)

- Adverse drug reactions increase exponentially when taking more than four prescription medications. (7,8,9)
- 46.1% of Medicare patients take more than five prescriptions every day. (7,8,9)
- 53.6% Medicare patients take prescription medications written by more than two or more providers. (7,8,9)
- Drug interactions are preventable and account for 3-5% of total preventable in-hospital adverse drug reactions. (10,11)
- Adverse events due to drug interactions are one of the important contributors to the number of emergency room visits and hospital admissions seen. (10,11)
- Adverse drug reactions cause 1 in 5 hospital patients to experience an injury or death. (4,5,6)
- Hospital patients with adverse drug reactions stay in the hospital twice as long, cost twice as much and have a twice as greater risk for mortality. (4,5,6)





\$136 Billion

The cost of adverse drug reactions nationwide is \$136,000,000,000 every year. (4,5,6)

The total cost of adverse drug reactions is larger than diabetic and cardiovascular care. (4,5,6)

Clinical Summary

- Access to the latest molecular technology
- Most comprehensive panels, designed with clinically significant variants
- 48-72 Hour turn-around-time after receipt to the laboratory
- Insurance verification
- Easy to interpret reporting
- Unmatched Clinical Support MD, PharmD,
 PhD, & Nursing
- Our HIPAA Compliant Portal: cloud based and easy to use







Collection Directions



- 1. Use one puritan double swab per patient.
- 2. Place barcode with patient DOB on the swab tube
- 3. Label the Requisition with the matching barcode and information as in #2. If the swab or requisition is missing information, then the test cannot be run.
- 4. Open the swab package at the handle end (NOT the collection tip end) and carefully peel the package away to within about an inch of the end. Uncap the dry transport tube and remove the swab from the package. Do not touch the collection swab from the package. Do not touch the collection tip of the swab.

- 5. Do not allow the tip of the swab to touch anything other than the inside of the patient's mouth and the inside of the packaging.
- 6. While holding the handle end of the swab, insert the tips along the inside of the cheek. Rotate the collection tips along the inside of the cheeks for 60 seconds. Pushing on the outside of the patient's cheek while rotating the swab will firmly press it between the gum and cheek and will enhance **cell collection**.
- 7. Hold the swab while it air dries for several seconds. **Do NOT blow on the swab.**
- 8. Break off top of Dry Transport tube and place swab into the tube then place it into Specimen Bag with Patient's requisition and documents.



Accepted Insurance Carriers





Federal Carriers:

- Medicare (Palmetto and Novitas MAC)
- Medicaid (ID, MT, OH, OR, SC, GA, CO)
- RR Medicare

Medicare Advantage Plans:


- United Healthcare
- Aetna
- Cigna
- Humana
- And most others with insurance verification

Commercial Insurance Carriers:

- Most commercial carriers accepted with insurance verification
- No BCBS



Test Requisition Form

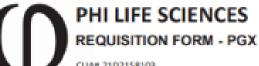

CLIA# 21D2158103 9430 KEY WEST AVE., #130 ROCKVILLE, MD 20850 CLIA# 42D2114196 AFFIX BAR CODE HERE

| 645 MEETING ST., SUITE 3, | CHARLESTON, SC 29403 | | | 741 | TIX DAN CO | DE HERE | | |
|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|--------------------------------------------------------------------------------------------|--------------------------------------------------------------------------------|-----------------------------------------------------------------------------|-----------------------------------------------------------------------------|--------------------------------------|
| TIENT INFORMATION | | | | | | | | |
| t Name | First Name | | M.I. | DOB (MM/I | DD/YYYY) | SSN | | |
| eet Address | | Ci | ty | | State | | Zip Cod | le |
| one Number | | Race | | Sex | Male C |) Female | □ 0tl | her |
| CILITY INFORMATION | | | | | | | | |
| ility Name | | | | Phone | | | | |
| dress | | Ci | ty | | State | | Zip Cod | le |
| SURANCE INFORMATION | | | | | | | | |
| urance Carrier | | | Carrier Pl | hone Numbe | г | | | |
| rier Address | | C | City State | | | Zip Code | | |
| Vorkers Comp/Auto/LOC 🖵 Self-Pay | Insurance (See Be | low) | | | | | | |
| mber ID | RX Group | RX BIN | | PC | N | | | |
| YSICIAN INFORMATION | | | | | | | | |
| ne | | Phone Nu | mber | NPI | Number | DI | EA Number | Г |
| ress | | C | ity | | State | Zip (| Code | |
| CIMEN INFORMATION | | | | | | | | |
| ection Date (MM/DD/YYYY) | | | | cation of Spe make sure all ap | | | | 1 |
| MENT OPTIONS | | | | | | | | |
| Medicare Advantage/Medicare Rep | lacement Plan 🔲 Me | dicare 🚨 | Medicaid C | TriCare 🚨 | Commercial | ☐ HSA | | |
| MPREHENSIVE | CARDIO & THROMBO | OSIS | PSYCHIATE | ₹Y | | PAIN M | ANAGEME | NT |
| ENERAL GENE PANEL AMMILIONES, Aprilopporters I, COME, CYPAN, CYP | CARDIAC GENE PANEL AGGS, ANG. GRAF, GALL, GALS, | | | IE PANEL 1930s, CHOBS, WORLS, 1381, HTICA, HTICC, TH | | CODMIT, OFFICE | ANAGEMENT (I, 09986, 09908, 0 I, 098M, VIOROS | |
| BCG2 APOLIPOPROTEINE CC NKK1/DRD2 CCOMT | P1A2 © CYP2C19 © CY P286 © CYP2C9 © CY | | | DIFS C | | | OPRM1 C | |
| DICAL HISTORY | | | | | | | | |
| 10 Codes: | | | Ances | try: | | | | |
| TENT INFORMED CONSENT | | | | | | | | |
| IENT PAYMENT OPTIONS: SURANCE: Attach a copy of front & back of i | nsurance card DINVOICE | PRACTICE / IN: | STITUTIONAL BILL | L / FACILITY BILL | ☐ Lab will co | ntact you for | card informat | tion |
| ort actinowledgment and authorization for insurant offits to be guid directly to Phi Life Sciences and auth any business associate of the insurer (TPR, TPR, etc. porsible for any amounts Phi Life Sciences bills direct or my policy. I understand that I am legally respons? | ce billing and report release: If II horize Phi Life Sciences to release. .). I authorize Phi Life Sciences to thy to me, including amounts tha | have provided m e medical inform o be my Designal at my insurer det | ly insurance information concerning in ted Representative termines are my re- | ation for direct inc ny tecting, includin for purposes of ap iponsibility after o | urance/3rd party g upon request m pealing any denic alculating deduct | billing: I hereb ny genetic testi- al of health ben bles, co-payme | y authorise my ng results, to m efits. I understa ents and co-insu | insurance y incurer and that I |
| ENT SIGNATURE | - | | | | DATE | - | | |
| NFIRMATION OF INFORMED CONSE | ENT AND MEDICAL NE | CESSITY | | | | | | |
| med Concert and Statement of Medical Recessity: mation has been supplied about genetic testing are satient opt-out for the use of their sample for resea | d that an appropriate Phi Life Sci | iences informal e | | | | | | |
| SICIAN SIGNATURE: | | | | DATE: | | | | |
| SICIAN SIGNATURE: | | | | FOLLOW U | IP DATE: | | | |

General PGx Results





PATIENT INFORMATION

NAME: Patient, Patient ACC #: PHI Sample DOB: 01/01/2001

SEX: Female

SPECIMEN DETAILS

SPECIMEN TYPE: Buccal Swab COLLECTION DATE: 7/27/2017 REPORT DATE: 8/2/2017

ORDERED BY

General Gene Panel

Test Details

| Gene | Genotype | Phenotype | Clinical Consequences |
|------------------------------|---------------------------|---------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| ABCG2 | 421C>A C/C | Normal Function | Consistent with a normal ABCG2 transporter function. The patient's risk for statin-induced adverse events is normal. |
| ANKK1/DRD2 | DRD2:Taq1A G/G | Unaltered DRD2 function | Consistent with a normal dopamine receptor D2 function. |
| Apolipoprotein E | ε3/ε3 | Normal APOE function | Not associated with type III hyperlipoproteinemia - No increased risk of cardiovascular disease |
| COMT | Val158Met A/G | Intermediate COMT Activity | Consistent with a reduced catechol O-methyltransferase (COMT) function. |
| CYP1A2 | *1F/*1F | Normal Metabolizer - Higher Inducibility | Consistent with a typical CYP1A2 activity in absence of inducing substances. Rapid Metabolism occurs in presence of inducers such as barbiturates, cruciferous vegetables, carbamazepine, rifampin and smoking. |
| CYP2B6 | *1/*1 | Normal Metabolizer | Consistent with a typical CYP2B6 activity. This test did not identify risks for side effects or loss of efficacy with drug substrates. |
| CYP2C19 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C19 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2C9 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C9 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2D6 | *4/*41 | Intermediate Metabolizer | Consistent with moderate to substantial deficiency in CYP2D6 activity. Increased risk for side effects or loss of efficacy with drug substrates. |
| CYP3A4 | *1/*1B | Normal Metabolizer | Consistent with a typical CYP3A4 activity. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| СҮРЗА5 | *3/*3 | Poor Metabolizer | Consistent with a poor CYP3A5 activity. This phenotype is the most common in the general population. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| DPYD | *1/*1 | Normal Metabolizer | Consistent with a typical DPD activity and a typical risk of side effects with conventional doses of fluoropyridines. |
| DRD2 | -241A>G T/C | Heterozygous for rs1799978 C Allele | Associated with a favorable response to Risperidone. |
| Factor II Factor V Leiden | 20210G>A GG 1691G>A GG | No Increased Risk of Thrombosis | The patient's risk of thrombosis is not increased (average risk of dotting is about 1 in 1000 for anyone in a year). However, because this test cannot find all of the inherited reasons for abnormal clotting, other factors may affect this risk assessment. |
| HTR2A | -1438G>A C/T | Heterozygous for the T Allele (rs6311) | The patient carries one copy of the variant allele at rs6311 which may be associated with greater serotonin 2A receptor gene expression. |
| HTR2A | rs7997012 A/G | Heterozygous for the A allele (rs7997012) | Reduced response to citalopram and escitalopram |
| HTR2C | 2565G>C G/C | Heterozygous for the C allele (rs1414334) | This genotype is associated with risperidone- and clozapine-induced metabolic syndrome. |
| HTR2C | -759C>T C/T | Heterozygous for the C allele (rs3813929) | Consistent with altered satiety signaling mediated by the serotonin receptor 2C (HTR2C). Increased incidence of metabolic side effects (weight gain, hyperglycemia, hyperlipidemia) with atypical antipsychotic medications. |

| ITGB3 | 176T>C T/T | Normal Platelet Reactivity | The patient does not carry the 176T>C mutation of the integrin $\beta 3$ gene. Unless other genetic or circumstantial risk factors are present, he is not expected to have an increased platelet reactivity. |
|---------|---------------------------------|------------------------------------------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| LPA | rs10455872 A/A rs3798220 T/T | No increased risk of cardiovascular disease | The patient is a non carrier of the risk alleles in LPA gene for both the variants (rs3798220 and rs10455872). |
| MTHFR | 677C>T CT | Reduced MTHFR Activity | The patient carries one MTHFR C677T mutation (heterozygous) and the patient's MTHFR activity is reduced slightly. This is not associated with an increased risk of hyperhomocysteinemia. |
| MTHFR | 1298A > C AC 677C > T CT | No Increased Risk of Hyperhomocysteinemia | The patient's reduced MTHFR activity is not a risk factor for hyperhomocysteinemia. Unless other risk factors are present, the patient is not expected to have an increased risk for venous thromboembolism (VTE). |
| OPRM1 | A118G A/A | Normal OPRM1 Function | Consistent with a normal OPRM1 receptor signaling efficiency induced by exogenous opioids. This is associated with a good analgesia following standard opioid doses and a poor response to naltrexone. |
| SLCO1B1 | 521T>C T/T | Normal Function | Consistent with a typical SLCO1B1 transporter function. The patient's risk for statin-induced myopathy is not increased. |
| TPMT | *1/*1 | Normal Metabolizer | Consistent with a typical TPMT activity and a typical risk of side effects with conventional doses of thiopurines. |
| VKORC1 | -1639G>A G/G | Low Warfarin Sensitivity | VKORC1 is the site of action of warfarin. The patient may require an increase in warfarin dose. |

Alleles Tested: ABCG2 421C>A; ANKK1/DRD2 DRD2:Taq1A; Apolipoprotein E £2, £4, (£3 is reference); COMT Val158Met; CYP1A2 *1F; CYP2B6 *6, *9; CYP2C19 *2, *3, *4, *4B, *5, *6, *7, *8, *9, *10, *17; CYP2C9 *2, *3; CYP2D6 *2, *3, *4, *4M, *6, *7, *8, *9, *10, *12, *17, *29, *35, *41, *5 (gene deletion), XN (gene duplication); CYP3A4 *1B, *22; CYP3A5 *3, *3C, *6, *7; DPYD *2A, *9A, *9B, rs67376798 A, *13; DRD2 -241A>G; Factor II 20210G>A; Factor V Leiden 1691G>A; HTR2A -1438G>A, rs7997012; HTR2C -759C>T, 2565G>C; ITGB3 176T>C; LPA rs3798220, rs10455872; MTHFR 1298A>C, 677C>T; OPRM1 A118G; SLCO1B1 521T>C; TPMT *3A, *3B, *3C; VKORC1 -1639G>A

Psychiatry PGx Results







PATIENT INFORMATION

NAME: Patient, Patient ACC #: PHI Sample DOB: 01/01/2001

SEX: Female

SPECIMEN TYPE: Buccal Swab COLLECTION DATE: 7/27/2017 **RECEIVED DATE:** 7/28/2017 REPORT DATE: 8/2/2017

SPECIMEN DETAILS

Doctor, Doctor

ORDERED BY

Psych Gene Panel

Test Details

| Gene | Genotype | Phenotype | Clinical Consequences |
|------------|----------------|---------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| ANKK1/DRD2 | DRD2:Taq1A G/G | Unaltered DRD2 function | Consistent with a normal dopamine receptor D2 function. |
| COMT | Val158Met A/G | Intermediate COMT Activity | Consistent with a reduced catechol O-methyltransferase (COMT) function. |
| CYP1A2 | *1F/*1F | Normal Metabolizer - Higher Inducibility | Consistent with a typical CYP1A2 activity in absence of inducing substances. Rapid Metabolism occurs in presence of inducers such as barbiturates, cruciferous vegetables, carbamazepine, rifampin and smoking. |
| CYP2B6 | *1/*1 | Normal Metabolizer | Consistent with a typical CYP2B6 activity. This test did not identify risks for side effects or loss of efficacy with drug substrates. |
| CYP2C19 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C19 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2C9 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C9 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2D6 | *4/*41 | Intermediate Metabolizer | Consistent with moderate to substantial deficiency in CYP2D6 activity. Increased risk for side effects or loss of efficacy with drug substrates. |
| CYP3A4 | *1/*1B | Normal Metabolizer | Consistent with a typical CYP3A4 activity. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| CYP3A5 | *3/*3 | Poor Metabolizer | Consistent with a poor CYP3AS activity. This phenotype is the most common in the general population. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| DRD2 | -241A>G T/C | Heterozygous for rs1799978 C Allele | Associated with a favorable response to Risperidone. |
| HTR2A | -1438G>A C/T | Heterozygous for the T Allele (rs6311) | The patient carries one copy of the variant allele at rs6311 which may be associated with greater serotonin 2A receptor gene expression. |
| HTR2A | rs7997012 A/G | Heterozygous for the A allele (rs7997012) | Reduced response to citalopram and escitalopram |
| HTR2C | 2565G>C G/C | Heterozygous for the C allele (rs1414334) | This genotype is associated with risperidone- and dozapine-induced metabolic syndrome. |
| HTR2C | -759C>T C/T | Heterozygous for the C allele (rs3813929) | Consistent with altered satiety signaling mediated by the serotonin receptor 2C (HTR2C). Increased incidence of metabolic side effects (weight gain, hyperglycemia, hyperlipidemia) with atypical antipsychotic medications. |
| MTHFR | 677C>T CT | Reduced MTHFR Activity | The patient carries one MTHFR C677T mutation (heterozygous) and the patient's MTHFR activity is reduced slightly. This is not associated with an increased risk of hyperhomocysteinemia. |
| VKORC1 | -1639G>A G/G | Low Warfarin Sensitivity | VKORC1 is the site of action of warfarin. The patient may require an increase in warfarin dose. |

Alleles Tested: ANKK1/DRD2 DRD2Taq1A; COMT Val158Met; CYP1A2 *1F; CYP2B6 *6, *9; CYP2C19 *2, *3, *4, *4B, *5, *6, *7, *8, *9, *10, *17; CYP2C9 *2, *3; CYP2D6 *2, *3, *4, *4M, *6, *7, *8, *9, *10, *12, *17, *29, *35, *41, *5 (gene deletion), XN (gene duplication); CYP3A4 *1B, *22; CYP3A5 *3, *3C, *6, *7; DRD2 -241A>G; HTR2A -1438G>A, rs7997012; HTR2C -759C>T, 2565G>C; MTHFR 1298A>C, 677C>T; VKORC1 -1639G>A

Cardiology PGx Results







PATIENT INFORMATION

NAME: Patient, Patient
ACC #: PHI Sample
DOB: 01/01/2001

SEX: Female

RECEIVED DATE: REPORT DATE:

SPECIMEN DETAILS

SPECIMEN TYPE: Buccal Swab
COLLECTION DATE: 7/27/2017
RECEIVED DATE: 7/28/2017

ORDERED BY

Doctor, Doctor

Cardiac Gene Panel

Test Details

| Gene | Genotype | Phenotype | Clinical Consequences |
|------------------------------|---------------------------------|------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| ABCG2 | 421C>A C/C | Normal Function | Consistent with a normal ABCG2 transporter function. The patient's risk for statin-induced adverse events is normal. |
| Apolipoprotein E | ε3/ε3 | Normal APOE function | Not associated with type III hyperlipoproteinemia - No increased risk of cardiovascular disease |
| COMT | Val158Met A/G | Intermediate COMT Activity | Consistent with a reduced catechol O-methyltransferase (COMT) function. |
| CYP1A2 | *1F/*1F | Normal Metabolizer - Higher Inducibility | Consistent with a typical CYP1A2 activity in absence of inducing substances. Rapid Metabolism occurs in presence of inducers such as barbiturates, cruciferous vegetables, carbamazepine, rifampin and smoking. |
| CYP2C19 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C19 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2C9 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C9 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2D6 | *4/*41 | Intermediate Metabolizer | Consistent with moderate to substantial deficiency in CYP2D6 activity. Increased risk for side effects or loss of efficacy with drug substrates. |
| CYP3A4 | *1/*1B | Normal Metabolizer | Consistent with a typical CYP3A4 activity. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| СҮРЗА5 | *3/*3 | Poor Metabolizer | Consistent with a poor CYP3A5 activity. This phenotype is the most common in the general population. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| Factor II Factor V Leiden | 20210G>A GG 1691G>A GG | No Increased Risk of Thrombosis | The patient's risk of thrombosis is not increased (average risk of clotting is about 1 in 1000 for anyone in a year). However, because this test cannot find al of the inherited reasons for abnormal clotting, other factors may affect this risk assessment. |
| ITGB3 | 176T>C T/T | Normal Platelet Reactivity | The patient does not carry the 176T>C mutation of the integrin β3 gene. Unless other genetic or circumstantial risk factors are present, he is not expected to have an increased platelet reactivity. |
| LPA | rs10455872 A/A rs3798220 T/T | No increased risk of cardiovascular disease | The patient is a non carrier of the risk alleles in LPA gene for both the variants (rs3798220 and rs10455872). |
| MTHFR | 1298A>C AC 677C>T CT | No Increased Risk of Hyperhomocysteinemia | The patient's reduced MTHFR activity is not a risk factor for hyperhomocysteinemia. Unless other risk factors are present, the patient is not expected to have an increased risk for venous thromboembolism (VTE). |
| SLCO1B1 | 521T>C T/T | Normal Function | Consistent with a typical SLCO1B1 transporter function. The patient's risk for statin-induced myopathy is not increased. |
| VKORC1 | -1639G>A G/G | Low Warfarin Sensitivity | VKORC1 is the site of action of warfarin. The patient may require an increase in warfarin dose. |
| | | | |

Alleles Tested: ABCG2 421C>A; Apolipoprotein E £2, £4, (£3 is reference); COMT Val158Met; CYP1A2 *1F; CYP2C19 *2, *3, *4, *48, *5, *6, *7, *8, *9, *10, *17; CYP2C9 *2, *3; CYP2D6 *2, *3, *4, *4M, *6, *7, *8, *9, *10, *12, *17, *29, *35, *41, *5 (gene deletion), XN (gene duplication); CYP3A4 *1B, *22; CYP3A5 *3, *3C, *6, *7; Factor II 20210G>A; Factor V Leiden 1691G>A; ITGB3 176T>C; LPA rs3798220, rs10455872; MTHFR 1298A>C, 677C>T; SLCO1B1 521T>C; VKORC1 - 1639G>A

Pain Management PGx Results







PATIENT INFORMATION

SPECIMEN DETAILS

ORDERED BY

Doctor, Doctor

NAME: Patient, Patient DOB: 01/01/2001

SPECIMEN TYPE: Buccal Swab

RECEIVED DATE: 7/28/2017 REPORT DATE:

Pain Management Panel

Test Details

| Gene | Genotype | Phenotype | Clinical Consequences |
|------------------------------|---------------------------|---------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| COMT | Val158Met A/G | Intermediate COMT Activity | Consistent with a reduced catechol O-methyltransferase (COMT) function. |
| CYP1A2 | *1F/*1F | Normal Metabolizer - Higher Inducibility | Consistent with a typical CYP1A2 activity in absence of inducing substances. Rapid Metabolism occurs in presence of inducers such as barbiturates, cruciferous vegetables, carbamazepine, rifampin and smoking. |
| CYP2B6 | *1/*1 | Normal Metabolizer | Consistent with a typical CYP2B6 activity. This test did not identify risks for side effects or loss of efficacy with drug substrates. |
| CYP2C19 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C19 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2C9 | *1/*2 | Intermediate Metabolizer | Consistent with a moderate deficiency in CYP2C9 activity. Potential risk for side effects or loss of efficacy with drug substrates. |
| CYP2D6 | *4/*41 | Intermediate Metabolizer | Consistent with moderate to substantial deficiency in CYP2D6 activity. Increased risk for side effects or loss of efficacy with drug substrates. |
| CYP3A4 | *1/*1B | Normal Metabolizer | Consistent with a typical CYP3A4 activity. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| CYP3A5 | *3/*3 | Poor Metabolizer | Consistent with a poor CYP3A5 activity. This phenotype is the most common in the general population. Caution is advised when prescribing narrow therapeutic index drugs. Alternative drugs or dose adjustment may be required if CYP3A inhibitors or inducers are co-prescribed. |
| Factor II Factor V Leiden | 20210G>A GG 1691G>A GG | No Increased Risk of Thrombosis | The patient's risk of thrombosis is not increased (average risk of clotting is about 1 in 1000 for anyone in a year). However, because this test cannot find all of the inherited reasons for abnormal clotting, other factors may affect this risk assessment. |
| OPRM1 | A118G A/A | Normal OPRM1 Function | Consistent with a normal OPRM1 receptor signaling efficiency induced by exogenous opioids. This is associated with a good analgesia following standard opioid doses and a poor response to naltrexone. |
| VKORC1 | -1639G>A G/G | Low Warfarin Sensitivity | VKORC1 is the site of action of warfarin. The patient may require an increase in warfarin dose. |

Alleles Tested: COMT Val 158Met; CYP1A2 *1F; CYP2B6 *6, *9; CYP2C19 *2, *3, *4, *4B, *5, *6, *7, *8, *9, *10, *17; CYP2C9 *2, *3; CYP2D6 *2, *3, *4, *4M, *6, *7, *8, *9, *10, *12, *17, *29, *35, *41, *5 (gene deletion), XN (gene duplication); CYP3A4 *1B, *22; CYP3A5 *3, *3C, *6, *7; Factor II 20210G>A; Factor V Leiden 1691G>A; MTHFR 1298A>C, 677C>T; OPRM1 A118G; VKORC1 -1639G>A

Methodology

Caution: It is important to note that the combination of genetic metabolizer and non-genetic factors, such as, but not limited to: body size, age, inhibition caused by interference of drugs or food, liver function, and kidney function produce the overall response to a given drug dose.

Methods: This specimen was analyzed for gene mutations by real-time PCR (TaqMan SNP Genotyping, Thermo Fisher) developed by Phi Life Sciences, LLC. These assays were validated following the 1988 CLIA standards. Performance characteristics were validated by Phi Life Sciences Laboratory with analytical specificity and sensitivity of >99% for detection of the variants above.

Lab Disclaimer: The FDA has neither cleared nor approved these assays, nor is FDA pre-market review required. These tests are used for clinical purposes and should not be regarded as investigational or for research only. Diagnosis and treatment decisions are the sole responsibility of the practitioner and does not replace the need for clinical and therapeutic drug monitoring. Hence, the interpretation and commentary are provided to the practitioner for educational purposes only and should not be taken as diagnostic or treatment recommendations. For further assistance with interpretation of these results or referral for genetic counseling, please contact Phi Life Sciences' Clinical Support at 888-576-5445.

Testing Performed By: Phi Life Sciences, LLC, CLIA No. 42D2114196, CLIA No. 21D2158103, CAP 807914301, 645 Meeting St, Suite 3, Charleston, SC 29403. **Laboratory Director: Edwin Naylor, PhD**

Disclaimer: The information presented on this report is provided as general educational health information. The content is not intended to be a substitute for professional medical advice, diagnosis, or treatment. Only a physician, pharmacist or other healthcare professional should advise a patient on the use of the medications prescribed. The pharmacogenetic assay involves use of reporting software and genotype-phenotype associations performed by Translational Software (www.translationalsoftware.com). The software has not been evaluated by the Food and Drug Administration. The software, and the report generated by the software, is not intended to diagnose, treat, cure, or prevent any disease. A qualified designee within the lab uses Translational Software to generate and subsequently review the report. The pharmacogenetic report is one of multiple pieces of information that clinicians should consider in guiding their therapeutic choice for each patient. It remains the responsibility of the health-care provider to determine the best course of treatment for a patient. Adherence to dose guidelines does not necessarily assure a successful medical outcome.

References



- 1. http://www.ahrq.gov/qual/aderia/aderia.htm.
- 2. Committee on Quality of Health Care in America: Institute of Medicine. To err is human: building a safer health system. Washington, D.C.: National Academy Press; 2000.
- 3. Lazarou J, Pomeranz B, Corey PN. Incidence of adverse drug reactions in hospitalized patients: A meta-analysis of prospective studies. JAMA1998;279:1200–1205.
- 4. Gurwitz JH, Field TS, Avorn J, McCormick D, Jain S, Eckler M, et al. Incidence and preventability of adverse drug events in nursing homes. Am J Med 2000;109(2):87–94.
- 5. Johnson JA, Bootman JL. Drug-related morbidity and mortality. A cost-of-illness model. Arch Intern Med1995;155(18):1949–1956.
- 6. Leape LL, Brennan TA Laird N, Lawthers AG, Localio AR, Barnes BA et al. The nature of adverse events in hospitalized patients. Results of the Harvard Medical Practice Study II. N Engl J Med 1991;324(6):377–384.
- 7. Classen DC, Pestotnik SL, Evans RS, Lloyd JF, Burke JP. Adverse drug events in hospitalized patients. Excess length of stay, extra costs, and attributable mortality. JAMA 1997;277(4):301–306.
- 8. Schappert SM. Ambulatory care visits to physician offices, hospital outpatient departments, and emergency departments: United States,1997. National Center for Health Statistics. Vital Health Stat. 1999;13(143).
- 9. National Association of Chain Drug Stores. 2000 community pharmacy results. 2001. Alexandria, VA.
- 10. Jacubeit T, Drisch D, Weber E. Risk factors as reflected by an intensive drug monitoring system. Agents Actions 1990;29:117–125.
- 11. Leape LL, Bates DW, Cullen DJ, Cooper J, Demonaco HJ, Gallivan T, et al. Systems analysis of adverse drug events. ADE Prevention Study Group. JAMA 1995;274(1):35–43. 12. Raschetti R, Morgutti M, Menniti-Ippolito F, Belisari A, Rossignoli A, Longhini P, et al. Suspected adverse drug events requiring emergency department visits or hospital admissions. Eur J Clin Pharmacol 1999;54(12):959–963.
- 13.http://www.fda.gov/drugs/developmentapprovalprocess/developmentresources/drugint eractionslabeling/ucm110632.htm.