## METFORCE® BACK Instructions



## **Directions for Use**

- 1. Lay brace out flat with interior facing up
- 2. Secure back and/or side panels, if desired
- 3. Wrap around waist at midsection

- 4. Secure brace at front of body
- 5. Tighten brace to desired comfort level using pulley system

Notice rigid front panels, side panels, and back panel can be removed as desired.

## Directions for Universal Use

- 1. Lay brace out flat with interior view facing up
- 2. Ensure belt is sized appropriately by matching label to pleat on belt
- 3. To resize, remove right and left belts from back board. Fold right and left belts to desired size and reattach to back board
- 4. Secure back and/or side panels, if desired
- 5. Wrap around waist at midsection
- 6. Secure brace at front of body
- 7. An extender is included to provide additional length to belt if needed
- 8. Determine if belt is appropriately sized. If not, return to step 1
- 9. Once belt is appropriately sized, belt can be cut to eliminate excess fabric. Be sure to always allow for two extra sizes when cutting
- 10. Tighten brace to desired comfort level using pulley system

Notice rigid front panels, side panels, and back panel can be removed as desired.