


The rugged **Palm Grip™** is manufactured using high quality materials and is designed for longevity. It is made from a combination of machined and cast stainless steel components with plastic molded grips on the instrument body and trigger. Ergonomically designed to fit comfortably in the hand. The upward piston design offers less stress to the hand and wrist.

- Accurate, multi-dose vaccinator
- Simple maintenance and minimal replacement parts
- Easy to disassemble and clean
- Dose adjustable
- Includes standard Draw-Off tube

KEY FEATURES:

- Strong rugged metal and nylon construction
- Adjustable 0.1 ml - 1 ml model and 1 ml - 5 ml model
- Designed to feel comfortable in the hand and minimize fatigue
- Highly accurate

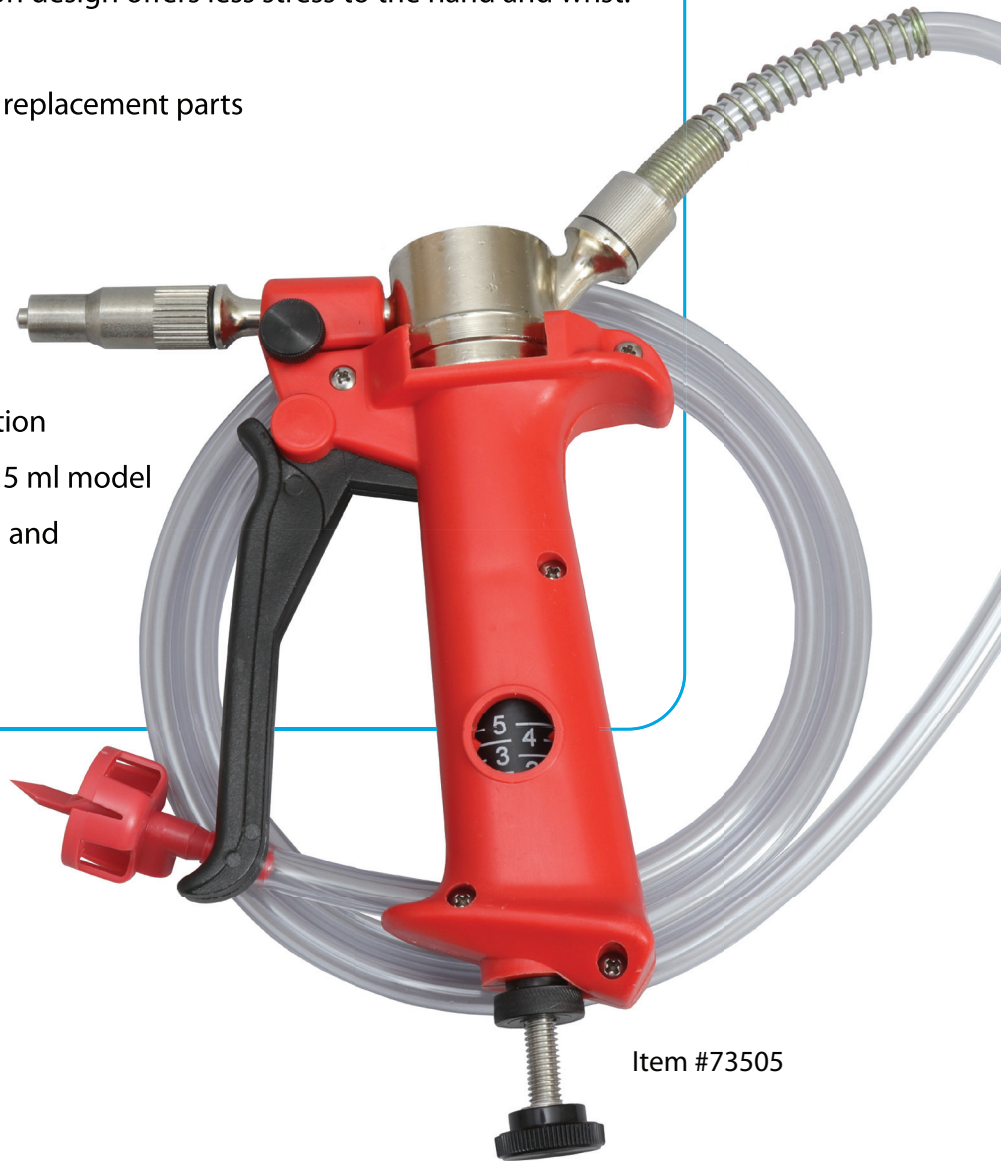

Item #73505

Item	Product Description	UOM
73500	Palm Grip™ 0.1 - 1.0 ml Multi-Dose Vaccinator	Each
73505	Palm Grip™ 1 - 5 ml Multi-Dose Vaccinator	Each


**ACCURATE,
MULTI-DOSE
VACCINATOR**

SPARE PARTS KIT INCLUDES:

Item	Product Description	UOM
05-08-120	1 x Palm Grip™ Outlet Spring - (A)	Each
05-05-210	1 x Palm Grip™ Outlet Check - (B)	Each
05-04-222	1 x Palm Grip™ Outlet O-Ring** - (C)	Each
05-04-019	1 x Palm Grip™ Main Seal, 0-1 ml - (D)	Each
05-04-110	1 x Palm Grip™ Main Seal, 1-5 ml - (D)	Each
05-04-103	1 x Palm Grip™ Inlet O-Ring** - (E)	Each
05-08-020	1 x Palm Grip™ Inlet Spring - (F)	Each
05-05-210	1 x Palm Grip™ Outlet Check - (G)	Each

* See diagram below with the spare parts labeled A-G. When replacing these parts, lightly lubricate with Genesis Power Lube or petroleum jelly.

** O-Rings may be black or orange.

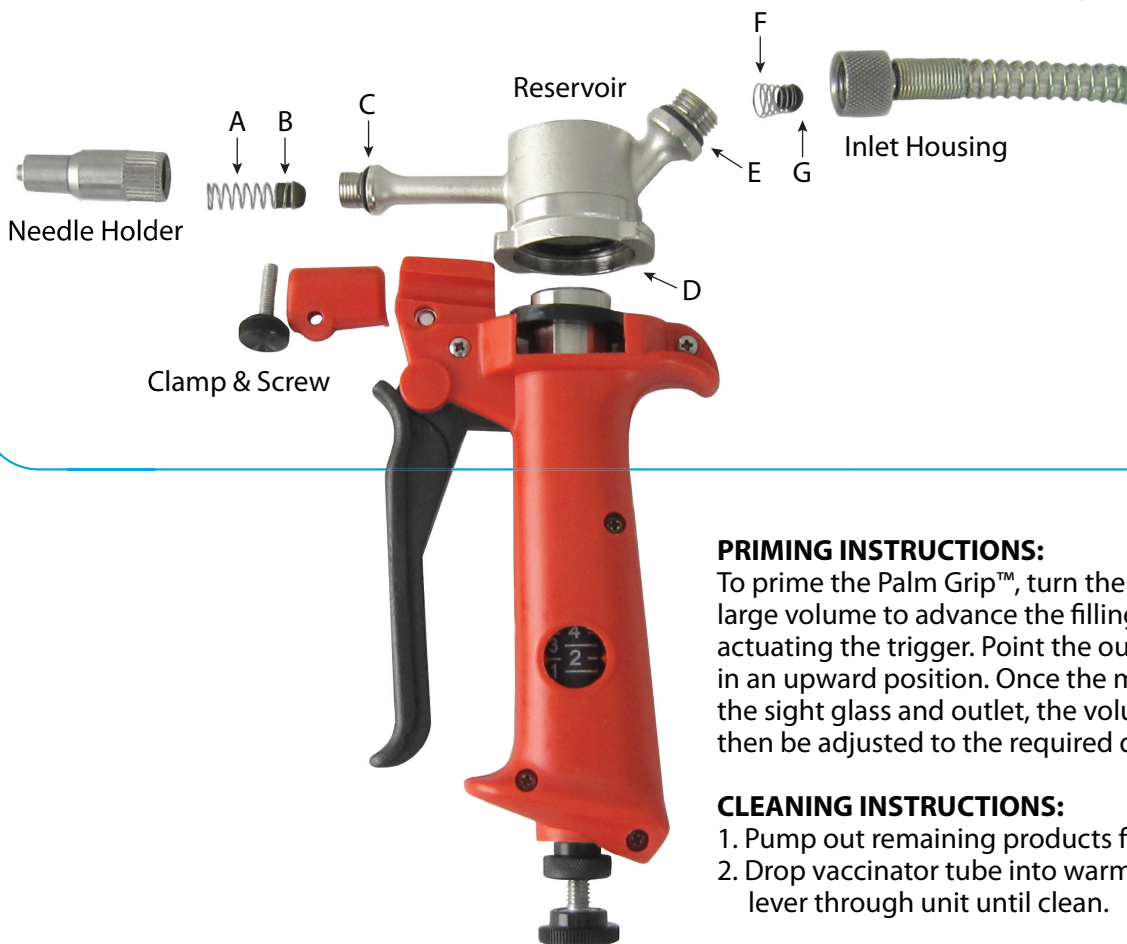

PRIMING INSTRUCTIONS:

To prime the Palm Grip™, turn the volume adjustment screw to a large volume to advance the filling of the delivery tube by actuating the trigger. Point the outlet end of the Palm Grip™ in an upward position. Once the medicant has completely filled the sight glass and outlet, the volume adjustment screw may then be adjusted to the required dosage.

CLEANING INSTRUCTIONS:

1. Pump out remaining products from Palm Grip™.
2. Drop vaccinator tube into warm soapy water and actuate the lever through unit until clean.

For more thorough cleaning, remove the Clamp & Screw. Hold the Palm Grip™ in one hand twisting the Reservoir counterclockwise to remove from the body. Clean with hot water. Unscrew the Inlet Housing and Needle Holder to expose the check balls and springs. O-rings and check balls may be lubricated with petroleum jelly to prolong life.