

Hi-Flow™ Illustration (Basic)



Nozzle Fitting Instructions



1. Place Nozzle through the opening of the Barrel Cap.
2. Slide Ball & Spring Check Valve into the Barrel (Insert ball side first).
3. Attach Nozzle by screwing the Barrel Cap and Nozzle onto the Barrel. Do not overtighten. Ensure Nozzle Gasket is present.

Hi-Flow™ Operating Instructions

1. Complete Nozzle Fitting Instructions.
2. While depressing the handle, turn the Adjustment Ring to desired volume (dosage marks shown on Barrel).
3. Connect Draw-Off Tubing (with Spike or Threaded Cap) to desired container.
4. Hold the Hi-Flow™ dosing instrument in a vertical position while directing the Nozzle away from any person into a waste container. Prime the Hi-Flow™ by slowly depressing the handle as many times as necessary to clear all air bubbles.
5. The Hi-Flow™ is ready for dosing. Each full handle depression will deliver the desired volume.

Hi-Flow™ Cleaning Instructions – For best results and to maintain longevity, daily cleaning is recommended.

1. When daily dosing is complete, dispense all product waste into a waste container.
2. Place the Draw-Off Tubing into a container of soap and warm water and flush the system by depressing the handle until all waste is completely flushed. Continue to flush the system with air until the dosing instrument is completely empty.
3. Unscrew the Barrel from the instrument body. Place two to three drops of vegetable oil or silicone oil on the O-Ring.
4. Re-attach the Barrel to the instrument and store in a cool, dry place.