


## The COVID-19 IgG/IgM Rapid Cassette Test

**NOW, AVAILABLE FOR THE SALE AND SHIPMENT WITHIN AND THROUGHOUT THE  
UNITED STATES UNDER THE EMERGENCY UTILIZATION AUTHORIZATION NUMBER  
PEAU #2000056, ISSUED BY THE US FDA ON MARCH 23, 2020**


# The COVID-19 IgG/IgM Rapid Cassette Test

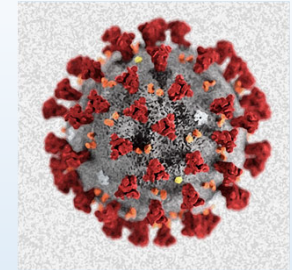

- HIGH SENSITIVITY
- HIGH SPECIFICITY
- RESULTS IN 2-10 MINUTES
- TESTED IN FIELD
- FAST DELIVERY
- OVER 20 MILLION TEST KITS SHIPPED
- COST EFFECTIVE
- APPROVED FOR USE IN THE E.U.
- EASY TO READ RESULTS
- HIGH QUALITY
- NOW MAKING 300,000 TEST PER DAY AND INCREASING PRODUCTION
- INSTRUCTIONS IN MULTIPLE LANGUAGES
- DELIVERS BOTH IGG AND IGM TEST RESULTS




# The COVID-19 IgG/IgM Rapid Cassette Test

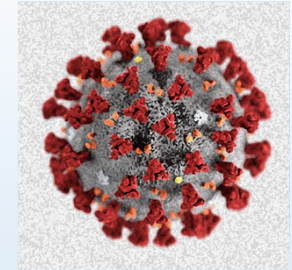

The **COVID-19 IgG/IgM Rapid Test Cassette** is designed for use with Whole Blood, Serum or Plasma. It is an easy-to-use and accurate solid phase immunochromatographic assay for the rapid, qualitative and differential detection of both IgG and IgM antibodies to 2019 Novel Coronavirus with excellent sensitivity and specificity.



# The COVID-19 IgG/IgM Rapid Cassette Test

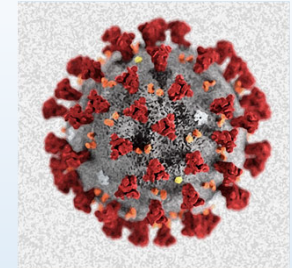


With the "known" international rates of infection now approaching 400,000 people and almost 20,000 deaths, we **WILL** see these numbers increasing drastically as the disease progresses. We will also see these numbers increase as quality and accurate testing takes place worldwide.

People who are symptomatic for COVID-19 are more easily identified than those who are asymptomatic. This is the problem!

***BUT, what is the solution to identifying asymptomatic and symptomatic individuals?***

The answer is simple. An accurate and easy to use rapid diagnostic is needed for infield, hospital and other professional use. The **COVID-19 IgG/IgM Rapid Cassette Test** delivers results quickly and requires just a drop of whole blood, serum or plasma providing test results in minutes with an exceptional degree of accuracy.

# The COVID-19 IgG/IgM Rapid Cassette Test

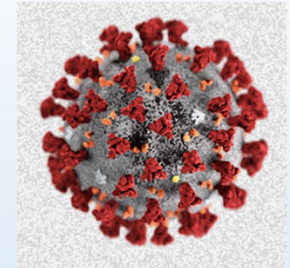


What does it take to run the ***COVID-19 IgG/IgM Rapid Cassette Test***? The easy to read instructions are included in every kit. But, what are some of the basic questions that we must answer?

Blood sample size needed to run the test- 5-10 microliters  
Time necessary to view the test results- 2-10 minutes


<b>IgG</b>	Sensitivity	98%
	Specificity	100%

<b>IgM</b>	Sensitivity	95.7%
	Specificity	100%

# The COVID-19 IgG/IgM Rapid Cassette Test


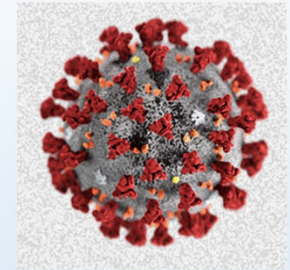


The COVID-19 IgG/IgM Rapid Cassette Test Kit comes in a prepackaged box containing 25 COVID-19 cassettes, 25 pipettes, and diluent sufficient to run 25 test.

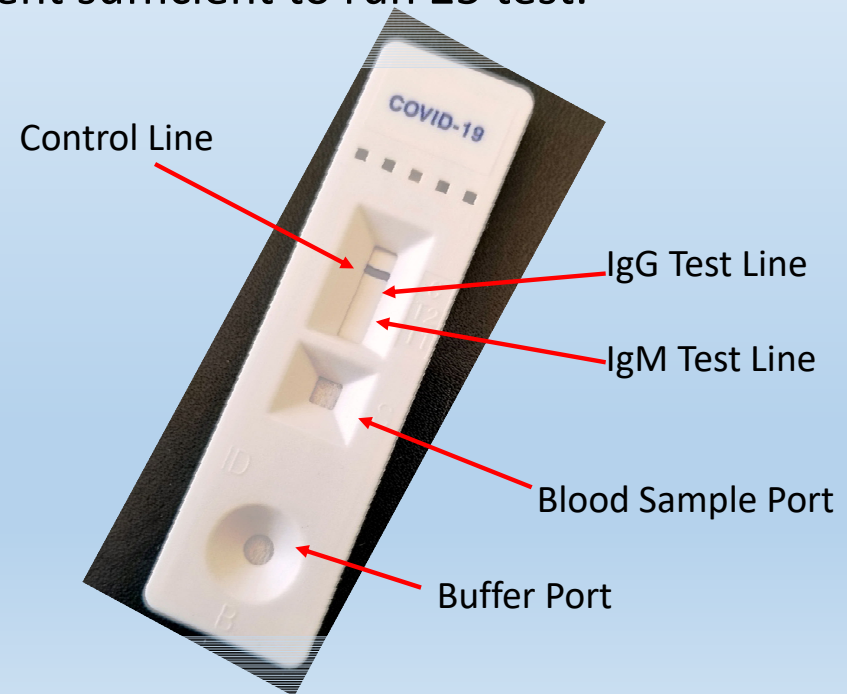



# The COVID-19 IgG/IgM Rapid Cassette Test

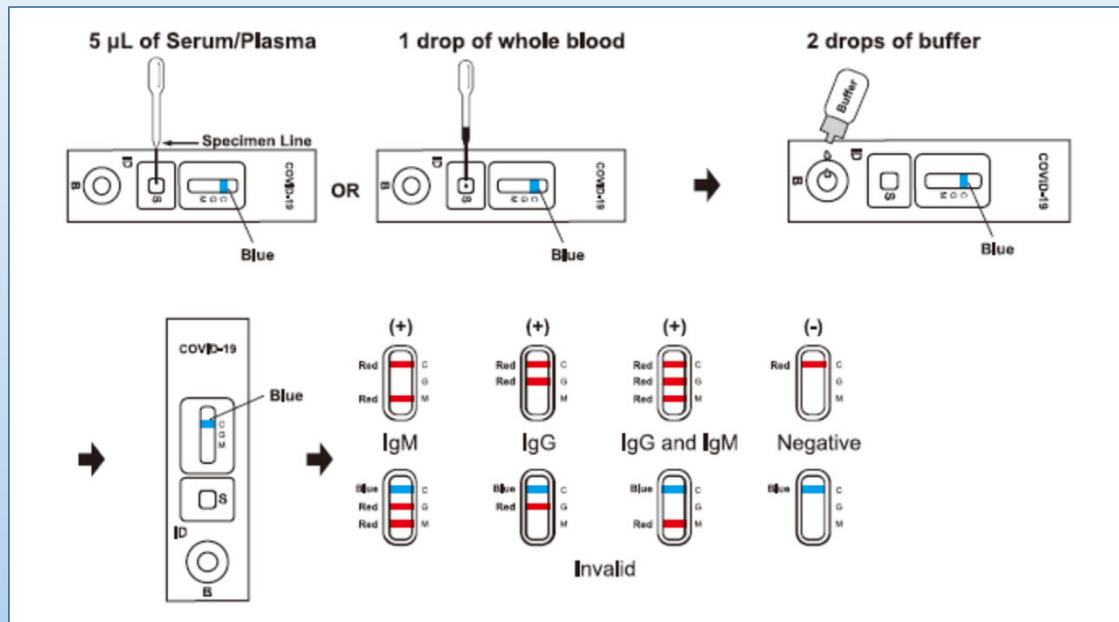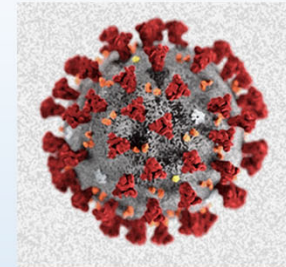

**NEGATIVE:** The colored line in the Control (C) Region changes from blue to red. No line in IgG and IGM line indicated **NEGATIVE**.


**IgM POSITIVE:** The colored line in the Control (C) Region changes from blue to red. A colored IgM line indicates **POSITIVE for IgM**.

**IgG POSITIVE:** The colored line in the Control (C) Region changes from blue to red. A colored IgG line indicates **POSITIVE for IgG**.

**IgG and IgM POSITIVE:** The colored line in the Control (C) Region changes from blue to red. Two colored line in the IGM and the IgG indicates **POSITIVE for IgM and IgG**.

**INVALID:** The Control line in the Control (C) Region remains partially or completely blue and fails to change from blue to red.

# The COVID-19 IgG/IgM Rapid Cassette Test

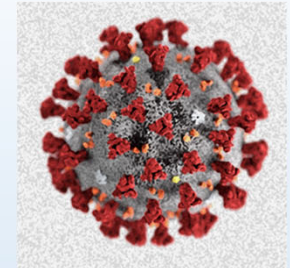

Our **COVID-19 IgG/IgM** has undergone multiple evaluations infield to confirm accuracy of our diagnostic.

The following pages represent 8 such sites totaling 681 patients. This patients represent only a few of the 1000s of patient testing that has taken place.

Wuhan University People Hospital  
No. 238 Jiefang Road, Wuchang District,  
Wuhan City, Hubei Province.

## Results Analysis

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	34	1	35
Negative	1	63	64
Total	35	64	99

Total Agreement =97.98%

Wuhan No.1 People Hospital  
No. 215 Zhongshan Avenue Qiaokou District,  
Wuhan City, Hubei Province

## Results Analysis

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	0	2	2
Negative	0	98	98
Total	0	100	100

Total agreement =98%

Wenzhou Central Hospital  
Lucheng District, Wenzhou City,  
Zhejiang Province.


## Results Analysis:

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	78	0	78
Negative	2	0	2
Total	80	0	80

Total Agreement =97.5%



# The COVID-19 IgG/IgM Rapid Cassette Test

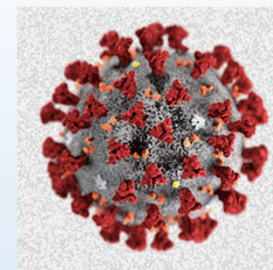

Jiande No.1 People Hospital  
No. 599 Yanzhou Ave, Xinanjiang St,  
Jiande City, Zhejiang Province.

**Reference Reagent:**  
Results Analysis

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	0	1	1
Negative	0	79	79
Total	0	80	80

Total agreement = 98.75%

Taizhou Centra Hospital  
No. 999 DongHai Avenue, Jiaojiang District,  
Taizhou City, Zhejiang Province

**Results Analysis**

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	2	0	2
Negative	0	79	79
Total	2	79	81

Total agreement = 100%


Zhejiang Province No.2 Prison Hospital  
No. 3 Qiushan Street, Linpin Town,  
Yuhang District, Hangzhou City, Zhejiang Province.

**Results Analysis**

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	0	0	0
Negative	0	35	35
Total	0	35	35

Total agreement = 100%

# The COVID-19 IgG/IgM Rapid Cassette Test

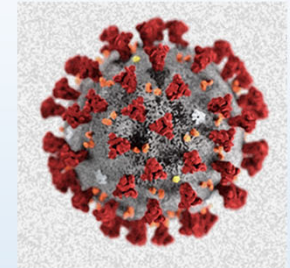

Children's Hospital,  
Zhejiang University School of Medicine  
No. 3333 Binshen Road, Binjiang District,  
Hangzhou City, Zhejiang Province.

## Results Analysis

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	1	0	1
Negative	0	11	11
Total	1	11	12

Total agreement= 100%

Zhejiang University Affiliated  
No.1 Hospital  
No. 79 Qinchun Road, Hangzhou City,  
Zhejiang Province.

## Results Analysis

Test Reagent	Reference Reagent		Total
	Positive	Negative	
Positive	103	0	103
Negative	9	14	23
Total	112	14	126

Total agreement= 92.86%