



Bath Transfer Bench

This bench is designed for safe access to bathtub enclosures. It is designed for use with shower curtains closures (just tuck the shower curtain around the bench to prevent water from flowing out), but not with shower doors, as they will not allow access in that small opening.

The bench is adjustable so that the user can be positioned facing the shower head.

To adjust...underneath the bench are;

- 1) backrest slots, with push button adjustments on the backrest,
or
- 2) bolts or screws that can be removed and repositioned.

Place transfer bench upside-down on a flat surface. Depress push buttons on each adjustable suction leg and slide onto the bench frame on the side where the backrest slots are located (**see Figure 1**).

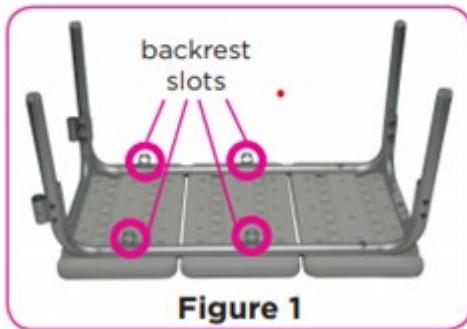


Figure 1

Turn transfer bench onto its side. Slide the backrest into the backrest slots and secure by engaging the push buttons (**see Figure 3**). The backrest can be attached to either side of bench, depending of which way user would like to face.

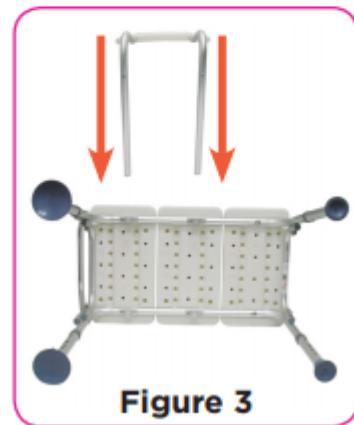


Figure 3

© 2012 Drive Medical, Inc.