

Subject: Alpha Protech N95 Respirators – **Made in America Shipping Now**
REQUIRES FIT TEST TRAINING – SHIPPNG 2 WEEKS FROM PURCHASE.

Desktop Alert: #1 Reseller of these products for Alpha Protech Worldwide

Press Release: <https://www.pr.com/press-release/824207>

FIT Testing superior to 3M 1860 N95

Certified Department of Defense Vendor Since 2007.

Relationship Between Desktop Alert and Alpha Protech: 15+ Years

*In a 2020 study on the efficacy of facemasks in preventing acute respiratory infection, surgical masks worn by patients with such infections (rhinovirus, influenza, seasonal coronavirus [although not SARS-CoV-2 specifically]) were found to reduce the detection of viral RNA in exhaled breaths and coughs. **Specifically, surgical facemasks were found to significantly decreased detection of coronavirus RNA in aerosols and influenza virus RNA in respiratory droplets.** The detection of coronavirus RNA in respiratory droplets also trended downward. Based on this study, the authors concluded that surgical facemasks could prevent the transmission of human coronaviruses and influenza when worn by symptomatic persons and that this may have implications in controlling the spread of COVID-19.*

In a 2016 systematic review and meta-analysis, Smith et al found that N95 respirators did not confer a significant advantage over surgical masks in protecting healthcare workers from transmissible acute respiratory infections. SOURCE:

<https://www.medscape.com/answers/2500114-197511/are-surgical-facemasks-more-effective-than-n95-masks-in-preventing-coronavirus-disease-2019-covid-19>

Product Description	NIOSH Approved	FDA Certified	Donning Info
Critical Cover® PFL® N-95	X	X	http://www.alphaprotech.com/userfiles/doccenter/brochures/DP.pdf https://www.youtube.com/watch?v=xiGYt2_ftlk

Particulate Respirator			
------------------------	--	--	--



L092824

AlphaProTech
L I T T E R A T U R E

OCT 21 2009

Abbreviated 510(k)
For Alpha Pro Tech, Inc
895 N95 Respirator Mask with Positive Facial Lock and Magic Arch

SECTION 5
510(k) SUMMARY

Submitter:

Alpha Pro Tech, Inc.
236 North 2200 West
Salt Lake City, Utah 84116
Registration Number 1721683

Contact:

David Kitchen
Corporate QA/RA Manager
Telephone: 801-355-5816
Fax: 801-355-2534
E-mail: dkitchen@alphaprotech.com

Trade Name:

Alpha Pro Tech 895 N95 Respirator Mask, Positive Facial Lock with Magic Arch

Model Numbers:

895

Common Name:

Healthcare N95 Particulate Respirator Mask

Predicate Device:

Alpha Pro Tech's 895 N95 Respirator Mask has the same technological characteristics as Aearo Company's Pleats Plus 1050 Respirator Masks in that both are NIOSH approved, both are made of nonwoven meltblown polypropylene; both incorporate a forming nosepiece, both employ a flat pleated style; and both are secured using dual elastic head straps.

Classification:

Device Class – Class II
Product Code – MSH – Surgical N95 Respirator
CFR Section – 21 CFR 878.4040

Device Description:

Alpha Pro Tech's 895 N95 Respirator Mask consists of a polypropylene meltblown filter media sandwiched between non-woven wetlaid outer and inner covers. The mask is secured using two polyester covered non-latex rubber headbands. Fitting around the face is facilitated using a nose wire and an adjustable chin wire made from aluminum. No fiberglass media is used in this product.

Intended Use:

Alpha Pro Tech's 895 N95 Respirator Mask meet the CDC guidelines for TB Exposure Control within healthcare facilities. These devices are intended to be worn by healthcare personnel during surgical procedures to protect both the patient and healthcare personnel from the transfer of

Page 5 of 31



Alpha Pro Tech N-95 Particulate Respirator

PFL® (Positive Facial Lock®) Design
with Magic Arch® and Twist Seal®



Donning Instructions



Step 1. Open Mask

Grasp nose piece and chin piece and snap open to activate the Magic Arch®. A "pocket" is formed in the mask.

Note: Mask must be fully open.



Step 2. Conform to Face

Place chin in pocket created, place bottom headband below ears and top band at crown of head. Conform nose piece to nose and facial contours.



Step 3. Fit to Face

Holding mask at nose piece, gently pull down under chin so mask fits snug against face.



Step 4. Positive Facial Lock®

Pinch any excess material together under the chin and twist the Twist Seal® chin piece. Perform Fit Check (see below*).

*Fit Check - Place both hands along the edges of the mask and exhale. If air escapes at your nose, adjust the nose piece. If air leaks at the edges, untwist the Twist Seal chin piece and repeat Step 3.

Warning

1. This product does not eliminate the risk of contracting any disease or infection. Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards.

2. Follow all instructions and warnings on the use of this respirator. Failure to do so will reduce respirator effectiveness, wearer protection and may result in sickness or death. Wear during all times of exposure. For additional information refer to your supervisor and/or see dispenser box.

3. Before use, the wearer must be trained by the employer in proper respirator use in accordance with applicable safety and health standards for the contaminant and exposure level in the assigned work area.

Positive Face Lock, Magic Arch and Twist Seal are registered trademarks of AlphaProTech, Inc.

Reorder Part No. 635

AlphaProTech
INCORPORATED

Customer Service - Phone: 800.749.1363 • Fax: 905.479.8703 • sales@alphaprotech.com

APT-3027 5M 9/03

7/20/2020

K890128 - Mask, Surgical FDA 510(k) APPLICATION FOR ALPHA PRO TECH, INC.



[SEARCH](#) [COMPANIES](#) [PRODUCT CODES](#) [DEVICE NAMES](#) [CONTACTS](#) [APPLICATIONS](#) [DECISIONS](#) [CONTACT](#)

FDA 510(k) Application Details - K890128

Device Classification Name	Mask, Surgical
	<input type="checkbox"/> More FDA Info for this Device
510(k) Number	K890128
Device Name	Mask, Surgical
Applicant	ALPHA PRO TECH, INC. 
	6102 SOUTH MEMORIAL TULSA, OK US Other 510(k) Applications for this Company
Contact	W. C KLINTWORTH Other 510(k) Applications for this Contact
Regulation Number	876.4040
	<input type="checkbox"/> More FDA Info for this Regulation Number
Classification Product Code	FXX Other 510(k) Applications for this Device
	<input type="checkbox"/> More FDA Info for this Product Code
Date Received	01/13/1989
Decision Date	02/14/1989
Decision	SE - Substantially Equivalent
Classification Advisory Committee	SU - General & Plastic Surgery
Review Advisory Committee	SU - General & Plastic Surgery
Statement / Summary / Purged Status	
Type	Traditional
Reviewed By Third Party	N
Expedited Review	

[FDA Source Data](#)

[Search](#) | [Companies](#) | [Product Codes](#) | [Device Names](#) | [Contacts](#) | [Applications](#) | [Decisions](#) | [Contact](#)



N-95 Particulate Respirator

With Magic Arch® and Positive Facial Lock®

The Alpha Pro Tech N-95 Particulate Respirator, designed specifically for use in a healthcare setting, is NIOSH-approved as a Type N-95 Respirator, meets CDC guidelines for TB exposure control and is FDA approved as a surgical mask.

One-Size-Fits-All

To help you reduce costs and inventory, the N-95 Respirator uses our exclusive Magic Arch/Positive Facial Lock "one-size-fits-all" design. This unique design feature permits the respirator to be "custom fit" to virtually any face, no matter what shape or size. This allows you to stock only one size for all your TB protocol needs.

The traditional pleated style N-95 offers all this without the drawbacks of cone and duckbill-style respirators which restrict downward vision and collapse easily when stored.

- **Integral Magic Arch®** — Provides a larger breathing chamber with no restriction of downward vision as duckbill or cone masks do.
- **Positive Facial Lock® (PFL)** — Allows total facial closure for "custom fit" on any size face when donned correctly.
- **No Natural Rubber Latex** — Comfortable, knit tricot mesh covered bands do not contain natural rubber latex, eliminating latex allergy concerns.
- **Traditional Pleated Design** — Saves inventory space, cannot be crushed when stored, as duckbill and cone respirators can, and is easy to carry with you.

Indications for Use:

Alpha Pro Tech's N-95 Respirator Mask meets the CDC guidelines for TB Exposure Control within healthcare facilities. These devices are intended to be worn by healthcare personnel during surgical procedures to protect both the patient and healthcare personnel from the transfer of microorganisms, body fluids, and particulate material. It is intended to be a single use, disposable device.



AlphaProTech
INCORPORATED™



Desktop Alert IS NOW THE #1 TOP reseller of Alpha Protech PPE. Desktop Alert is also a registered small business at the Department of Defense and Certified Department of Defense Software Company. This N95 respirator is touted as a superior subset of the N95 Respirator class. The product line is Made in America.

Desktop Alert has 5 Million units that will ship across 4Q and into January and February 2021. Alpha Protech is a US publicly traded company on NASDAQ.

<https://finance.yahoo.com/quote/Apt/>

Product Description	Cases Per Pallet	Pallet Details	Inventory Validation
Critical Cover® PFL® N-95 Particulate Respirator	96 cases 210 Respirators Per Case	800lbs 20,160 pieces	Desktop Alert will conduct a conference call at the buyer's request with C Level officers at AP. Shipping will start 2-3 WEEKS after a Purchase Order has been issued.

Terms: 100% WIRE ONLY. Buyer is responsible for shipping insurance if desired. Seller NOT responsible for delivery delays. UPS Freight has experienced delays due to COVID-19.

Wire Information:

Account Owner: Desktop Alert Inc.

346 Main Street, Chatham NJ. 07928

Bank of America Acct # 009520365661 ABA # 026009593

Bank of America Address: 269 Main St, Chatham, NJ 07928