



FLUSH FRIDAYS

**PLEASE COMPLETE THE FOLLOWING STEPS TO ENSURE SAFE STORAGE
AND LONGER LASTING EQUIPMENT**

- Ensure that the Oxygen tank is turned on
- Turn the N₂O tank off
- Turn on blender and set to a 50/50 mixture of N₂O and O₂
- Set gas flow to 5 or more LPM
- Watch as the N₂O gauge decreases to 0 PSI
- A Nitrous Alarm may sound (digital units only)
- Turn off Oxygen Tank
- Turn off Blender
- Press the O₂ flush button until no more gas is present

CONGRATULATIONS! YOU HAVE FLUSHED YOUR SYSTEM