

Quick Start Guide


⚠️ READ THE COMPLETE OPERATING MANUAL ⚠️

1. Remove all packing materials and plug the unit into a grounded outlet. *Ensure that the warmer has a minimum of 3 inches of airspace at the side and rear walls.*

2. Turn the warmer 'ON' by the power switch located on the lower rear panel. The display will indicate the current temperature in degrees fahrenheit and begin heating toward the recommended temperature of 125°F (51°C). You can quickly change the set-point (100 to 125) and temperature unit (°F or °C) of the warmer to meet your requirements.

To change temperature set-point

Factory default 125°F

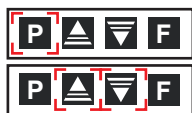


Press **P** once to see **SP**

Press **Up** or **Down** to Increase / Decrease

To exit after making a change, continue to press **P** until you see the warmer's temperature (process value) on the controller's display.

To change temperature unit (°C or °F)


Factory default °F

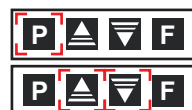

Press **P** twice to see **UNIT**


Press **Up** or **Down** to select between °F or °C

3. Begin inserting **Theraworx** products into product slots with the label facing up. Status indicators will illuminate red, indicating a slot has been filled. Avoid placing the product beyond the proximity sensor, as it will not activate the timer for that slot.

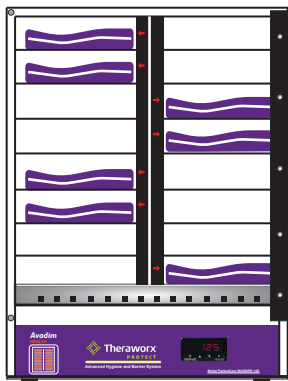

Stock wipe product in any combination

Activate proximity sensors by placing product near column

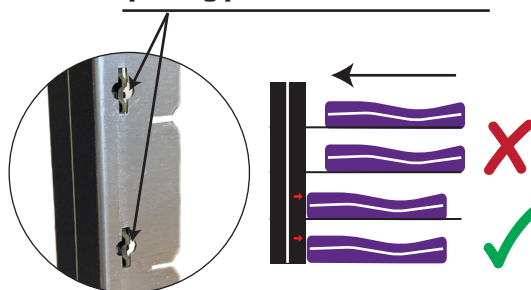


Avadim
HEALTH

Theraworx
PROTECT

Advanced Hygiene and Barrier System

Model PatientCare WARMER 16S

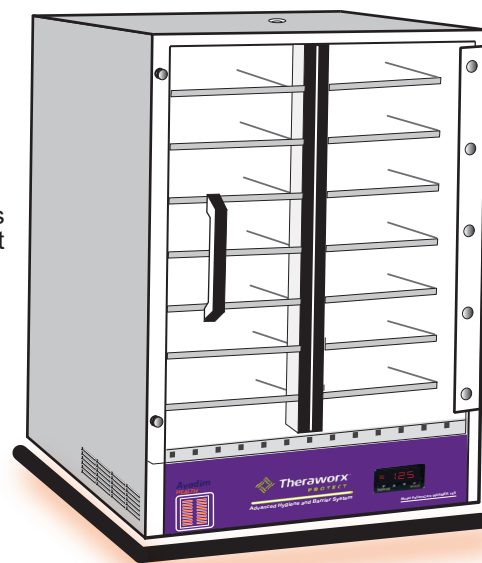

Status Indicators



Heating - Solid red LED

When wipe product is inserted, the status indicator LED will illuminate **SOLID RED**. This indicates the wipes are being warmed to the desired temperature. The time to fully heat product will be approximately 90 minutes.



At temperature - Solid green LED

When wipe product is fully heated, the status indicator light will illuminate **SOLID GREEN**. This indicates the wipes in the slot are ready for use.



Use first - Flashing green LED

When the slot indicator is **FLASHING GREEN**, this indicates the product should be used first, as it has been in the warmer in excess of 84 hours.



Discard - Flashing red LED

When any wipe product has been inserted for more than 168 hours, the status indicators will begin **FLASHING RED**. This indicates the product has been inserted in the warmer for more than 168 hours and should be discarded.

