




READYBATH[®]

NEXTGEN WARMER


Quick Start Guide

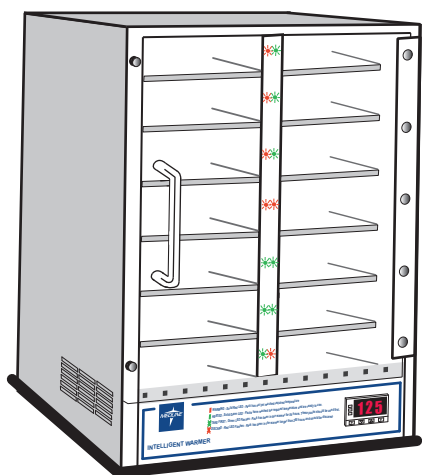

Model 14S

1. Remove all packing materials and plug the unit into a grounded outlet. Be sure the warmer has a minimum of 3" of airspace at side and rear panels.
2. Turn Warmer on by the power switch located at lower rear panel. The display will indicate the current temperature and begin heating towards the recommended temperature of 51°C/125°F.
3. Begin inserting **ReadyBath products** into product slots with label facing up, until the bag hits the "backstop" on the shelf. Status indicators will illuminate red indicating a slot has been filled.
4. The **ReadyBath** products are fully heated when status indicators illuminate green, (about 90 minutes).

Status Indicators

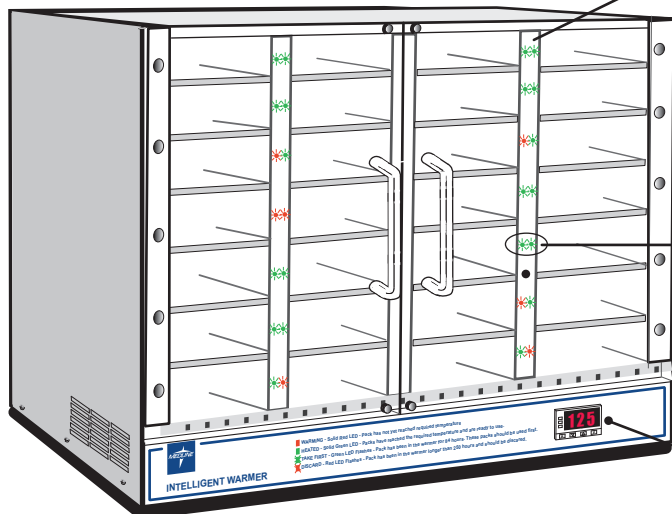

Model 28S

Status Indicators, located in the Warmer, will illuminate red or green, to inform when Bath Wipe is fully heated and ready for use.

Flashing green indicates "next-to-take" based on the Warmer's first-in-first-out product rotation system.

Flashing red indicates product should be discarded.

The Digital Controller is set to maintain a consistent temperature of 51°C/125°F and display the chambers current temperature.