



820 Oak Creek Dr, Lombard, IL 60148

How to load Liocan Intraocular Lens in the supplied Injector Set

1. Open the cartridge and place viscoelastic both in the front and in the cartridge chamber where the IOL will be placed.
2. Place the IOL over the viscoelastic with the notches on the haptics properly positioned on the up-left and bottom-right ("Z" fashion), and gently push down on the IOL with a closed forceps that has elongated tips.
3. Assure that the IOL is correctly positioned in the rails and perpendicular to the steady wing of the cartridge. Then, gently close the cartridge.
4. Place more viscoelastic into the end of the cartridge.
5. Before inserting the tip of the cartridge into the eye, assure that the IOL is not stuck in the cartridge by advancing the plunger slightly. The IOL should move smoothly.
6. Insert the cartridge's tip into the eye and slowly, but without stopping, inject the IOL into the capsular bag. The wings of the closed cartridge should always remain at 12 o'clock position during the injection. Do not rotate your hand at all.

For additional reference, please check the videos in our website courtesy of John S. Sapienza, Diplomate ACVO, Long Island Veterinary Specialists, New York.

<http://www.additiontechnology.com/liocan>

This is an addendum to the Instructions For Use supplied with Liocan Intraocular Lens. Carefully read the instructions prior to use the device. Failure to do so may result in patient complications.