



820 Oak Creek Dr, Lombard, IL 60148

How to load AJL Capsular Tension Ring into the supplied Injector

1. Remove the plastic tip from the injector.
2. Gently retract the plunger until the black piston in the barrel is adjacent to the finger grip.
3. Remove the CTR from its carrier, pressing the plunger while its arrows are pointing up.
4. Using a forceps, gently introduce the CTR into the grooved tip of the injector, leaving just one tip of the CTR outside of the grooved tip.
5. Introduce the tip of the injector into the capsular bag and gently inject the CTR. It will return into its original round position.

For additional reference, please contact us.

This is an addendum to the Instructions For Use supplied with AJL Capsular Tension Rings. Carefully read the instructions prior to use the device. Failure to do so may result in patient complications.