

AJL
vet

COLLAGEN MEMBRANE

DESCRIPTION AND INDICATIONS

Ophthalmic medical device made from type I collagen, indicated for corneal repair in animals, such as corneal perforations, ulcers, and stromal defects.

Enables the growth of the epithelium of the ocular surface and facilitates the migration of epithelial cells for their revitalization.



DIRECTIONS FOR USE

Before using the product, it must be hydrated for 20 minutes in a sterile saline solution.

After that, it may be custom cut for its use and if needed, different layers can be stacked.

For veterinary use only.

DIFFERENT MODELS

Ref: MCV22100
2cm (wide) x 2cm (high) x 100um (thick)

Ref: MCV22200
2cm (wide) x 2cm (high) x 200um (thick)

HOW SUPPLIED

Dry, sterile and single.