

AJLVET PRELOADED CTR

DIFFERENT MODELS

Ref: ACV001302PL

Open diameter: 13mm
Closed diameter: 11 mm

Ref: ACV001602PL

Open diameter: 16mm
Closed diameter: 14 mm



DESCRIPTION AND INDICATIONS

AJLVET capsular tension rings, made of polymethylmethacrylate (PMMA) with a UV radiation blocking filter, are designed to stabilize the capsular bag, at any time during cataract surgery, in patients who present with, among others:

- Primary or secondary zonular weakness
- Partial zonulolysis
- Trauma
- Pseudoexfoliation syndrome
- Risk of capsular retraction (e.g. uveitis)

For Veterinary use only

DIRECTIONS FOR USE

1. Open the package and take the injector. (Figure 1)
2. Turn the wheel forward until the ring is fully inserted into the injector (Figure 2)
3. Unlock the injector by sliding the tab towards the center. (Figure 3)
4. Insert the tip of the injector into the capsular bag (Figure 4) and turn the wheel backwards to release the ring in a smooth and controlled manner.
5. The ring will release itself from the hook.
6. Turn the wheel forward until the hook is retracted and remove the injector from the eye.

