

STRAPLESS N95 MASK

Mask Sizing Tips

To ensure the mask fits comfortably on your face and provides optimal protection, it is important to follow all instructions and to select a properly sized mask.



Below are tips for selecting a properly sized mask:

1. Mask edges should fit outside of your eye-line and not quite to your ears.
2. If you have access to all sizes, it is recommended to try the large size first. If this does not fit properly, try the small or extra large size to find the right fit for you.
3. Ensure an adequate tab can be formed under the chin to allow for a good seal.
4. If the mask is too loose or too tight, the mask will not fit comfortably. The mask should allow you to comfortably speak and move your head during wear.



US +1 636.675.3319



Email: info@peerlessmedical.com



Website: www.peerlessmedical.com

*MANUFACTURED FOR AVERY DENNISON MEDICAL BY GLOBAL SAFETY FIRST, LLC UNDER TC#84A-8133

IMAGE USE WITH PERMISSION OF AVERY DENNISON MEDICAL MTR-MKT-000378-A