



## DESCRIPTION

DermaBind TL is a dehydrated amniotic membrane covering that preserves the comprehensive collagen matrix, glycoconjugates, and glycosaminoglycans.

## THE DERMABIND TL PROCESS

DermaBind TL undergoes a proprietary preservation method, which retains all native layers of the placental membrane, including the amnion and chorion with the spongy layer intact, unlike many placental allografts on the market.

The DermaBind TL process does not use unnecessary chemicals, antibiotics or preservatives and is never frozen.

The finished product is physician-friendly, durable, easily applied to a wound and can be stored at room temperature for up to 3 years.



## COMPOSITION

DermaBind TL contains 400+ proteins, including collagens, fibrins, elastins, glucosaminoglycans and glycoconjugates.



Global Supply Exchange, LLC  
6900 Tavistock Lakes Blvd, Suite 400  
Orlando, Florida 32827

Reid Albright  
reid.albright@globalsupplyexchange.com  
610-871-0333

Exclusively Distributed By



Medical Care Supplies and Equipment, LLC

EasyMedicalCare.com  
(219) 232-8036  
sales@EasyMedicalCare.com  
5534 Saint Joe Rd, Fort Wayne, Indiana 46835

## INTENDED USE

DermaBind TL can be used as a protective wound covering as either a partial and full thickness acute and chronic wounds.

- Trauma Wounds
- Dehisced Wounds
- Diabetic Foot Ulcers (DFUs)
- Pressure Injuries
- Venous Leg Ulcers (VLUs)


Can be used at the onset and for the duration of the wound, weekly or at the discretion of the treating physician.

## APPLICATION

- Using aseptic technique, remove DermaBind TL from the package; the product may be trimmed as necessary
- Place the DermaBind TL patch onto the wound
- Assure DermaBind TL is in intimate contact with the wound bed; the product can be hydrated once placed on the wound bed
- Adhere DermaBind TL using preferred fixation method, then cover DermaBind TL with a primary non-adherent dressing
- Apply secondary dressings specific to the wound type over the non-adherent layer, which should remain in place for up to one week

## SIZE & STORAGE

- Available in a wide range of shapes and sizes
- Maintain at ambient temperature prior to patient application
- Do not freeze
- May be stored for up to 3 years at room temperature.

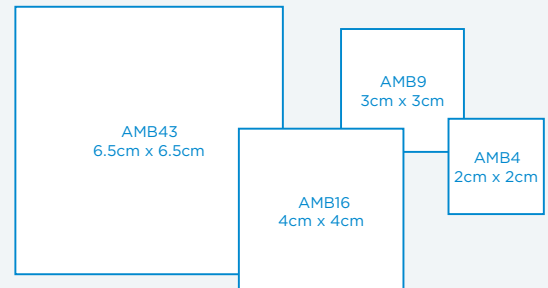

## REIMBURSEMENT

- Medicare contractors cover DermaBind TL for VLUs and DFUs†
- DermaBind TL can be applied to, and billed for, other wound types
- Confirm coverage prior to treatment through our Benefits Program

†Consult with your LCD (local coverage determination) for the specific wound types covered.  
VLU = venous leg ulcer; DFU = diabetic foot ulcer

**DermaBind TL  
billing**

**Q 4225**

### CPT codes

Site Preparation CPT†  
15002-15005

Application CPT codes†  
15271-15278

†Subject to guidelines and approvals, please contact our billing team for further details

**BILLING SUPPORT**  
**REIMBURSEMENT@HEALTHTECHWC.COM**



Global Supply Exchange, LLC  
6900 Tavistock Lakes Blvd, Suite 400  
Orlando, Florida 32827

Reid Albright  
reid.albright@globalsupplyexchange.com  
610-871-0333

Exclusively Distributed By



Medical Care Supplies and Equipment, LLC

EasyMedicalCare.com  
(219) 232-8036  
sales@EasyMedicalCare.com  
5534 Saint Joe Rd, Fort Wayne, Indiana 46835