

Wound Care Product Guide

Saving you time and money

	Minimal drainage Stage III-IV; full thickness	Moderate drainage Stage III-IV; full thickness	Heavy drainage Stage III-IV; full thickness
ALGINATES Up to 30/wound/mo		Moderate to heavy drainage only	
COLLAGENS Up to 30 pads or 30 gms/ wound/ mo	Minimal/Mod drainage (Pad/powder)		
	Minimal drainage (gel)		
FOAMS Up to 12/wound/mo		Moderate to heavy drainage only	
HYDROCOLLOIDS Up to 12 sheets/wound/mo	Minimal/Mod drainage		
HYDROGELS Up to 30pads or 3oz/wound/mo	Minimal drainage (sheets)		
	Minimal/Mod drainage (gel)		

Wrap/Tape/Gauze

Secondary dressings hold the primary dressing in place and provide a physical barrier to reduce the incidence of contamination and infection. Options based on wound type, location and primary dressing requirements.

Compression Garments

Compression garments are used in the treatment of an open venous stasis ulcer. These garments are applied directly over the secondary dressing to reduce edema associated with venous insufficiency.

HYDROGELS

Designed to donate moisture to dry wounds/burns and maintain a moist wound bed. Supports autolytic debridement. Gentle application and removal. Secondary dressing is necessary. Commonly uses: eschar and burns

COLLAGENS

Encourage fibroblast production and new tissue growth. Provides sacrificial MPPs to protect the bodies own healthy collagen. Non adherent to reduce trauma with dressing changes. Secondary dressing is necessary. Common uses: pressure ulcers, venous ulcers, surgical wounds, diabetic ulcers, trauma

ALGINATES

Highly absorbent but maintains a moist wound bed. Soft and adaptable to irregular wounds and can be packed. Easy removal for minimal damage to healthy tissue. Secondary dressing necessary. Common uses: pressure ulcers, infected wounds and venous stasis wounds

FOAMS

Highly absorbant and supports autolytic debridement. May use with or without boarder and/or transparent dressing. Secondary dressing necessary. Common uses: moist wound types with heavy drainage.

HYDROCOLLOIDS

Occlusive, water-resistant wound cover to maintain a moist wound bed. Supports autolytic debridement. Conformable to irregular areas and self-adhering. Secondary dressing not necessary. Common uses: heel ulcers, sacral ulcers, min-mod drainage.

Orders billed to insurance and delivered to patient within 2 days. Same day pick up available. In-services available upon request. Home Health patients eligible if order is received and mailed PRIOR TO start of care. Hospice patients eligible.