CARE INSTRUCTIONS

EN Remove the fabric strap from thumb guide and hand wash in warm water with a gentle detergent. Rinse in clear water, squeeze out excess moisture and lay on flat surface to dry. The body of the thumb support can be cleaned in warm water with a cloth and mild detergent. Allow to dry naturally away from any heat source and out of the sun.

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EN CAUTION

If allergic skin rash occurs please consult your doctor immediately. Do not use continuously for more than 24 hours.

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INDICATION

- Osteoarthritis of the CMC joint
- Postoperative treatment of the CMC joint
- After arthroplasty of the CMC joint.
- Instability of the CMC joint

CLASSIFICATION

Class 1 medical device, non sterile, non measurement function. This product fulfils the Medical Device Regulation (EU) 2017/45

Manufactured in China for ACTIVIVAS

ACTIVIVAS.COM

ACTIVIVAS

Effective

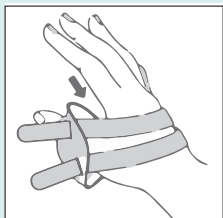
THUMB SUPPORT

Remarkable Support & Relief

Application & Care Instructions

FITTING INSTRUCTIONS


EN Loosen both bands of the brace, gently open the brace and slide over the thumb with the bands at the back of the hand. Using the thumb and forefinger of the other hand, shape the metal insert to fit comfortably around the base of the thumb.

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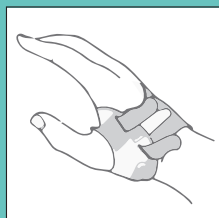
EN Keeping your fingers and thumb slightly bent and in a functional position, secure the band in the palm of your hand and then adjust the two upper bands for support and comfort.

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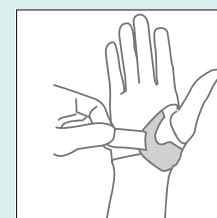EN Now adjust the band in the palm to fit the thumb comfortably. Spend a little time finding a balance between comfort and support. After the first few uses this will become easier to achieve.

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