

THERMOLAX™ PET HEATING PAD

Safety & Use Guide

Gentle Far Infrared Comfort for Dogs & Cats

ThermoLax™ combines gentle far infrared (FIR) warmth with Jade and Tourmaline stones to create a soothing, comfortable resting experience for your pet.

ThermoLax™ is designed for comfort and relaxation—not to diagnose, treat, cure, or prevent any disease or medical condition.

IMPORTANT SAFETY INFORMATION

Always supervise your pet while using the ThermoLax™ heating pad. Pets can become very comfortable and may remain on a warm surface longer than is appropriate. Prolonged exposure to heat can cause overheating or burns, especially in senior, ill, very young, sedated, or mobility-impaired pets that may have difficulty moving away from the heat.

Before Each Use

- Inspect the pad, controller, power cord, and plug for damage.
- Place the pad on a flat, stable, dry surface.
- Make sure your pet can easily move on and off the pad at all times.
- Use the lowest comfortable heat setting when beginning.
- Allow your pet to become accustomed to the warmth gradually.
- Always follow the temperature and session-time limits provided with your specific ThermoLax™ model.

NEVER:

- Never leave your pet unattended while the heating pad is operating.
- Never use the heating pad inside a closed crate or carrier where your pet cannot move away from the heat.
- Never place the pad directly against your pet's skin unless your specific ThermoLax™ pet model is expressly designed and tested for direct-contact use.
- Never use a damaged cord, controller, plug, or heating pad.
- Never allow your pet to chew or scratch the power cord or controller.
- Never cover the controller or block ventilation openings.
- Never use the pad around water or on a wet surface.
- Never force a pet to remain on the heating pad.
- Never use the heating pad as a substitute for veterinary care.

EXTRA CAUTION IS REQUIRED FOR:

- Pregnant or nursing pets
- Very young or elderly pets
- Pets with limited mobility or difficulty getting up
- Pets with reduced sensation or neurological problems
- Pets with diabetes or circulation problems

- Pets with an illness, injury, open wound, skin condition, or recent surgery
- Pets taking medication that may affect alertness or temperature regulation
- Pets recovering from anesthesia or sedation

Consult your veterinarian before use if any of the above applies. Pets with reduced mobility may be particularly vulnerable because they may not be able to move away from heat when necessary.

WATCH YOUR PET

During use, watch for signs that your pet is becoming too warm or uncomfortable.

- Pants excessively
- Becomes restless or repeatedly tries to move away
- Appears unusually tired, weak, confused, or unresponsive
- Shows signs of discomfort
- Has unusually red or irritated skin
- Attempts to leave the pad repeatedly

If you are concerned that your pet has overheated or sustained a burn, discontinue use and contact your veterinarian promptly.

HOW TO USE THERMOLAX™

- 1. Start Low** — Begin with the lowest comfortable setting.
- 2. Start Gradually** — For the first few sessions, use a short session and observe how your pet responds.
- 3. Provide an Escape** — Your pet should always have a comfortable, unheated area immediately available.
- 4. Stay Nearby** — Remain with your pet throughout the session.
- 5. Check Your Pet** — Monitor your pet's behavior and comfort during use.
- 6. Turn It Off** — Turn the ThermoLax™ off and unplug it after each session.

JADE & TOURMALINE

ThermoLax™ incorporates Jade and Tourmaline stones within the heating pad. The stones are part of the ThermoLax™ design and are intended to complement the gentle warming experience.

IMPORTANT

ThermoLax™ is intended to provide gentle warmth and relaxation. It is not a veterinary medical device and is not intended to diagnose, treat, cure, or prevent any disease or medical condition.

If your pet has pain, an injury, illness, mobility problem, or other health concern, consult your veterinarian before using a heating product.

When in doubt, ask your veterinarian.

THERMOLAX™

Far Infrared Heat • Jade • Tourmaline

www.thermolaxheat.com • (760) 294-6252 • thermolax@gmail.com