

PAIN

**INTER-
RUPTED**

LIFE

TRANSFORMED



ST. JUDE MEDICAL™

IMAGINE YOUR LIFE TRANSFORMED

If you or a loved one would like to experience a life transformed then it may be time to consult with your physician and consider the latest advancement in neurostimulation therapy: **BurstDR™ stimulation from St. Jude Medical.**

IT'S TIME FOR SOMETHING DIFFERENT.

Neurostimulation, also called spinal cord stimulation or SCS, has been recommended by doctors for over 40 years to manage chronic pain in the back, arms or legs.

BurstDR™ stimulation* is a new form of neurostimulation therapy. By understanding how the brain naturally manages pain, doctors created a new therapy that works by mimicking natural patterns found in the brain.³ **The result: relief from pain and the chance to reclaim your life.**^{1,2}



Sam B. | BurstDR™ stimulation patient 18 months after implant and able to keep things shipshape on the waters of San Francisco Bay.

The majority
of patients say
they prefer


BURSTDR™
STIMULATION^{1,2}



HOW BURSTDR™ STIMULATION WORKS.

When you feel chronic pain, it is because your nerves are sending pain signals to your brain. BurstDR™ stimulation works to reduce pain by altering the pain signals as they travel to the brain.³

- 1 Pain signals travel up the spinal cord to the brain.
- 2 A generator, similar to a cardiac pacemaker, sends BurstDR stimulation pulses to a thin wire called a lead.
- 3 The lead delivers these pulses to nerves along the spinal cord.
- 4 The pulses modify the pain signals as they travel to different parts of the brain.
- 5 The pulses change the way your body perceives pain—providing potential relief from the physical pain as well as the suffering[†] associated with pain.^{1,2}

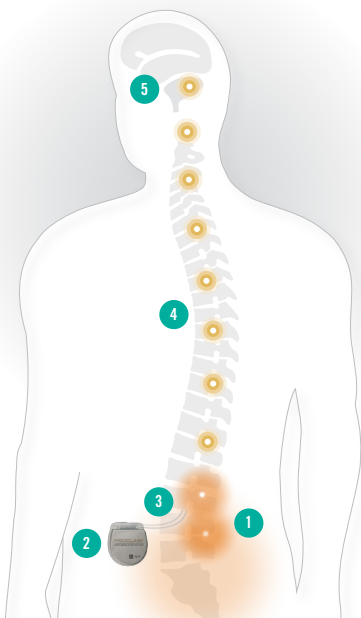



ST. JUDE MEDICAL™ INVISIBLE TRIAL SYSTEM

TRY BURSTDR™ STIMULATION BEFORE YOU COMMIT.

If your doctor decides you are a candidate for BurstDR™ stimulation, you can try the therapy using a temporary system. During the evaluation period, you will be able to assess how well BurstDR stimulation controls your pain throughout the day and during different activities. If the evaluation period is successful, you can have the system implanted.

PROVEN SUPERIOR.

BurstDR stimulation has been clinically proven in numerous studies around the world to:

- Provide superior relief from pain and suffering[†] compared to traditional neurostimulation therapy.^{1,2}
- Reduce or eliminate the tingling sensation felt with traditional neurostimulation.^{1,2}
- Improve patients' ability to perform everyday activities.^{†,1,2}

POTENTIAL RISKS AND COMPLICATIONS.

The placement of the leads is a surgical procedure that exposes you to certain risks. Complications such as infection, swelling, bruising and possibly the loss of strength or use in an affected limb or muscle group (i.e. paralysis) are possible. Be sure to talk to your doctor about the risks associated with the placement of a neurostimulation system.

[†]Pain and suffering as measured by VAS.

KNOWLEDGE NEVER HURTS.

To learn more about **BurstDR™** stimulation, please complete this card and return postage-free. Or visit our patient website PowerOverYourPain.com to submit your request online.

name

address

city

state | zip

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
phone

email

CONTACT ME. Please have a member of the St. Jude Medical clinical team contact me with additional information.

RECEIPT OF FUTURE UPDATES.

I would like to receive future updates about neurostimulation therapy.



NO POSTAGE
NECESSARY
IF MAILED
IN THE
UNITED STATES

BUSINESS REPLY MAIL


FIRST-CLASS MAIL

PERMIT NO. 3742

PLANO, TX

POSTAGE WILL BE PAID ADDRESSEE

ATTN: PREP REQUEST
ST JUDE MEDICAL
6300 BEE CAVE ROAD
BUILDING TWO, SUITE 100
AUSTIN, TEXAS 78746

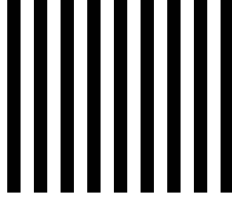

*BurstDR™ stimulation, patented technology exclusively from St. Jude Medical,™ is also referred to as Burst stimulation in clinical literature.

†Pain and suffering as measured by VAS.

‡Based on PGIC scores of moderately better improvement or higher.

1. St. Jude Medical™ Proclaim™ Neurostimulation System Clinician's Manual. Plano, TX. 2016.
2. St. Jude Medical™ Prodigy™ Neurostimulation System Programming and Reference Manual. Plano, TX. 2016.
3. De Ridder, D., Vanneste, S., Plazier, M., & Vancamp, T., (2015). Mimicking the Brain: Evaluation of St. Jude Medical's Prodigy Chronic Pain System with Burst Technology. *Expert Review of Medical Devices*, 12(2), 143–150.

Rx Only

Brief Summary: Prior to using these devices, please review the Instructions for Use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events and directions for use.

Indications for Use: Spinal cord stimulation as an aid in the management of chronic, intractable pain of the trunk and/or limbs, including unilateral or bilateral pain associated with the following: failed back surgery syndrome and intractable low back and leg pain. **Contraindications:** Patients who are unable to operate the system or who are poor surgical risks, with multiple illnesses, or with active general infections should not be implanted. **Warnings/Precautions:** Diathermy therapy, implanted cardiac systems, magnetic resonance imaging (MRI), explosive or flammable gases, theft detectors and metal screening devices, lead movement, operation of machinery and equipment, postural changes, pediatric use, pregnancy, and case damage. **Adverse Effects:** Painful stimulation, loss of pain relief, surgical risks (e.g., paralysis). User's guide must be reviewed for detailed disclosure.

Unless otherwise noted, ™ indicates that the name is a trademark of, or licensed to, St. Jude Medical or one of its subsidiaries. ST. JUDE MEDICAL and the nine-squares symbol are trademarks and service marks of St. Jude Medical, Inc. and its related companies. © 2016 St. Jude Medical, Inc. All Rights Reserved.

St. Jude Medical Inc.

Global Headquarters
One St. Jude Medical Drive
St. Paul, MN 55117
USA
T +1 651 756 2000
F +1 651 756 3301

St. Jude Medical S.C., Inc.

Americas Division
6300 Bee Cave Road
Bldg. Two, Suite 100
Austin, TX 78746
USA
T +1 512 286 4000
F +1 512 732 2418

SJM Coordination Center BVBA

The Corporate Village
Da Vincilaan 11-Box F1
B-1935 Zaventem, Belgium
T +32 2 774 68 11
F +32 2 772 83 84

PowerOverYourPain.com



ST. JUDE MEDICAL