



Cleaning and Disinfecting the Under the Weather® IntubationPod™

We recommend following the CDC.gov guidelines for cleaning and disinfecting IntubationPod™ enclosures after each use. [Click here for details](#)

All surfaces of the IntubationPod should be cleaned and disinfected before and after each use with germicidal disinfecting wipes or EPA-registered disinfectants.

- When using an EPA-registered disinfectant (List N), follow the label directions for safe, effective use. Be sure to follow the contact time, which is the amount of time the surface should be visibly wet. [EPA-registered disinfectants](#)

How to clean and disinfect the IntubationPod:

- Wear disposable gloves to clean and disinfect.
- Practice routine cleaning of frequently touched surfaces.
- More frequent cleaning and disinfection may be required based on level of use.

The CDC also recommends cleaning surfaces using soap and water and following with disinfectant to reduce the number of germs, dirt and impurities and to kill germs on surfaces.

Diluted household bleach solutions may also be used if appropriate for the surface.

- Unexpired household bleach will be effective against coronaviruses when properly diluted.
- Check the label to see if bleach is intended for disinfection and polyester/plastic surfaces
- Follow manufacturer's instructions for mixing measurements, application and proper ventilation.