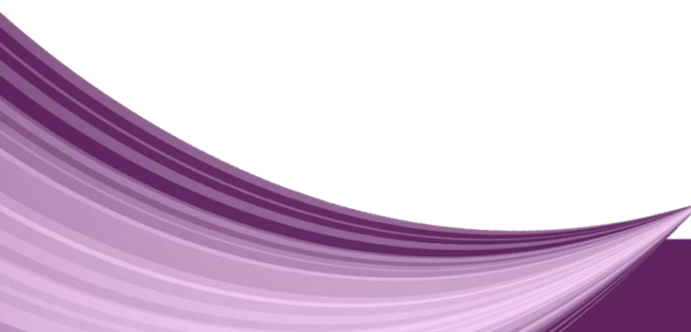


## **Handling of The Cases**

- Remove the instrumentation access cover pressing lightly the locking system between the case and the cover.
  - To close the case, position the cover on the base and press on it.
  - For sterilization, the disinfected, clean and dry instruments should be conditioned in the case, which should be packed in surgical grade crepe paper or in plastic bags suitable for autoclave.
  - During the sterilization, the case should not lean on the autoclave walls.
- 

# Sanitation Instructions

1. Disassemble the case
2. Submerge the part fully into enzymatic detergent (diluted according to the manufacturer).
3. Wash in ultrasonic washer for approximately 10 to 15 minutes.
4. Rise with plenty of distilled water until the solution wastes are fully removed The utilization of nylon brushes is recommended.
5. Dry with a clean and dry cloth or with compressed air.
6. Perform a visual inspection, watching if there are faults with the cleaning process. If any wastes are still left, the part should be submerged again in detergent- step 1- and if necessary, the cleaning should be made with the help of a nylon brush. Repeat the rinsing and drying sequence.
7. If applicable, place the product in a suitable case duly sanitized and dried.
8. Select the package according to the sterilization process. It is recommended to use a surgical grade paper self-sealing package with laminated film.

**NOTE:** Do not use descaling products and avoid drying parts still containing cleaning solution wastes because such procedures favor oxidation. The use of steel brushes is not recommended.

# **Sterilization Guidelines**

- It is recommended to follow the steam autoclave sterilization method and also, the parameters and procedures established in the standard BS EN ISO 17665-1:2006 (Sterilization of health care products. Moist heat. Requirements for the development validation and routine control of a sterilization process for medical devices).
  - Recommended parameters: 121°C, 30 min, at 1 atm. Alternatively you may proceed according to the autoclave manufacturer's instruction. Sterilization validity: 7 to 15 days, provided that the product is conditioned in a clean, dry environment away from sunlight.
- 