


➤➤ **Medentika** & Neodent  
a better **solution** ◀◀

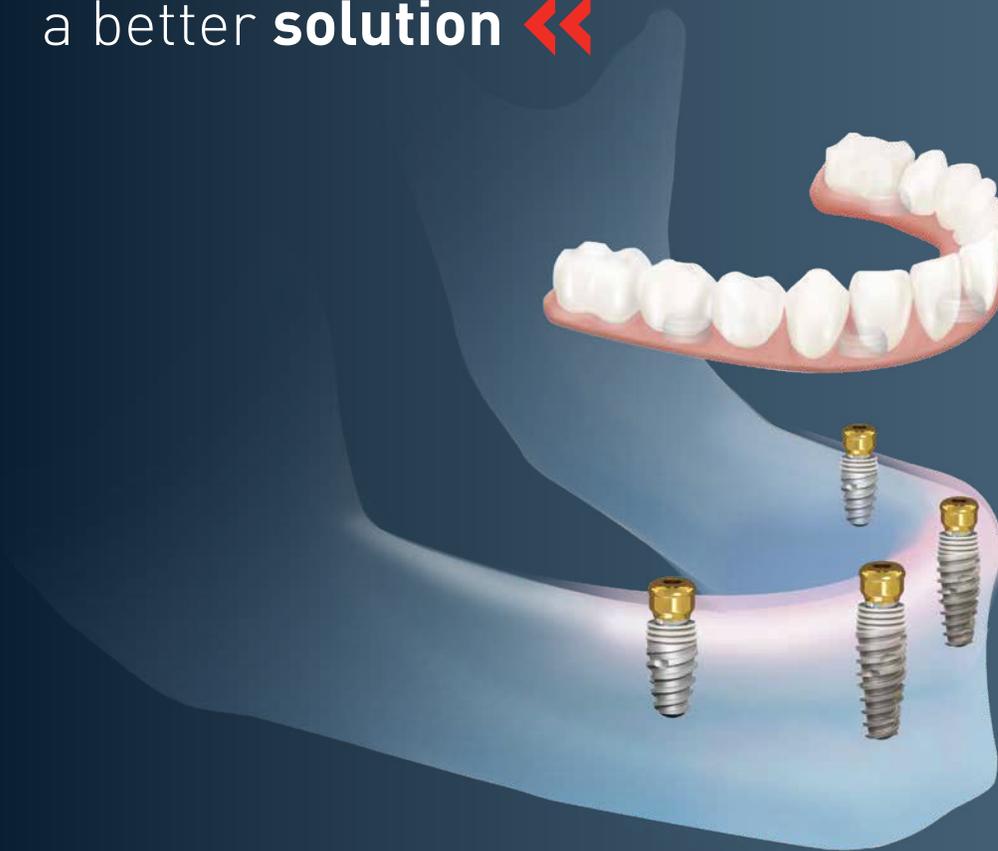

MedentiLOC abutments are Locator® style abutments available for the Neodent CM implant line. Available in various gingival heights. Compatible with Locator® housings.

Novaloc housings are designed to provide longer functionality by utilizing a PEEK insert. Compatible with Locator® and MedentiLOC® abutments.

# »» The top 10 Reasons to choose **Novaloc™** ««

## 1 Longer Life Span & Better Hygiene

The retention inserts are made of high-performance polymer PEEK and can optimally absorb lateral pressure with the patented design. The radial anchoring technology prevents damage to the retention inserts.




## 2 Outstanding Handling

Novaloc® retention inserts can be inserted and removed within 5 seconds. Accessories such as the very low forming matrix and the easy to use matrix housing extractor guarantee a stress-free handling.




## 3 Variable Angled Compatibility

You have the choice between 6 retention inserts with retention forces from ~0.5lbs to ~5.5lbs, including retention performance for divergences up to 20 degrees per implant site.

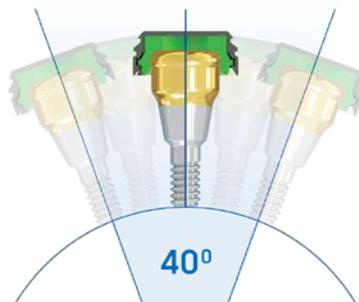


## 4 Less Retention Loss

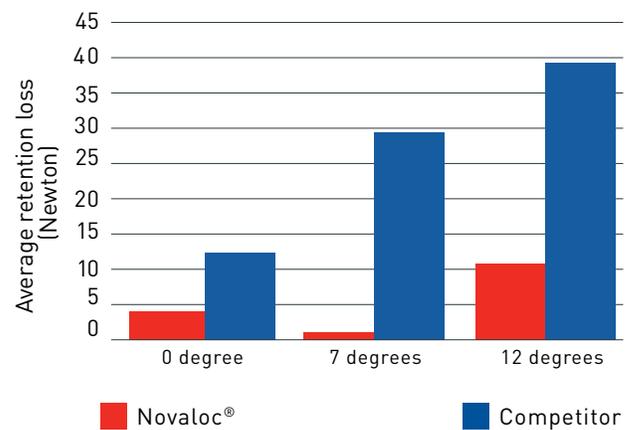

The retention inserts convince with low wear even with strong tangential friction caused by the divergent arrangement of the abutments.


Reference: Rianne Biemans (2013): Retentieverlies bij matrixsystemen voor de overkappingprothese. Nijmegen: Radboud University Nijmegen Medical Centre

## 5 Clearly Less Wear and Tear

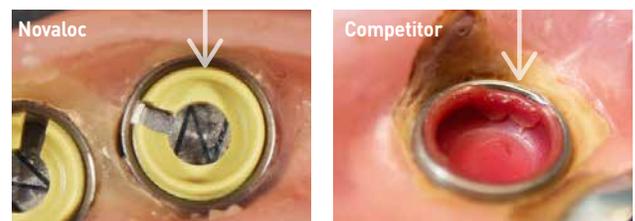

The harder PEEK liner is designed to engage more precisely, and is less susceptible to deforming.

## 6 Only Four Replacement Males

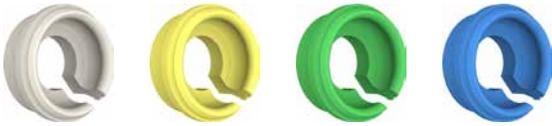

Novaloc® only requires 4 replacement males in 80% of all cases. Easy to understand color code according to retention value. The lighter the color, the lighter the retention. The darker the color, the heavier the retention.

## 7 New Forming / Fixing Matrix

Used for a safe and accurate positioning in the mouth during impression taking. Also, its smaller size, and strong retention makes it versatile. The multi-purpose forming/fixing matrix can be used in base plates, denture try-ins, and bite blocks as well.



## 8 5-second Matrix Housing Extractor



Special design: No more hot fingers!



1. Heat extraction head



2. Heat matrix housing briefly



3. Apply leverage to remove matrix housing

## 9 Reposition Aid and Demounting Tool

**1. Use the stainless steel finish**

**2. Insert at an angle into the bole of the mounting**

**3. Pivot the tip back to easily pop out and remove the mounting insert from the housing.**

**1. Mount model analogue**

**2. Insert model analogue**

**3. Absolutely safe handling**

## 10 The User-Friendly Equipment Box



- No extra caps in inventory
- Ensures overview and order
- No mixing when the cover is closed
- Ordering number legible in the cover
- Three special instruments for all sorts of manipulations

# MedentiLOC for Neodent CM Implant Line

	Part No.	Description	Specifications	Amount Per Pkg
	Y 2000 Y 2100 Y 2200 Y 2300 Y 2400	MedentiLOC abutment straight 1.0mm 2.0mm 3.0mm 4.0mm 5.0mm	<ul style="list-style-type: none"> <li>• Titanium grade 5 KV</li> <li>• Titanium nitride coated</li> <li>• 3.75mm Diameter</li> <li>• Uses 1.26mm driver</li> </ul>	1 pc
	2010.101	Novaloc® Equipment Box	Incl. 3 tools (without load)	1 pc
	2013.901	Altus™ Gingiva Height Measuring Tool	Stainless Steel, PEEK	1 pc
	2010.601	Novaloc® Processing package titanium	<ul style="list-style-type: none"> <li>• Titanium matrix housing (1 pc)</li> <li>• Retention insert white (1 pc)</li> <li>• Retention insert yellow (1 pc)</li> <li>• Retention insert green (1 pc)</li> <li>• Mounting collar, silicone (1 pc)</li> </ul>	2 pcs
	2010.611	Novaloc® Processing package PEEK	<ul style="list-style-type: none"> <li>• PEEK matrix housing (1 pc)</li> <li>• Retention insert white (1 pc)</li> <li>• Retention insert yellow (1 pc)</li> <li>• Retention insert green (1 pc)</li> <li>• Mounting collar, silicone (1 pc)</li> </ul>	2 pcs
	2010.701	Novaloc® Matrix housing, titanium (incl. mounting insert)	Matrix housing: titanium Mounting insert: POM	4 pcs
	2010.702	ovaloc® Matrix housing, titanium (incl. mounting insert)	Matrix housing: PEEK Mounting insert: POM	4 pcs
	2010.703 special accessory	Novaloc® Matrix housing, titanium with extended attachment option (incl. mounting insert)	Matrix housing: titanium Mounting insert: POM	4 pcs
	2010.710 special accessory	Novaloc® Retention insert red	PEEK Retention force: ~0.5 lbs	4 pcs
	2010.711	Novaloc® Retention insert white	PEEK Retention force: ~1.5 lbs	4 pcs
	2010.712	Novaloc® Retention insert yellow	PEEK Retention force: ~2.5 lbs	4 pcs
	2010.713	Novaloc® Retention insert green	PEEK Retention force: ~3.5lbs	4 pcs

	Part No.	Description	Specifications	Amount Per Pkg
	2010.714	Novaloc® Retention insert blue	PEEK Retention force: ~4.5 lbs	4 pcs
	2010.715 special accessory	Novaloc® Retention insert black	PEEK Retention force: ~5.5 lbs	4 pcs
	56	Novaloc® Model analogue	Stainless steel	1 pc
	2010.722	Novaloc® Forming / fixing matrix	PEEK	4 pcs
	2010.723	Novaloc® Processing spacer, white	POM C	4 pcs
	2010.724	Novaloc® Mounting collar, silicone	Silicone	10 pcs
	2010.725	Novaloc® Mounting/processing insert	PEEK	4 pcs
	2010.731	Novaloc® Demounting tool for mounting inserts + model analogue reposition aid (blue)	Aluminium, steel	1 pc
	2010.741	Novaloc® Mounting and demounting tool for retention inserts (brown)	Aluminium, steel	1 pc
	2010.751	Novaloc® Matrix housing extractor (grey)	Aluminum, steel	1 pc
	M 03-1 M 09-1 M 10-1 M 11-1	Placement instrument Hex 1.26 Contra-angle long Contra-angle short Manual and ratchet long Manual and ratchet short	Versions: Contra-angle, Manual and ratchet Manual and ratchet Hardened stainless steel	1 pc
	M 12	Torque ratchet	<ul style="list-style-type: none"> <li>• With infinitely variable torque setting</li> <li>• 10-40 Ncm</li> <li>• Hardened stainless steel</li> </ul>	1 pc
	USK2010.203	Complete Novaloc Starter Kit	Includes: 2010.101    2010.711    2010.722 2010.701    2010.712    2010.723 2010.702    2010.713    2010.724 2010.710    2010.714    56 x 4 2010.711    2010.715	1 Kit
	USK2010.103	Beginner Novaloc Starter Kit	Includes: 2010.103 2010.601 x 2 (4 pks total)	

Medentika is distributed by:

Instradent USA, Inc.  
60 Minuteman Road  
Andover, MA 01810

Phone: 855-412-8883  
Fax: 885-412-8884  
info.us@instradent.com  
www.instradent.us

»» Passion for  
Precision ««



[www.medentika.us](http://www.medentika.us) 855-412-8883 toll free



Products indicated with © are registered brand names of respective manufacturers.