



A NEW MINDSET

*NEODENT[®] ZIRCONIA IMPLANT SYSTEM
WORKING PROTOCOL*



Zirconia
Implant System



ZiLock[™]
C connection

SUMMARY

| Zi Implant Ø3.75 mm working protocol

| Zi Implant Ø4.3 mm working protocol

| Prosthetic procedures



Ø3.75

SURGICAL PROCEDURES

Implant lengths



- 10.0 180.002
- 11.5 180.003
- 13.0 180.004

Drills sequence for all bone type



- 1 Initial (optional)
- 103.170



- 2 Ø2.0
- 103.425



- 3 Ø3.5
- 103.562 short
- 103.561 medium
- 103.563 long



- 4 Ø3.75
- 103.565 short
- 103.564 medium
- 103.566 long

Direction Indicator



- Ø3.0/3.75
- 128.020

Drills sequence for types I, II and III bones



- 5 Countersink Ø3.75
- 103.488

Drill sequence for types I and II bones



- 6 Bone Tap Ø3.75
- 111.046

Healing phase



- Zi Cover Screw
- 117.023



- Zi Healing Abutment
- 106.233 3.75x1.5 mm
- 106.234 3.75x2.5 mm
- 106.235 4.5x1.5 mm
- 106.236 4.5x2.5 mm

Drivers and Torque Wrench



- Neo Screw Driver Manual
- 104.058 short
- 104.060 medium
- 104.059 long



- Implant Driver for Contra-Angle
- 105.002



- Implant Driver for Torque Wrench (short)
- 105.001



- Torque Wrench
- 104.050

Implant lengths



- 10.0 180.006
- 11.5 180.007
- 13.0 180.008

Drills sequence for all bone type

 1 Initial (optional) 103.170	 2 Ø2.0 103.425	 3 Ø3.5 103.562 short 103.561 medium 103.563 long	 4 Ø4.3 103.571 short 103.570 medium 103.572 long
---	---------------------------------	--	--

Direction Indicator



Ø3.6/4.3
128.022

Drills sequence for types I, II and III bones



5
Countersink Ø4.3
103.450

Drill sequence for types I and II bones



6
Bone Tap Ø4.3
111.048

Healing phase



Zi Cover Screw
117.023



Zi Healing Abutment
 106.233 3.75x1.5 mm
 106.234 3.75x2.5 mm
 106.235 4.5x1.5 mm
 106.236 4.5x2.5 mm

Drivers and Torque Wrench



Neo Screw Driver Manual
 104.058 short
 104.060 medium
 104.059 long



Implant Driver for Contra-Angle
105.002



Implant Driver for Torque Wrench (short)
105.001



Torque Wrench
104.050

Abutment selection



- Zirconia Base**
- 135.254 3.75x4x1.5 mm
 - 135.255 3.75x4x2.5 mm
 - 135.256 4.5x4x1.5 mm
 - 135.257 4.5x4x2.5 mm



- CR Abutment**
- 114.854 4.0x5.0x1.5 mm
 - 114.855 4.0x5.0x2.5 mm
 - 114.856 4.5x5.0x1.5 mm
 - 114.857 4.5x5.0x2.5 mm



- CR Abutment 17°**
- 114.858 4.0x5.0x1.5 mm
 - 114.859 4.0x5.0x2.5 mm
 - 114.860 4.5x5.0x1.5 mm
 - 114.861 4.5x5.0x2.5 mm



- CR Abutment Analog**
- 4.0x5.0 mm
 - 101.105
 - 4.5x5.0 mm
 - 101.106



- CR Abutment Burn Out coping**
- 4.0x5.0 mm
 - 118.367
 - 4.5x5.0 mm
 - 118.368

Impression



- Implant Scanbody**
- 108.184



- | | regular | long |
|---------|---------|------|
| 108.186 | 108.187 | |
| 108.188 | 108.189 | |

Model production



- Hybrid Repositionable Analog**
- 101.080

Temporary




- PEEK CR abutment**
- 114.888 4.0x5.0x1.5 mm
 - 114.889 4.0x5.0x2.5 mm
 - 114.886 4.5x5.0x1.5 mm
 - 114.887 4.5x5.0x2.5 mm



- Impression Coping / Provisional Coping**
- 108.201 4.0x5.0 mm
 - 108.202 4.5x5.0 mm

Final coping

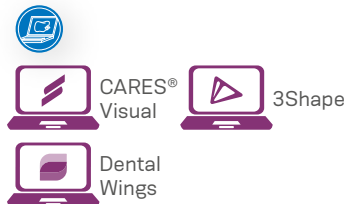

- Burn-out Coping**
- 118.343 3.75x4 mm
 - 118.325 4.5x4 mm

Drivers



- Neo Screw Driver Manual**
- 104.058 short
 - 104.060 medium
 - 104.059 long



- Neo Screwdriver torque wrench**
- 105.133 short
 - 105.132 medium
 - 105.134 long

毎日、とびっきりの笑顔を

NUOVI SORRISI OGNI GIORNO

CHAQUE JOUR DE NOUVEAUX SOURIRES

новые улыбки каждый день

JEDEN TAG EIN NEUES LÄCHELN

NUEVAS SONRISAS TODOS LOS DÍAS

NYA LEENDEN VARJE DAG



A Straumann Group Brand

© Neodent® 2022. All rights reserved.

Neodent®, Zi™ and ZiLock™ are trademarks or registered trademarks of JJGC Indústria e Comércio de Materiais Dentários S.A. CARES is a trademark or registered trademark of Straumann Holding AG. 3Shape is a trademark or registered trademark of 3Shape A/S. exocad is a trademark or registered trademark of exocad GmbH. Dental Wings is a trademark or registered trademark of Dental Wings Inc. The reproduction of this material without prior authorization is prohibited. Some products may not be available for purchase yet. For more informations, please contact your local distributor.

Neodent® and/or other trademarks and logos from Neodent® that are mentioned herein are the trademarks or registered trademarks of Straumann Holding AG and/or its affiliates. All rights reserved.

NOVOS SORRISOS TODOS OS DIAS

SOURIRES



Straumann North American Headquarters
Straumann USA, LLC
60 Minuteman Road
Andover, MA 01810
Phone 800/448 8168 (US) • 800/363 4024 (CA)
Fax 978/747 2490
www.straumann.us • www.straumann.ca

ifu.neodent.com.br/en
www.neodent.us • www.neodent.ca

USLIT.2140 4/22 V1