




DirecFit Screw: Direct-To-MUA screw

Library: www.Straumann.us/connectivity

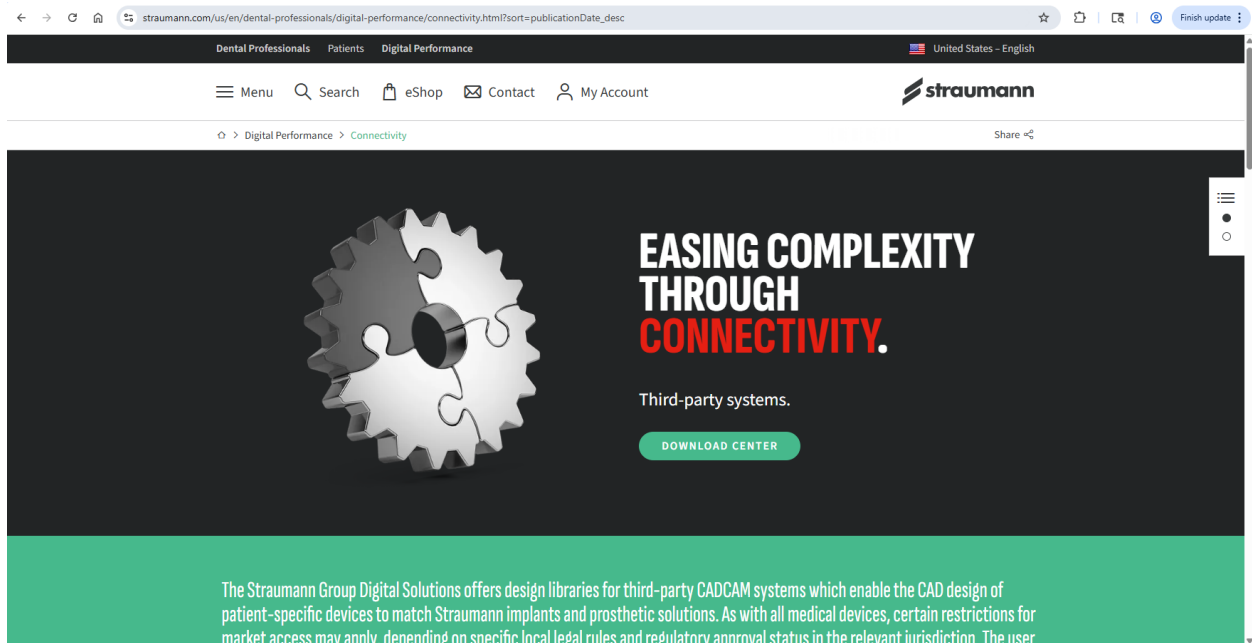

Select Neodent, then download MUA analog folder and Neodent Local milling

Makes sure you are downloading files for EXOCAD

Digital Performance > Connectivity Share

Search **Step 2** Select all Neodent Clear filters

MUA analog files
DirectFit Files

Step 1

- Partner Company
 - 3Shape Dental System (12)
 - exocad (8)
- Country
 - Global (15)
 - Canada (13)
 - United States of America (9)
- Company Brand
 - Medentika (81)
 - Straumann (63)
 - Neodent (20)
 - Nuvo (8)
- Workflow
- Prosthetic Area
- Product Reference

Neodent Model Analogs (local printing) exocad - 9.0 02/25/2026 ZIP, 8.9 MB	Download
Neodent (local Milling) exocad - 9.0 02/25/2026 ZIP, 47.4 MB	Download
Neodent local Milling 3Shape DS 2024 - 8.0 02/19/2026 ZIP, 37.0 MB	Download
Neodent local Milling 3Shape DS 2023 - 8.0 02/19/2026 ZIP, 37.0 MB	Download
Neodent local Milling 3Shape DS 2022 - 8.0 02/19/2026 ZIP, 37.0 MB	Download
Neodent centralized Milling - IIC	

ml?sort=publicationDate_desc&tags=organizationalInfo%3AcompanyBrand%2Fneodent&pageOffset=0

Contact My Account




Recent download history

- Neodent_Analogs_local-printing (1).zip
8.9 MB • 2 minutes ago
- Neodent_local-milling (2).zip
47.4 MB • 9 hours ago

Full download history

Select all Neodent Clear filters

Neodent Model Analogs (local printing) exocad - 9.0 02/25/2026 ZIP, 8.9 MB	Download
Neodent (local Milling) exocad - 9.0 02/25/2026 ZIP, 47.4 MB	Download

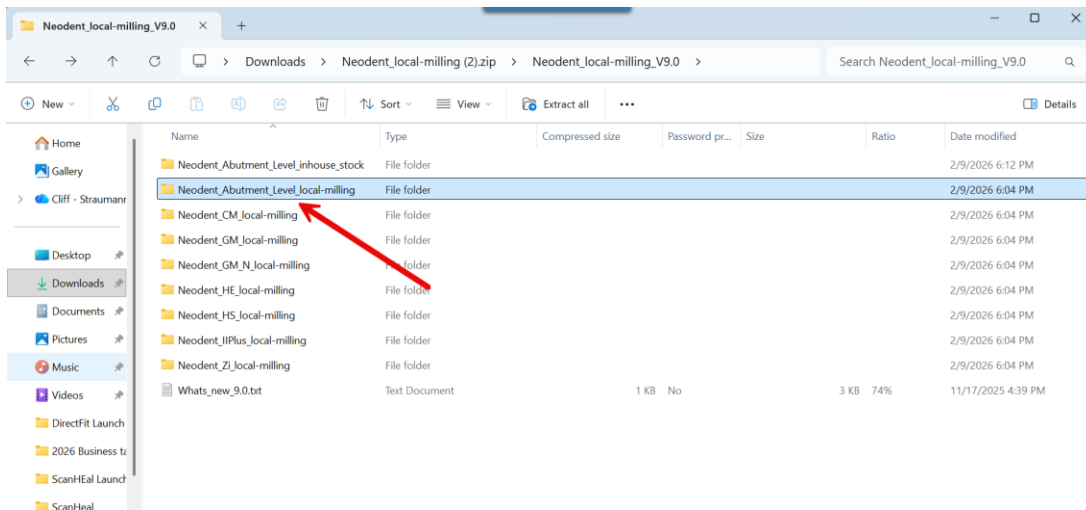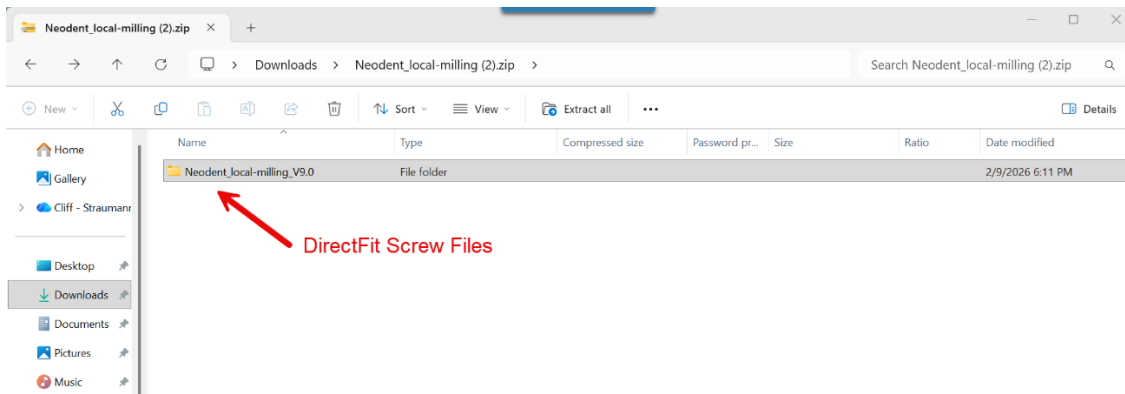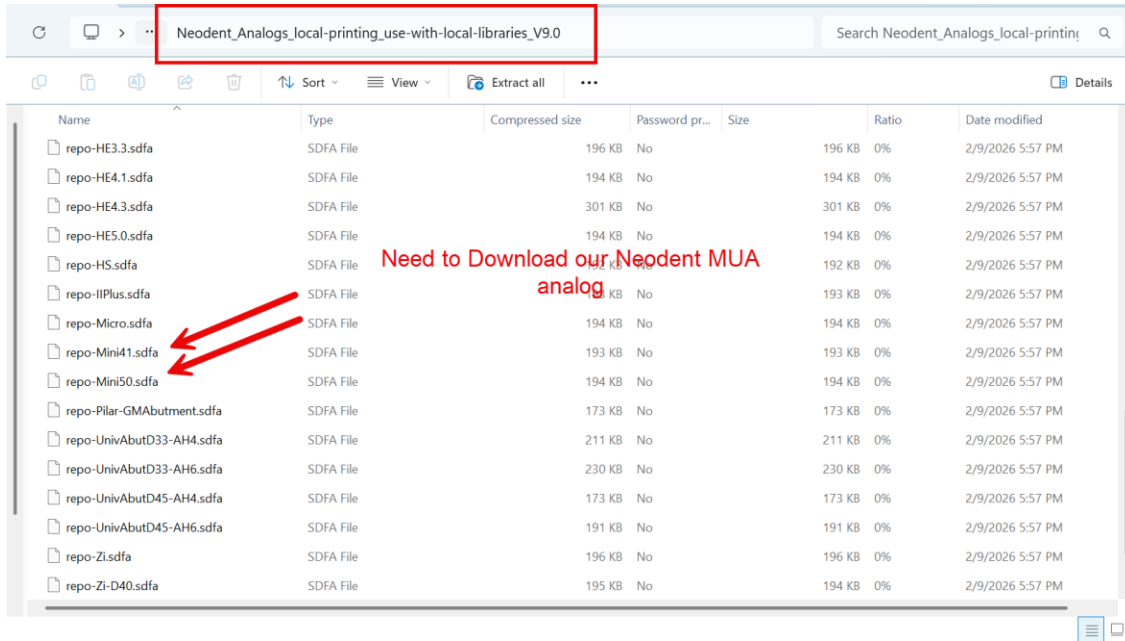

Mini-Conical Abutment is our standard size MUA


This is our 3:5 SMALL restorative shoulder MUA Micro-conical abutment

Horizontal thickness around Direcfit screw -Zirconia-0.4mm -PMMA-1.0mm


Name	Type	Compressed size	Password pr...	Size	Ratio	Date modified
I_M-Mini50.sdfa.partInfo	PARTINFO File	3 KB	No		4 KB 33%	2/9/2026 3:42 PM
I_M-Pilar-GMabutment.sdfa	S DFA File	173 KB	No		173 KB 0%	2/9/2026 3:42 PM
I_M-Pilar-GMabutment.sdfa.partInfo	PARTINFO File	3 KB	No		4 KB 33%	2/9/2026 3:42 PM
I_M-ZITL-UB.sdfa	S DFA File	196 KB	No		196 KB 0%	2/9/2026 3:42 PM
I_M-ZITL-UB.sdfa.partInfo	PARTINFO File	3 KB	No		4 KB 33%	2/9/2026 3:42 PM
MIN_BODY-LOSHCMini41.sdfa	S DFA File	100 KB	No		100 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Micro.sdfa	S DFA File	30 KB	No		30 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Micro-DirectFitScrew_0.4.sdfa	S DFA File	35 KB	No		35 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Micro-DirectFitScrew_1.0.sdfa	S DFA File	37 KB	No		37 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Mini41.sdfa	S DFA File	No			30 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Mini50.sdfa	S DFA File	31 KB	No		31 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Mini-DirectFitScrew_0.4.sdfa	S DFA File	31 KB	No		31 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Mini-DirectFitScrew_1.0.sdfa	S DFA File	32 KB	No		32 KB 0%	2/9/2026 3:42 PM
MIN_BODY-Pilar_GMAbutment.sdfa	S DFA File	30 KB	No		30 KB 0%	2/9/2026 3:42 PM
ND_Neodent.png	PNG File	23 KB	No		38 KB 40%	2/9/2026 3:42 PM
preview.png	PNG File	99 KB	No		101 KB 3%	2/9/2026 3:42 PM

97 items | 2 items selected 61.5 KB


PLATFORMS: GM Helix®, Helix GM Long, Zygoma GM, GM Zygoma-S




Mini Conical



Helix Short

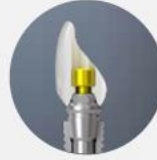

Micro Conical



GM Narrow GM Helix

Chairside or In Lab Digital workflow

GM Mini Conical Abutment (Straight)				GM Exact Mini Conical Abutment (Angled)				
	0.8 mm	1.5 mm	2.5 mm		17°	30°	45°**	
GM Mini Conical Abutment (Straight)	115.243	115.244	115.245		115.275	115.278	115.281	
	3.5 mm	4.5 mm	5.5 mm		115.276	115.279	115.282	
	115.246	115.247	115.248		115.277	115.280		
					45° slim**	52°***	60°***	
					115.302	115.300	115.285	
					2.5 mm	115.303	115.301	115.286