

Important Information : Disposable Face Masks

Instructions for use

To protect your health and the health of others it is important to follow these instructions.

- You should wear this mask when in contact with people **other than those that you live with**.
- Always check that the mask **fits well** and **covers both your nose and mouth**.
- **WARNING:** if you are sick, this mask may not be suitable. **Seek advice from a doctor**.
- This mask is not classified as ppe or for the use of medical staff.
- This Mask does not replace the additional guidelines (the regular **washing of hands, physical distancing, reduced contact with other persons** etc). **It adds a physical barrier when in close contact with other people** and should be used in conjunction with Government guidelines.

How to wear the mask?



WARNING: Before Wearing The MASK

01. Wash your hands with soap and water or an antibacterial gel or spray.
02. Inspect the mask and ensure there are no holes, tears or damage
03. It is recommended to wear the mask on bare skin, avoiding contact with hair.
04. Never modify the mask.

WARNING: Wearing The MASK

01. Hold the mask by the elastic straps.
02. Place it at the level of your mouth.
03. Pass the elastic straps behind the ears.
04. Adjust the mask so that it covers the nose, mouth and chin. You can pinch the nose bridge.

WARNING: Whilst you wear the MASK

01. Avoid touching and moving it.
02. Never put it in the waiting position on the forehead or the chin.

Information:

01. This Mask has demonstrated a filtration efficiency of greater than 99.9% for particles of 3 microns emitted.
02. The breathability of the mask has been measured at >96L.m2 s-1 and conforms to EN14683:2019.
03. This mask should NOT be worn for more than 4 hours.
04. This mask has been produced in Cyprus under the standards of CWA17553:2020.

WARNING: You MUST CHANGE THE MASK

01. When you have worn the mask 4h.
02. When you want to eat or drink.
03. If it becomes difficult to breathe.
04. If the mask gets wet.
05. If the mask is damaged.
06. If the mask becomes distorted and no longer fits properly.

WARNING: To Remove the MASK

01. Wash your hands with soap or a hydro alcoholic solution.
02. Unhook the elastic straps.
03. Remove the mask and dispose of it in an appropriate way.
04. To finish: wash your hands again with soap and water or alcohol-based hand rub.

Disclaimer:

Despite all the care taken in the manufacture of the masks, it belongs to the sole user and in no case to the manufacturer or the supplier to ensure the suitability of the mask with the characteristics of the usage environment. The recommendations and methods of use described above should be followed carefully. This Mask is not suitable for children under 12 years old. This mask is not to be used during exercise. The masks dimensions are 17.5cm x 6.5cm. Each mask has been visually inspected and the ear straps have been tested.