



Tender Cushion Printable Size Stencil

Tender Cushion Open Cell Equine Hoof Pads fit most horses of the size they are meant for; small, medium, or large, without any cutting at all. However, this cutting guide of common shoe sizes is available if needed.

1. Print on standard letter sized 8.5"x11" paper. Print at 100% actual size without stretching to fit the page.
2. Cut out the required stencil size. Remember to cut larger than expected. The size is flexible and does NOT have to be perfect. Avoid pad sticking out in front of the hoof to avoid tripping, but some pad sticking out from the sides and rear is perfect.
3. Check the size of the stencil against the horse's hoof before cutting your Tender Cushion. Drainage holes for abscess and special injuries can be cut into the stencil or just into the pad.
4. Hold the stencil to the Tender Cushion and mark the outline with a pen or marker. Cut the cushion using standard bandage scissors, regular scissors, or any safe cutting tool.