









It is a corner positioning chair which helps children and young people to adopt the correct sitting position with upright legs. It is worth saying that this position enforces the correct alignment of hips thanks to the Efficient Tilt-in-space adjustment

The use in the product of an innovative four-bar linkage, supported with the gas spring, greatly improves smooth Tilt-in-space adjustment..

It is recommended as the first therapeutic chair.

Ergonomic side supports and Efficient Tilt-in-space adjustment

The use in the product of an innovative four-bar linkage, supported with the gas spring, greatly improves smooth Tilt-in-space adjustment. With pelvic belt not only make the patient feel secure but also control the correct posture. Additionally, a vast array of adjustments is beneficial due to the fact that children grow fast.

There are 3 sizes and therefore it is perfect for children as well as young people aged from a few months to several years.

This device is both light and small thanks to which you can easily move the chair into different places.

Size 3 comes standard with braked swivel casters.




IT IS RECOMMENDED IN THE FOLLOWING CASES:

- Cerebral palsy various types and forms.
- Brain and spinal cord injuries with lower limbs paralysis and weakness, balance and motor coordination disorders with different etiology
- Demyelinating disease (e.g. multiple sclerosis SM).
- Meningocele.
- Muscles diseases with weakness and paralysis (dystrophy, myasthenia gravis, myositis).
- · Genetic disorders.
- Degeneration and neurological diseases.
- Other condition connected with muscles weakness, paralysis and musculoskeletal disorders.
- After head and spinal cord injuries and spine trauma with paralysis and paresis.



STANDARD EQUIPMENT:

Base | Seat | Removable abductor | Pelvic support adjustable | Pelvic belt | Chest support adjustable | Chest belt | Castors with brakes (size 3 only)

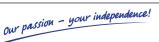

Akces-MED sp. z o.o. Jasionka 955B 36-002 JASIONKA



NOOK



ADVANTAGES:



Stable structure

The device ensures adequate stability and safety of Patient.



Easy to use

Intuitive way of adjustment and easy assembly.



Eco-friendly materials

We use only human and environment friendly materials.



Spacer upholstery

The device is equipped with breathable, hypoallergenic 3D spacer fabric. Now you can be sure that the skin stays dry for longer!



24 months warranty

We offer 24 months warranty on the product. We also provide after sales service.



Ergonomically shaped side supports control the correct position in the chair.



The tray is perfect for different activities e.g. playing or taking meal.



Large, profiled Efficient Tilt-in-space adjustment The use in the product of an innovative four-bar linkage, supported












BREATHABLE SPACER UPHOLSTERY









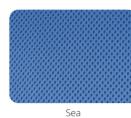





WATERPROOF UPHOLSTERY

Grass









Size 2



Green Grass Optional



Blue Sea Optional



Brown Sugar Optional



Hot Red Optional

EXTREME-MED UPHOLSTERY













Pillarbox *Optional

*Optional MEDICUS™ UPHOLSTERY

Citrus Green





Candy

*Optional





MED Sea Standard



MED Blue *Optional



MED Skin *Optional

Optional colors available for special request. Production time ca. 14 days.




Akces-MED sp. z o.o. Jasionka 955B 36-002 JASIONKA


export@akces-med.com +48 (17) 864 04 77 www.akces-med.com




NOOK



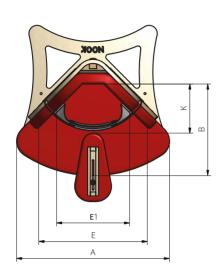

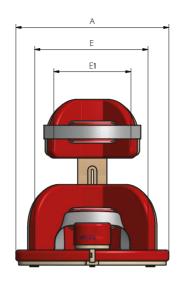

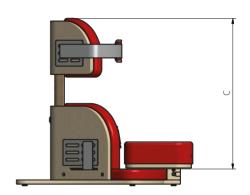

Symbol	Measurement	Unit	Size 1	Size 2	Size 3
А	Seat width	[cm]	50	56	67
В	Seat depth	[cm]	29	32	41
С	Backrest height	[cm]	33-41	47-58	51-66
Е	Pelvic support width	[cm]	26	30	36
E1	Chest support width	[cm]	26	30	40
K	Seat depth to the abduction block	[cm]	12-22	14-24	25-35
	Max user weight	[kg]	25	35	50
	Product dimensions and weight	Unit	Size 1	Size 2	Size 3
	Width	[cm]	50	56	67
	Lenght	[cm]	60	62	72
	Height*	[cm]	38	52	58*
	Weight	[kg]	6	8	11,5

 $^{{\}rm *In}$ case of mounted wheels dimension increases by 8 cm.

Note: Measurement tolerance range is +/- 1 cm / $\frac{1}{2}$ in.

Optional colors available for special request. Production time ca. 14 days.



Akces-MED sp. z o.o. Jasionka 955B 36-002 JASIONKA



NOOK





Safety



Soft cover for tray

Soft cover protects the user from hitting the tray. Indicated for patients with impaired limbs coordination.



Stabilization



Hexagon Pro™ headrest

Headrest adjustable in three dimensions. Due to its special design, it can be precisely adjusted to the user's needs.



Extended headrest

For patients with increased head circumference. It provides better side stabilization.



Head supporting belt

Attachable to the headrest by use of Velcro straps. Used for people with limited control of the head to prevent its falling.



Comfort



Cotton cover for headrest

It helps to keep the headrest clean. Easy to assemble/disassemble, it can be washed



Swivel castors (5pieces)

Mounted to the base. They allow easy movement between rooms.



Tray

Made of wood. It supports the therapy and allows you to play, eat, and work.



Tray for non verbal communication

It is a very important element in the rehabilitation of patients with speech dysfunction and, moreover, it makes it easier to perform daily activities. The set consists of 38 cards that may be put in random order into 24 places of the tray. It is covered with transparent and aesthetic plate made by Plexiglas. Parents and child can create their own symbols that they believe are significant for their communication.