

JOY™

TABLE



USER MANUAL

Our passion - your independence!

AKCES[®]MED



CAUTION! Manufacturer takes responsibility for proper usage of the device only when the product was purchased from qualified Akces-Med employee or at specialized medical store!



CAUTION! AKCES-MED Ltd. reserve the right to introduce technical and trading changes in the content of the instruction without warning.

Akces-Med. Sp. z o.o. Company introduced quality management system ISO 13485 regarding production, sales and service. The system is certified by Det Norske Veritas.



TABLE OF CONTENTS

1. Purpose of the manual.....	5
2. General construction of the table.....	6
3. Size table.....	7
4. Assembly of the board.....	8
5. Description of table adjustment	11
6. Safety rules.....	13



INTRODUCTION

Joy™ table is unique accessory to our Kidoo chair but not only. It can be used along with other chairs, school chairs or office chairs. All of this because of the height adjustment.

Wide tabletop provides a lot of possibilities – this is a place to play, learn, to have a meals or to use a computer. The table can be fitted to every chair, because of adjusted height. The table is stable because of firm construction. The tabletop is smooth and easy to maintain.

Before using the product, user is obliged to familiarize himself/herself with user manual. Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.

If you have any questions or remarks, do not hesitate to contact with us:

Medical consultant: mobile +48 508 382 509

Akces-MED Sp. z o.o.: ph. +48 17 864 04 77

e-mail: export@akces-med.com

Thank you for purchasing our product.



CAUTION! Before using the product, user is obliged to familiarize himself/herself with user manual. Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.

1 PURPOSE OF MANUAL

This manual contains basic information necessary for proper use of the JOY™ table, preparing it for work, and also how to properly maintain and clean it. This manual should be stored in an easily accessible location. It is designed for people who have direct care over people with disabilities and for physicians and physiotherapists using the equipment.



ATTENTION! Before using the JOY™ table the user is obligated to familiarize themselves with the content of this manual.

Please note that following the guidelines contained in this manual will allow you to use the product safely and extend its life and aesthetics.

2 GENERAL CONSTRUCTION OF THE TABLE

2.1 Standard equipment:


1. Wide board
2. Frame with adjustable height



Versions:



3 SIZE TABLE

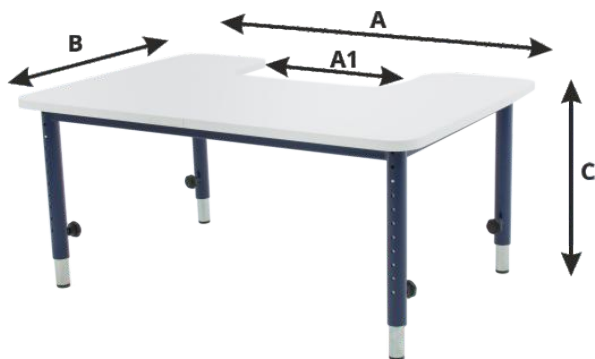

Symbol	Unit	Dimension
A	[cm]	90
A1	[cm]	40
B	[cm]	60
C	[cm]	40-70
Weight	[kg]	11,2
Maximum user weight	[kg]	50

Note: Measurement tolerance range is +/- 1 cm / ½ in

4 ASSEMBLY OF THE TABLE BOARD

4.1. Components:

1. Adjusting the tilting angle of the board (x2)

2. The hinge plate(x2)

3. Wood screw 4x16 (x12)

4. Conical screw M6x30 (x4)

5. Uderlay/washer 6.4 (x4)

6. M6 nut (x6)


7. Screw with Hexagon head - M6x 40 (x2)

8. Washer/underlay - PA 6.4 (x4)

9. Cap of nut M6 (x2)

10. Frame (x1)

11. Board (x1)

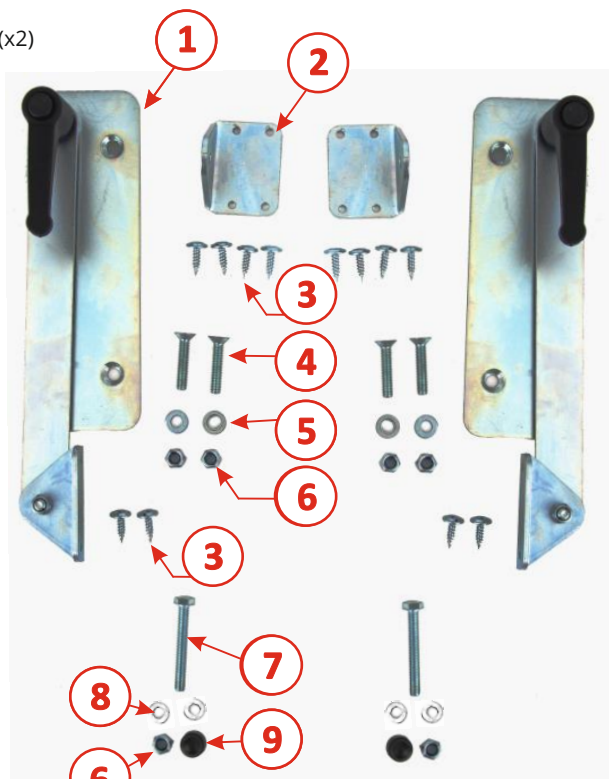


STEP 1

Assembly of the table begins with:

- Fastening the hinge plate (el.2) to the board (fig. 1) with screws (el.3) (fig. 2). Repeat this step on the other side.



- Then adjust the angle of inclination (el. 1) to the frame (fig.3) with screws (el. 4) (fig. 4). Repeat this step on the other side.

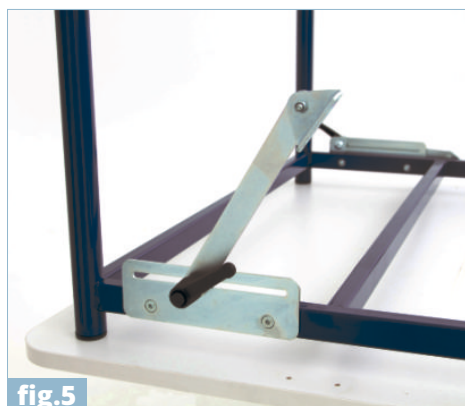

STEP 2

-Then adjust the angle of inclination (el. 1) to the frame (fig.3) with screws (el. 4) (fig. 4).

Repeat this step on the other side.



- Next adjust the angle of inclination to the board using the screws (el.3) (fig. 9,10).



5 DESCRIPTION OF TABLE ADJUSTMENT

5.1. The height of the table

- The height of the table is adjusted. To do that, there is recommended to turn it upside down.

- To start height adjustment, there is necessary to loosen each knob.

- Next, press the button and place the table leg in the required height.

- The final step is tightened tightly all of the knobs.



fig.11



fig.12



fig.13



CAUTION! Before use it is necessary to make sure that all knobs for adjustments are stable and locked!

5.2. Adjusting the tilt angle of the table

The tilt angle of the board can be changed with the handles located on both sides of the table. To change the tilt angle:

- twist the handles,
- adjust the correct angle of the board
- next tighten the handles.



fig.14



fig.15

JOY™ Table can be used with positioning chairs as well as with school chairs.



fig.16

6 SAFETY RULES

1. The product has to be used with compliance to guidelines included in this user manual! Before every using, there is necessary to prepare the patient and the device!

2. Before using the product, it is essential to make sure that there are no damages or different faults which could endanger patient's life. Make sure that all parts are operating correctly and are properly mounted. Moreover, make sure that there are no cracks as well as deformities

3. It is forbidden to use table for different purposes than it is intended to.

4. The table top should be cleaned with soft cloth and publicly available, mild cleaner.

5. Pay your attention during the board tilting. Please do not place the fingers between the board's and table's frame during adjustment.

CAUTION! The product contains small parts which children may swallow.



CAUTION! It is forbidden to make any changes concerning table's structure due to the fact that it is dangerous and it results in losing warranty.

CAUTION! This device can be only used inside the buildings.



PRODUCENT PRZEDMIOTÓW ORTOPEDYCZNYCH



Akces-MED Sp. z o.o.

Jasionka 955B

36-002 Jasionka

tel. (17) 864 04 70

www.akces-med.com

e-mail: produkcja@akces-med.com

Autoryzowany Przedstawiciel



Data wydania: **25.08.2016**

Data ostatniej aktualizacji: **24.04.2017**

Our passion - your independence!