
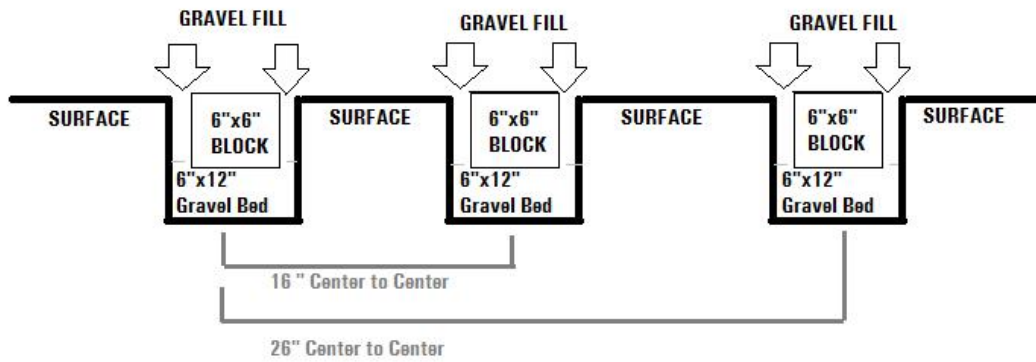


Serenity Healthcare Products SH1 Installation with concrete blocks.

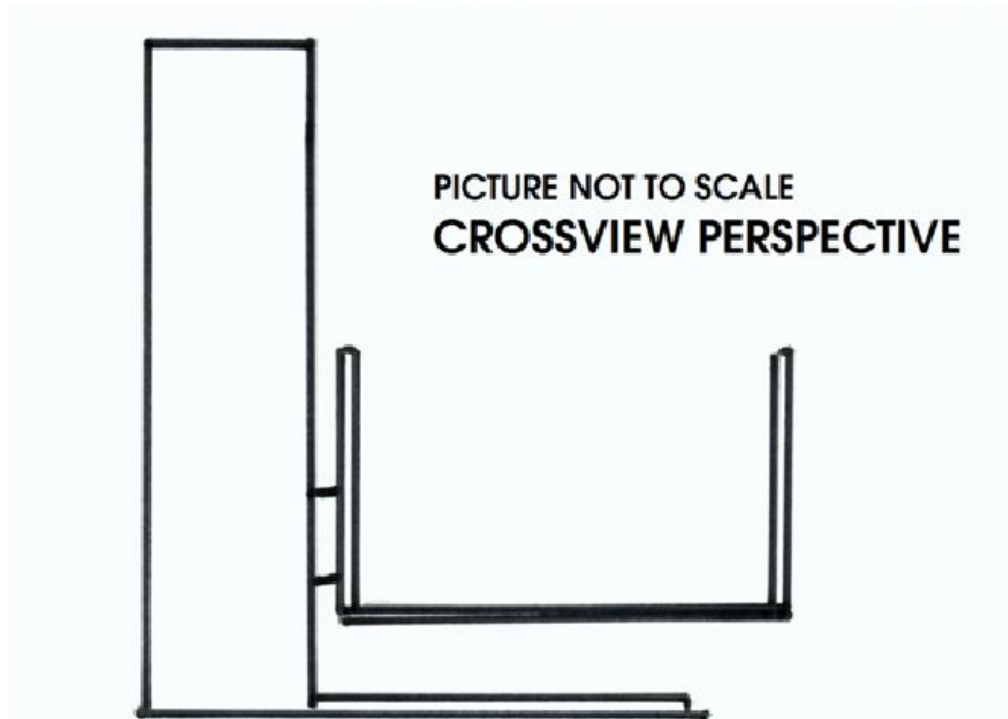

VPL-SH1 INSTALLATION GUIDE

Materials Needed:

3 Driveway Concrete Blocks – 6” x 6” x 39.5” length

6 Bags of Gravel

Directions:

1. Make certain of layout.
2. Dig 3 trenches 12” x 12” by 48” in length as pictured
3. Pour gravel bed 6” x 12” by 48” in length as pictured
4. Lay down 3 concrete blocks, level
5. Set Lift on Blocks. Mounting holes of Lift should be centered on blocks
6. Bolt Lift to Blocks
7. Fill surrounding area with gravel



NOTE: 6 Lag Bolts: 4” in length, ¼” - 5/16” in width, supplied with unit.