BodyMap[®]
VACUUM SYSTEM BY ORTHODESIGN

www.bodmapsystem.com www.reh4mat.com www.bodmapsystem.com www.reh4mat.com www.bodmapsystem.com www.reh4mat.com www.bodmapsystem.com www.reh4mat.com www.bodmapsystem.com

REH4MAT
MANUFACTURER OF MODERN ORTHOPAEDIC BRACES

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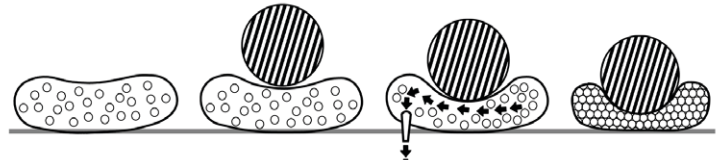
BodyMap System refers to first class medical products which are designed for adults as well as children with cerebral palsy. All these products provide perfect stabilization during seating or lying position. What is more, it is a perfect choice for the disabled due to the fact that it prevents bed sores.

BodyMap System uses vacuum method. This method refers to products which are filled with polystyrene granules thanks to which products can model according to patient's body shape. This method is well-known within The Western European countries and it is widely applied in stabilization process. Furthermore, it is also applied during transportation of patients in ambulances. Polystyrene granules move freely inside the device and, more to the point, it is easy to move them in order to shape cushion at your discretion.

BodyMap products are made of soft, elastic neoprene foam and they perfectly adjust to patient's body. Cushions are laminated by means of elastic polyester knitwear and some of them can be additionally laminated with the aid of knitwear with self-adhesive qualities which makes it possible to use additional stabilizing tapes.

Our system is designed with the intention of maintaining correct sitting. Cushions, backrests and pads make it possible to have correct support for trunk, hips, head and upper as well as lower limbs. BodyMap System is perfect for patients with cerebral palsy, scoliosis or hump. The product is shaped according to patient's body shape thanks to which unit pressure is diminished. It has a significant influence on bed sores prophylaxis. In addition, it is possible to determine the level of product's hardness with the aid of vacuum method.


Because of unique manufacturing process, the shape of our cushions remains the same for few weeks. This characteristic is advantageous because there is no need to pump the air down every few days.

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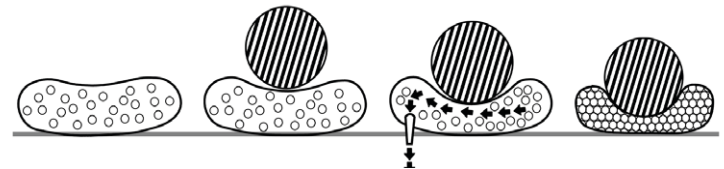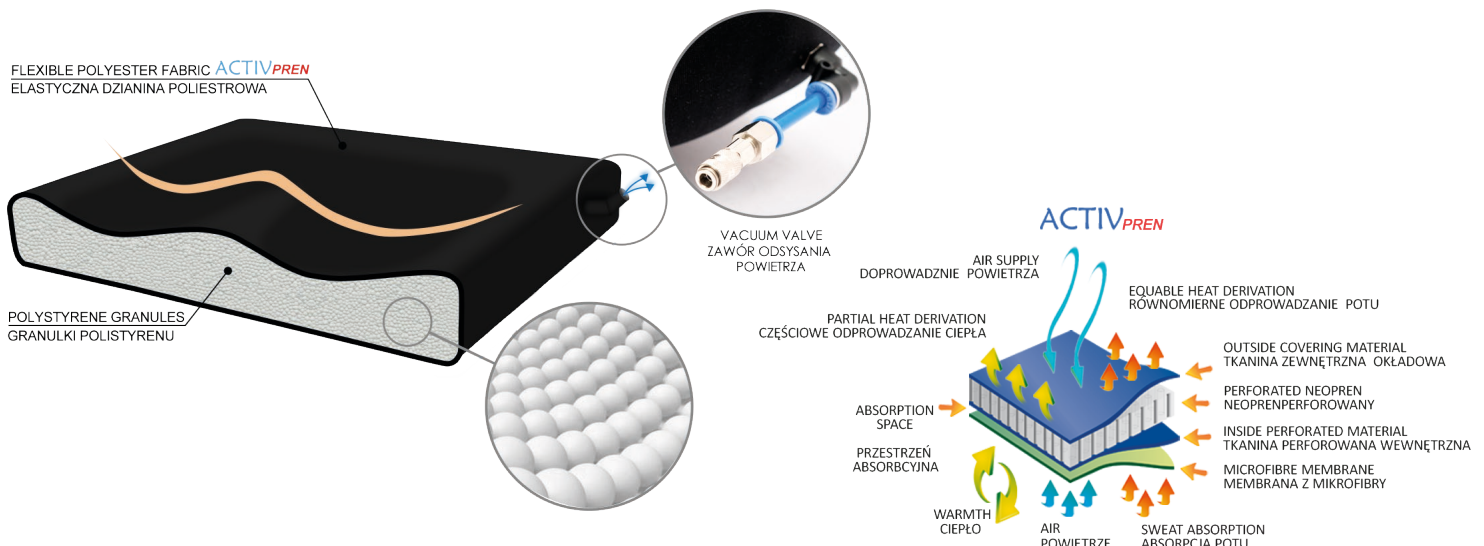
BODYMAP CUSHIONS
PODUSZKI BODYMAP

Seat cushion which make it possible to support hips in wheelchair, positioning chairs or ordinary chair. What is more, there is also possible to model abduction wedge, raise one hip in order to maintain correct position of spine or off-load sensitive body parts. The shape of seat cushion can be easily changed if necessary. One side of BM-A cushions is laminated by elastic polyester and the second one is laminated with self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's thighs. Self-adhesive function makes it easier to assembly the cushion.

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KEY FEATURES

- Shaped form last for a long time
- Easy to individual shape modeling
- Use in wheelchair, positioning chair, chair or in the car
- Optional cover made of thermo active foam

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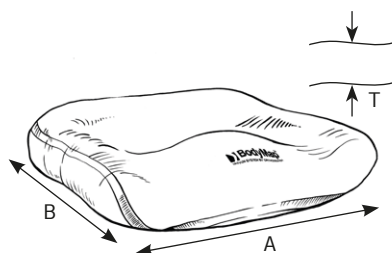
Included / W zestawie

Our pillow keeps good posture of patient while sitting. Special abductor wedge reduces contractures of adductors and maintains the legs in the required position. Anatomic shape and lateral supports of BM A+ pillow prevent against the anterior pelvic tilt what provides comfort for bony prominences such as coccyx and sitting bones. This solution reduces the risk of bedsores. Our BM A+ pillow is the great product for people with neurological conditions and as an orthopaedic prophylaxis after e.g. hip replacement.

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APPLICATION

- Bedsores
- Contracture of the adductor muscles of the hip
- Neurogenic disease with increased muscle tension within the pelvis
- Hip replacement

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
KEY FEATURES

- Shaped form last for a long time
- Easy to individual shape modeling
- Use in wheelchair, positioning chair, chair or in the car
- Optional cover made of thermo active foam
- Self adhesive function
- Waterproof

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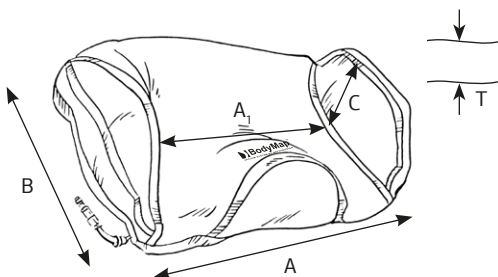Included / w zestawie

The back cushion provides comfort and correction for patients with scoliosis and poor posture. Cushion maintains and controls required posture. In addition to this, patients sit in comfort owing to the fact that backrest enables them to keep correct as well as stable posture regarding patients with hump. The shape of our cushion can be easily changed if necessary. One side of BM-B cushions is laminated by elastic polyester and the second one is laminated with self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's trunk. Self-adhesive function makes it easier to assembly the pillow.

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
KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Use in wheelchair, positioning chair, chair or in the car
- Control of correct posture
- Optional cover made of thermo active foam

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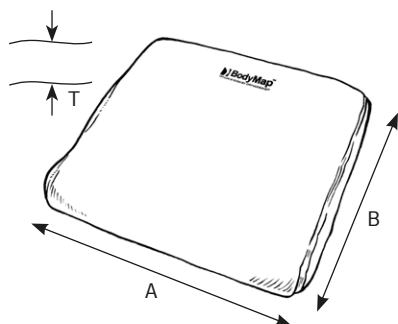B+ pillow is highly recommended for people with back and trunk muscles weakness and for everybody who's in sitting position for a long time, e.g. in the wheelchair. Our positioning cushion reduces excessive stress on skin tissue what limits the risk of bedsores.

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APPLICATION

- Weakness or limpness of paraspinal muscles, the trunk and abdominal muscles
- back and neck bedsores

KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Use in wheelchair, positioning chair, chair or in the car
- Control of correct posture
- Optional cover made of thermo active foam
- Self adhesive function
- Waterproof

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
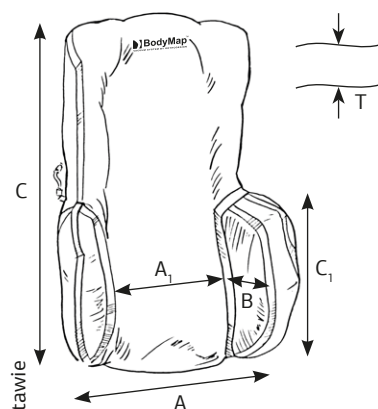
A – Cushion width
A₁ – Cushion width between pelottes
B – Pelottes depth
C – Cushion height
C₁ – Pelottes height
T – Cushion thickness

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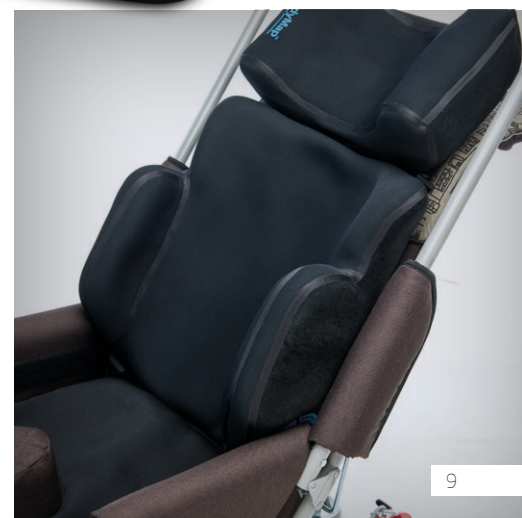Included / w zestawie

Backrest with side supports and headrest provides comfort for patients with scoliosis and poor posture. Moreover, it controls patient's posture. In addition to this, patients sit in comfort owing to the fact that cushions enable them to keep correct as well as stable posture regarding patients with hump. These products are perfect for children with cerebral palsy and patients with muscles weakness or spasticity. Perfect stabilization is possible because of wide side supports and, moreover, headrest is useful while adjusting the correct position of patient's head. The shape of cushion can be easily changed if necessary. One side of BM-C backrest is laminated by elastic polyester and the second one is laminated with the self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's trunk. Self-adhesive function makes it easier to assembly the cushion.

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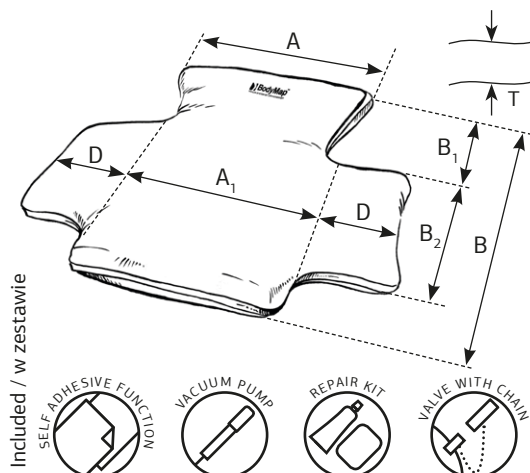
The vacuum chair which function both as backrest and seat. It is useful while transportation of patients in cars and wheelchairs. The cushion makes it possible to support hips and maintain correct posture. In addition to this, it's also helpful for patients with hump because these cushions enable them to have correct as well as stable posture. The shape of cushion can be easily changed if necessary. One side of BM-AB cushions is laminated by elastic polyester and the second one is laminated with self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's trunk and thighs. Self-adhesive function makes it easier to assembly the cushion.

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
KEY FEATURES

- Helps to transport person in vehicles, wheelchairs, etc.
- Easy to individual shape modeling
- Shaped form last for a long time
- Control of correct posture
- Can be used as a positioning chair
- Use in wheelchair, positioning chair, chair or in the car
- Optional cover made of thermo active foam

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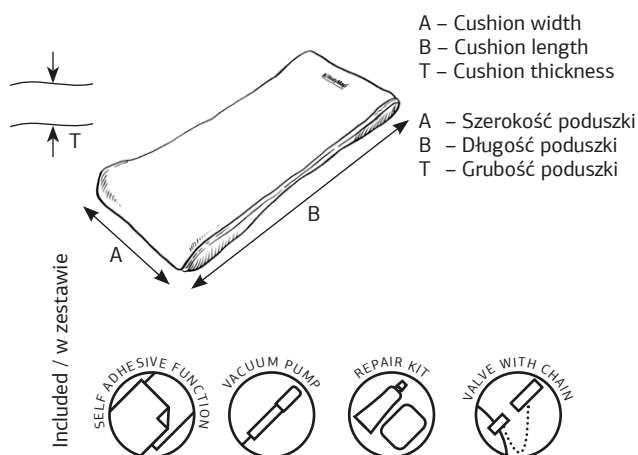BODYMAP AC

POSITIONING CHAIR WITH HEAD-REST AND LATERAL SUPPORT

This is vacuum chair with lateral supports and headrest which function both as backrest and seat. It is useful while transportation of patients in cars and wheelchairs and, more to the point, it helps to control posture. Furthermore, it's also useful for patients with hump because these cushions enable them to have correct as well as stable posture. Perfect stabilization is possible because of wide supports and, moreover, headrest is useful while adjusting the correct position of patient's head. The shape of cushion can be easily changed if necessary. One side of BM-AC cushions is laminated by elastic polyester and the second one is laminated with self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's trunk and thighs. Self-adhesive function makes it easier to assembly the cushion.

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INDICATIONS

- Cerebral palsy
- Scoliosis and back curvature
- Bed sores prophylaxis
- To keep the head in correct position
- Excessive muscles tension (hypersthenia)
- Myotonia

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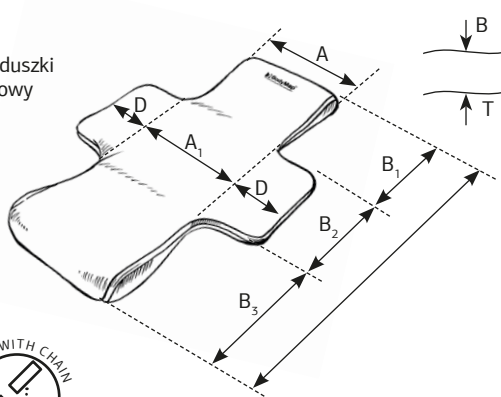
Included / w zestawie

BODYMAP D

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The orthopedic headrest can be easily shaped to patient's head. It can be mounted to most of the wheelchairs, positioning chairs as well as standing frames. The shape of headrest can be easily changed if necessary. One side of BM-D headrests is laminated with elastic polyester and the second one is laminated with self-adhesive fabric allows to attach additional tapes, straps as well as pelottes which stabilize patient's head. Self-adhesive function makes it easier to assembly the headrest. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by REH4MAT. This bed sheet has self-adhesive function.

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INDICATIONS

- Cerebral palsy
- Bed sores prophylaxis
- To keep the head in correct position
- Myotonia

KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Use in almost each wheelchair, positioning chair, rehabilitation stroller
- Optional cover made of thermo active foam

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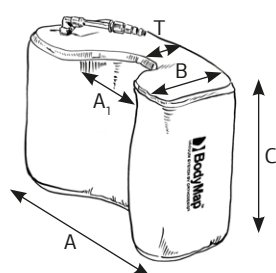
Included / w zestawie

Orthopedic headrest in butterfly shape can be easily shaped to patient's head. It can be mounted to most of the wheelchairs, positioning chairs as well as standing frames. The products have self-adhesive function which makes it possible to attach additional accessories such as head stabilizing belt.

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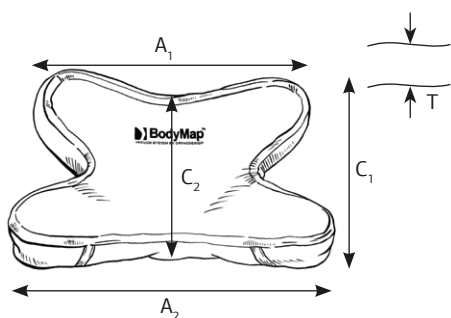
Included / w zestawie

Straight orthopedic headrest which can be easily shaped to patient's head. It can be mounted to most of the wheelchairs, positioning chairs as well as standing frames. The products have self-adhesive function which makes it possible to attach additional accessories such as head stabilizing belt.

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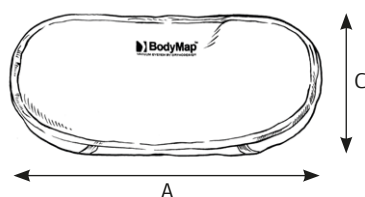
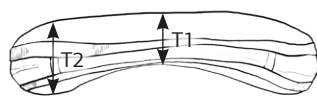
The triangular orthopedic headrest can be easily shaped to patient's head. It can be mounted to most of the wheelchairs, positioning chairs as well as standing frames. The products have self-adhesive function which makes it possible to attach additional accessories such as head stabilizing belt.

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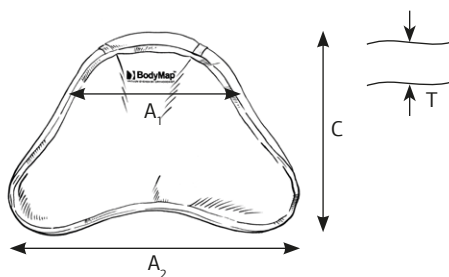
Included / w zestawie

BODYMAP DW

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The neck pillow maintains the head in the proper position while lying. High lateral supports prevent against unrequired head side bends in sleeping. BodyMap DW pillow supports occiput and C7 vertebra, preventing against the bedsores. It's a perfect solution in treatment of torticollis and as a neck stabilizer after neurological or orthopedical surgeries in cervical area.

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APPLICATION

- Bedsores of the neck area, especially at the height of C7
- Neurogenic disease or weakened by increased tension of the neck muscles
- Cervical torticollis
- A stabilization after surgery in the cervical spine

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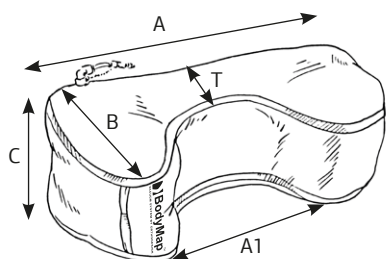
Included / w zestawie

Special upper limb support for wheelchairs, positioning chairs and standing frames. It can be easily modeled to the shape of patient's limbs. It is perfect for most wheelchairs, positioning chairs and standing frames. The shape of elbow support can be easily changed if necessary. One side of BM-E elbow support is laminated by elastic polyester and the second one is laminated with the self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's upper limb. Self-adhesive function makes it easier to assembly elbow support. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by REH4MAT. This bed sheet has self-adhesive function.

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- Cerebral palsy
- Hemiplegia (tetraplegia)
- Bed sores prophylaxis

KEY FEATURES

- Easy to individual shape modeling for better rehabilitation
- Shaped form last for a long time
- Use in almost each wheelchair, positioning chair, rehabilitation stroller, stander
- Optional cover made of thermo active foam

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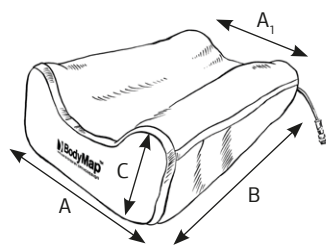
REAR VIEW

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A – External width – rear side
A₁ – External width – front side
B – Total cushion length
C – Height – rear side

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Included / w zestawie

Orthopedic lower limb support which can be easily modeled to the shape of patient's limb. It is perfect for bed sores prevention. What is more, it can be used as post-operative supply where it is necessary to immobilize lower limb. The shape of lower limb support can be easily changed if necessary. One side of BM-F lower limb support is laminated by elastic polyester and the second one is laminated with self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's lower limb. Self-adhesive function makes it easier to assemble lower limb support. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by REH4MAT. This bed sheet has self-adhesive function.

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INDICATIONS

- Cerebral palsy
- Spasticity of the upper limb
- As an immobilization of the lower limb
- Bed sores prophylaxis

KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Optional cover made of thermo active foam

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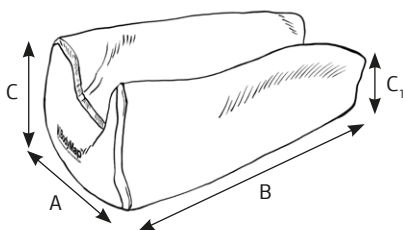
REAR VIEW

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A – External width – rear side
B – Total cushion length
C – Height – rear side
C₁ – Height – front side

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Included / w zestawie

Derotation splint which can be easily modeled to the shape of patient's limb. This is perfect for bed sores prevention. What is more, it can be used as post-operative supply where it is necessary to immobilize lower limb as well as foot. The shape of splint can be easily changed if necessary. One side of BM-G derotation splint is laminated by elastic polyester and the second one is laminated with the self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's lower limb. Self-adhesive function makes it easier to assemble the derotation splint. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by REH4MAT. This bed sheet has self-adhesive function.

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INDICATIONS

- Cerebral palsy
- Palsy and paresis of lower limbs
- Bed sores prophylaxis
- Immobilization of the lower limb
- Replacement of hip joint

KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Optional cover made of thermo active foam

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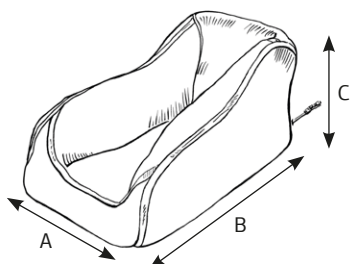
Included / w zestawie

One-sided trunk stabilizer which can be easily shaped to patient's body. They are perfect for bed sores prevention. What is more, they can be used as post-operative supply where it is necessary to immobilize the patient in side-lying position. The shape of trunk support can be easily changed if necessary. One side of BM-H trunk support is laminated by elastic polyester and the second one is laminated with the self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's trunk. Self-adhesive function makes it easier to assembly trunk support. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by REH4MAT. This bed sheet has self-adhesive function.

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INDICATIONS

- Cerebral palsy
- Bed sores prophylaxis
- As an immobilization on side position

KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Optional cover made of thermo active foam

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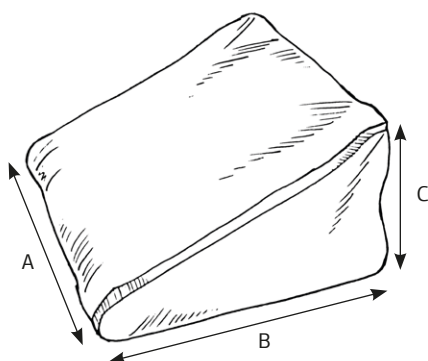
Included / w zestawie

The vacuum head support which can be easily shaped to patient's head. They stabilize head in lying position. Head support provides width adjustment as well as self-adhesive function thanks to which any kind of additional accessories can be attached e.g. head stabilizing belt. The shape of head support can be easily changed if necessary. One side of BM-I head support is laminated by elastic polyester and the second one is laminated with the self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's head. Self-adhesive function makes it easier to assemble head support. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by REH4MAT. This bed sheet has self-adhesive function.

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INDICATIONS

- Cerebral palsy
- Bed sores prophylaxis
- As an head immobilization
- Neck spine discopathy

KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Optional cover made of thermo active foam

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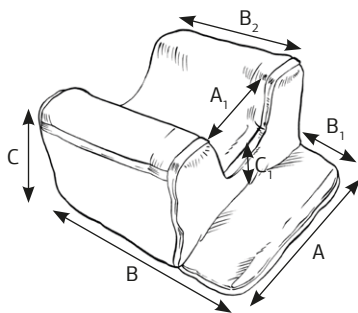
A – Total head support width
A₁ – Inner head support width
B – Total depth
B₁ – Depth
B₂ – Depth
C – Total height
C₁ – Height

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Included / w zestawie

Upper limb splint can be easily modeled to the shape of patient's limb. It's perfect for bed sores prevention. What is more, it can be used as post-operative support where it is necessary to immobilize limb. The shape of upper limb splint can be easily changed if necessary. One side of BM-J upper limb splint is laminated by elastic polyester and the second one is laminated with the self-adhesive fabric that allows to attach additional tapes, straps as well as pelottes which stabilize patient's lower limb. Self-adhesive function makes it easier to assembly our upper limb splint. If the product is placed in hospital bed, it is necessary to use special bed sheet produced by REH4MAT. This bed sheet has self-adhesive function.

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INDICATIONS

- Cerebral palsy
- Spasticity of the upper limb
- As an immobilizer of upper limb
- Bed sores prophylaxis

KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Self adhesive function
- Optional cover made of thermo active foam
- Waterproof

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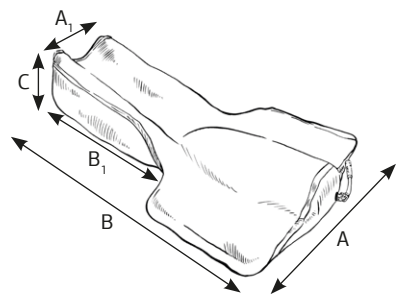
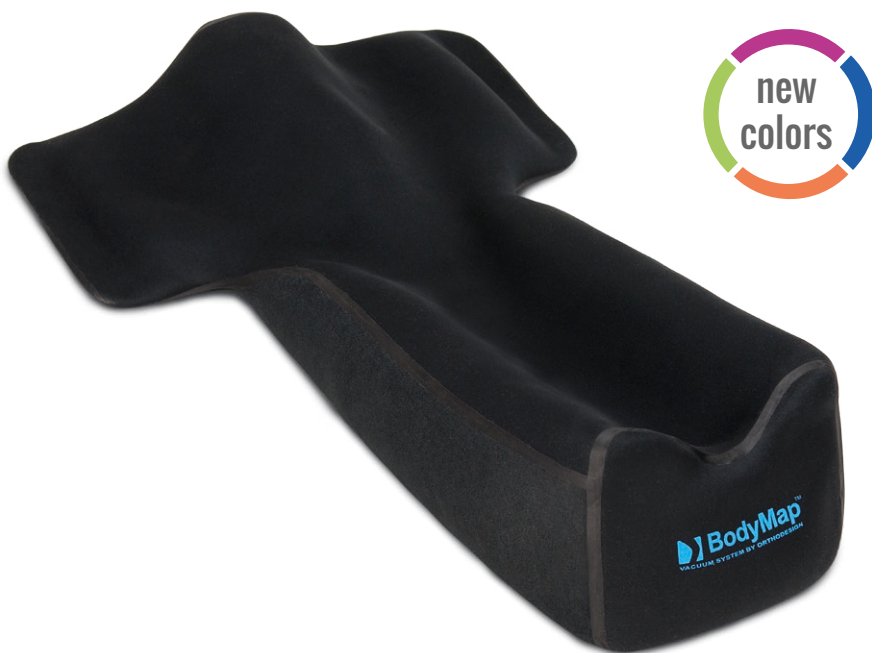
Included / w zestawie

Vacuum stabilizing mattresses which can be easily shaped to the patient's body. They are perfect for bed sores prevention due to the fact that they support pressure points. They enable patient to keep correct position for trunk, hips, head, upper as well as lower limbs. They are used so as to correct postural defects.

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KEY FEATURES

- Easy to individual shape modeling
- Shaped form last for a long time
- Optional cover made of thermo active foam

INDICATIONS

- Cerebral palsy
- Bed sores prophylaxis
- Poor position of trunk, pelvis, upper and lower limbs as well as a head
- Poor posture
- Excessive muscles tension (hypersthenia)

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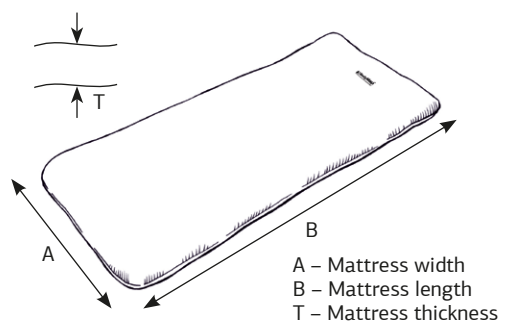
Included / w zestawie

A car seat cushion BM-L that provides patients with a comfortable base during a car trip. It gives the opportunity to correct scoliosis and poor postures, improving postural control. It also enables to obtain a correct and stable posture of patients with hump. The backrest pillow significantly increases comfort. With additional accessories, such as stabilizing belt, our cushion can be easily attached to a car seat.

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KEY FEATURES

- Shaped form last for a long time
- Easy to individual shape modeling
- Easy to install in the car
- Optional cover made of thermo active foam

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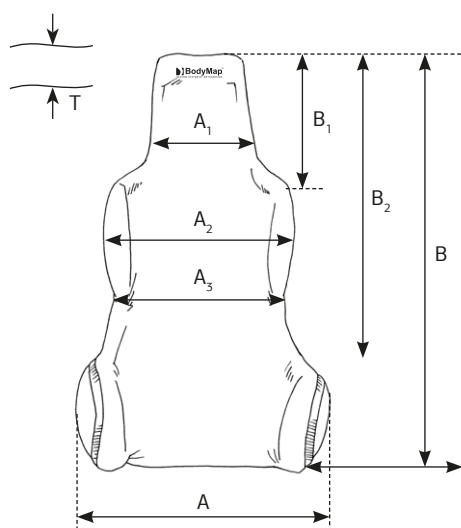
Included / w zestawie

Pillow M is the excellent product in addition to daily rehabilitation. If your patient uses our standing frame, wheelchair or positioning chair with table, after play or meal, you can mount our pillow on it and continue the therapy! You can achieve any shape of its surface, so you can place and maintain hands and forearms in the required position. Our table pillow corrects upper limb deformations precisely.

The dimensions of the cushion and the price calculated when ordering.

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APPLICATION

- Deformities of the upper limbs within the hands and forearms (neurogenic, rheumatoid, muscle and joint contractures)
- Bedsores of the forearm, wrist and hand
- Paresthesia resulting from intra-articular ulnar nerve compression
- Painful rheumatoid nodules

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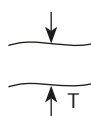
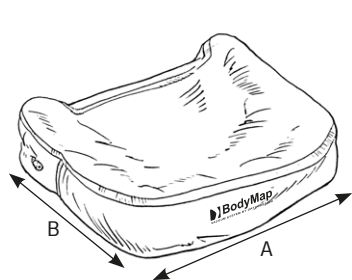
Feet correction pillow keeps deformed feet in the required therapeutic position. It is a great product for passive rehabilitation of club feet and other deformities in neurological or rheumatic conditions. The pillow can be used in sitting and standing position, e.g. during doing the homework in our standing frame.

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APPLICATION

- Bed sores in the feet and ankle joint (mainly calcaneus)
- Ankle joint deformations (feet equine varus and clubfoot deformity)
- Neurogenic diseases with enhanced or impaired muscle tension
- Hallux valgus
- Rheumatoid foot

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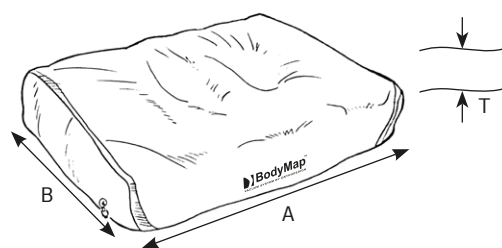
The roller keeps patient in the required therapeutic position and corrects deformities in thy lying position. It can be used also to off-loading the particular body parts. Our roller is the excellent support in positioning therapy of lying patients with pulmonological and trophic skin disorders.

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APPLICATION

- As a positional therapy of bedridden patients
- bed sores throughout the body
- degenerative and muscle contractures deformities

KEY FEATURES

- Shaped form last for a long time
- Easy to individual shape modeling
- Optional cover made of thermo active foam

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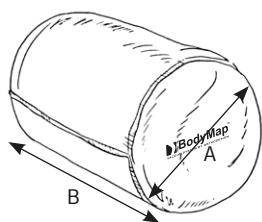
A – The diameter of the cushion
B – Cushion length

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Included / w zestawie

BodyMap P immobilizer stabilizes limb's fractures and dislocations without circumferential pressure. It's secured on the limb with self-gripping fastenings with buckles, added together with splint. You can attach them in every required place.

When the air is pumped out, the granulate beads form a solid mass, making the immobilizer rigid with patient's anatomic shape. It prevents against the fracture instability. When the circumferential of the limb is changed, because of smaller swelling, you can pump extract the air again to make the perfect anatomic fitting.

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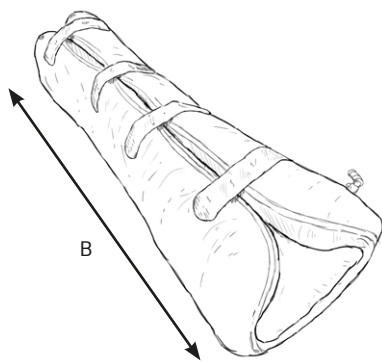
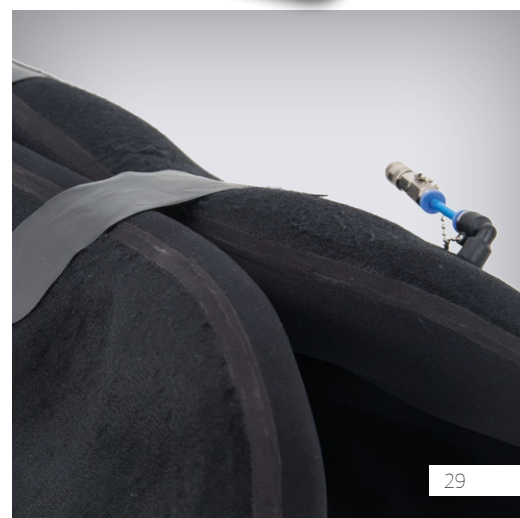
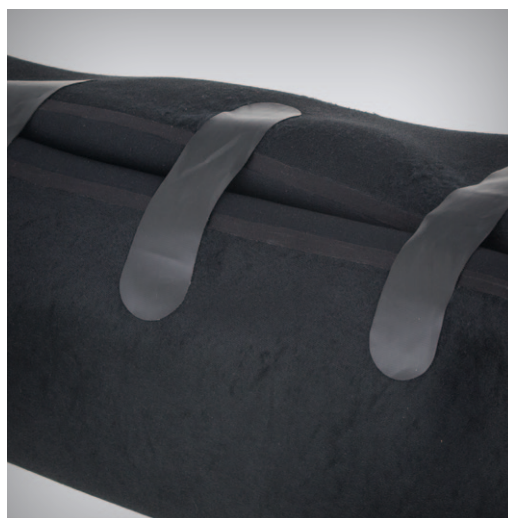
Included / w zestawie

BEAN BAG CHAIR chair is a sensory floor cushion created for disabled children. It's filled with micro beads that allows to maintain the body shape. Moreover, micro beads provide high proprioceptive feeling and improve neuromotor functions. The cushion may have a different shape each time, so it's excellent support to play, learning or relax.

In standard, the bean bag is made of elastic, skin-friendly neoprene, and it's covered by damage-resistant anti-migrated fabric from the bottom. For customer request, the sides of our floor cushion can be made of self-gripping material, what allows to attach additional straps and belts for improving the child stabilization.

Our BEAN BAG CHAIR provides your child safety and comfort every single day!

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
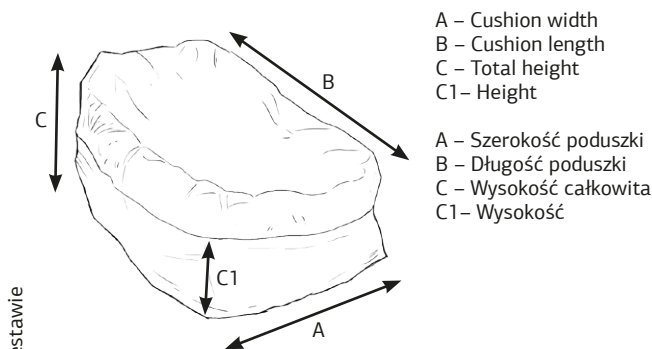
Included / w zestawie

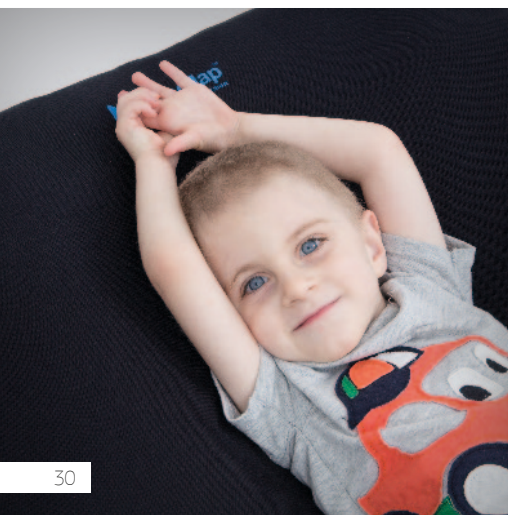STANDARD PUMPS / POMPKI STANDARDOWE

Each product is equipped with vacuum pump, depending on product size.

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BM_001 MANUAL VACUUM PUMP – SMALL

Handy, small vacuum pump. Perfect for small and medium cushions. Each cushion includes this vacuum pump free of charge.

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BM_002 MANUAL VACUUM PUMP – BIG

Very efficient manual vacuum pump. Perfect for mattresses and big cushions. Each mattress includes this vacuum pump free of charge.

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ADDITIONAL PUMPS / POMPKI DODATKOWE

BM_003 ELECTRIC VACUUM PUMP

Handy and very efficient electric vacuum pump. Equipped with electric charger. Can work with AA battery also (not included).

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BM_004 MANUAL DUAL-FUNCTION PUMP

Efficient, handy and made of high quality components. Strongly recommended for small cushions.

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BM_005 ELECTRIC DUAL-FUNCTION PUMP

With electric dual-function pump it is possible to adjust the hardness of the cushion to the individual needs of the patient. This pump allows precise pumping out or in the air in order to achieve the desired hardness. Noiseless, efficient and made of durable aluminum. Strongly recommended for large cushions and mattresses.

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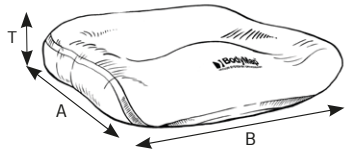
CUSTOM MADE PRODUCTS

Custom made products available. We can make size according your needs or we can fit cushion on any wheelchair. Pushchair, positioning chair, etc. Please note that all Body Map cushions are elastic, easy to mould and shape. We suggest to have margin and make bigger cushion which will always fit perfectly to the user's body.


BODYMAP A* SEAT CUSHION

A - The maximum depth of the seat
B - The maximum width of the seat
T - Cushion thickness


BODYMAP A+ SITTING POSITIONING CUSHION WITH THIGH ABDUCTOR WEDGE AND LATERAL SUPPORTS

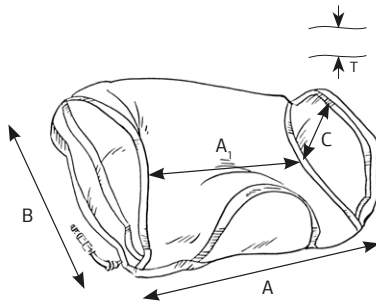A - Cushion width
A₁ - Seat width between pelottes
B - Cushion depth
C - Height of pelottes
T - Cushion thickness


BODYMAP B* BACK CUSHION

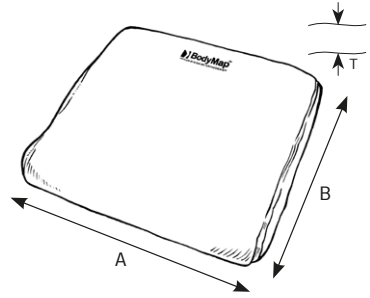A - The maximum width of the seat
B - Height from seat up to patient's shoulders
T - Cushion thickness


BODYMAP B+ BACKREST CUSHION WITH LATERAL SUPPORTS

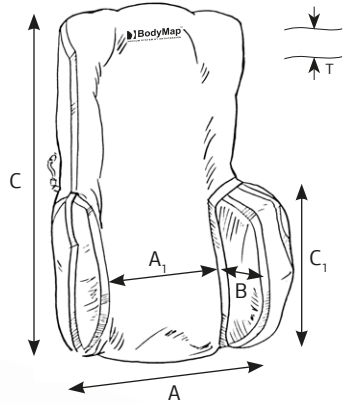A - Cushion width
A₁ - Cushion width between pelottes
B - Pelottes depth
C - Cushion height
C₁ - Pelottes height
T - Cushion thickness


BODYMAP C* BACK CUSHION WITH HEADREST AND LATERAL SUPPORT

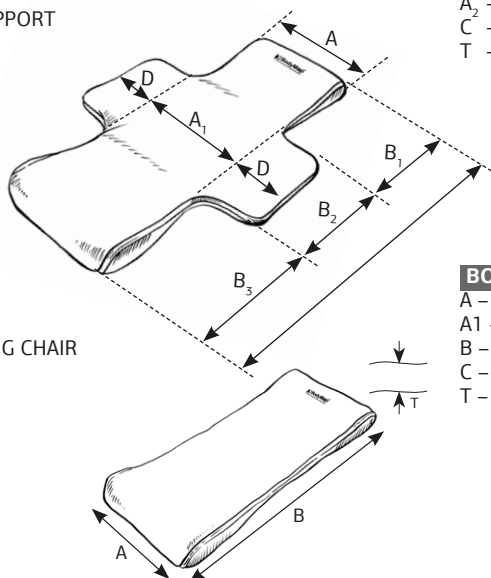A - Width
A₁ - Width
B - Total cushion length
B₁ - Headrest length
B₂ - Pelottes length
D - Pelottes width
T - Cushion thickness


BODYMAP AB* POSITIONING CHAIR

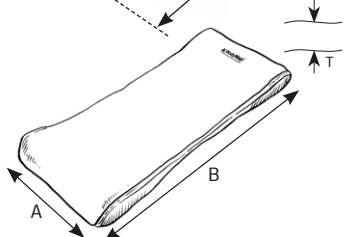A - Cushion width
B - Cushion length
T - Cushion thickness


BODYMAP AC* POSITIONING CHAIR WITH HEADREST AND LATERAL SUPPORT

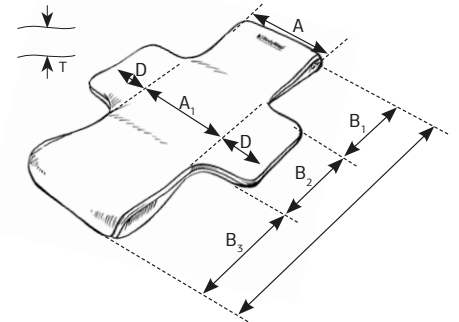A - Width
A₁ - Width
B - Total cushion length
B₁ - Headrest length
B₂ - Pelottes length
B₃ - Seat length
D - Pelottes width
T - Cushion thickness


BODYMAP D HEADREST

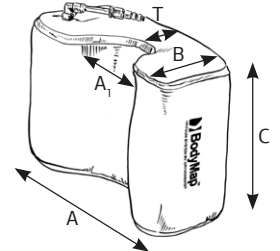A - Total headrest width
A₁ - Inner width of the headrest
B - Headrest depth
C - Total headrest height
T - Headrest thickness


BODYMAP DX BUTTERFLY HEADREST

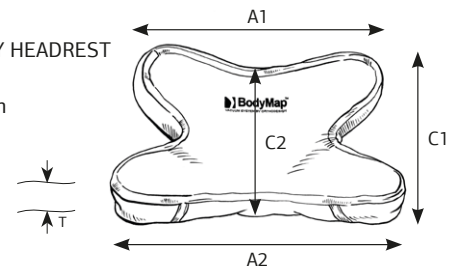A₁ - Headrest width - top
A₂ - Headrest width - bottom
C₁ - Headrest height
C₂ - Headrest height
T - Headrest thickness


BODYMAP DY STRAIGHT HEADREST

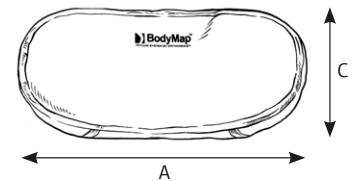A - Headrest width
C - Headrest height
T₁ - Headrest thickness
T₂ - Headrest thickness


TOP VIEW

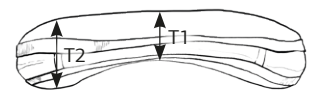
BODYMAP DZ TRIANGULAR HEADREST

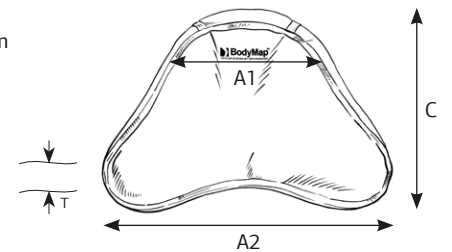A₁ - Headrest width - top
A₂ - Headrest width - bottom
C - Headrest height
T - Headrest thickness

BODYMAP DW NECK CUSHION

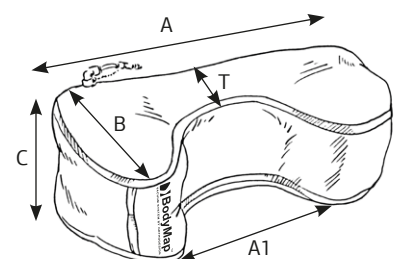
A - Total headrest width
A₁ - Inner width of the headrest
B - Headrest depth
C - Total headrest height
T - Headrest thickness

* Cushions that can be fitted on wheelchairs, pushchairs, etc.


BODYMAP E UPPER LIMB SUPPORT FOR WHEELCHAIRS, STROLLERS AND STANDING FRAMES

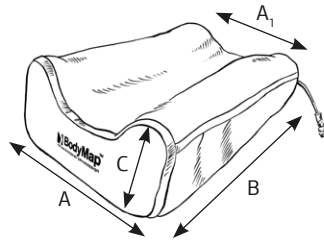- A – External width – rear side
- A₁ – External width – front side
- B – Total cushion length
- C – Height – rear side


BODYMAP L CAR SEAT CUSHION

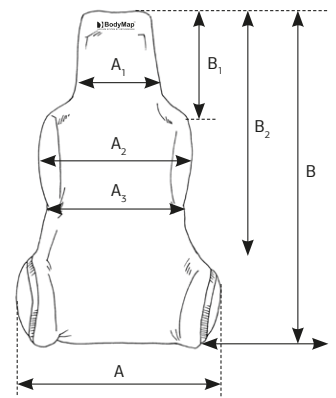- A – Total cushion width
- A₁ – Headrest width
- A₂ – Width
- A₃ – Width
- B – Total cushion length
- B₁ – Headrest length
- B₂ – Length
- T – Cushion thickness


BODYMAP F LOWER LIMB SUPPORT

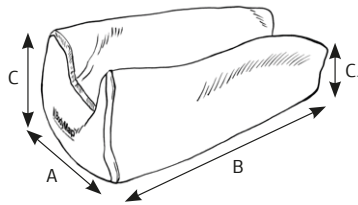- A – External width – rear side
- B – Total cushion length
- C – Height – rear side
- C₁ – Height – front side


BODYMAP M CUSHION STABILIZING THE UPPER LIMBS

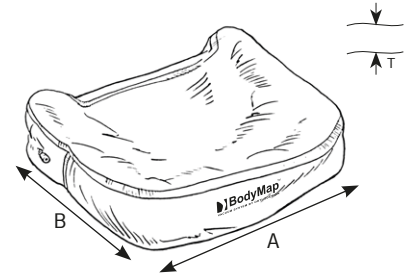- A – Cushion width
- B – Cushion length
- T – Cushion thickness

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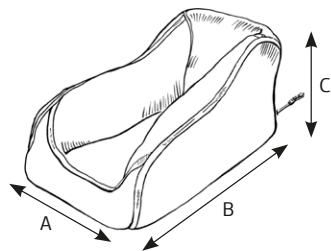
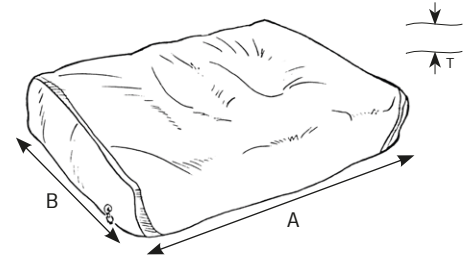
- A – Cushion width
- B – Cushion length
- T – Cushion thickness


BODYMAP H UNILATERAL TRUNK SUPPORT

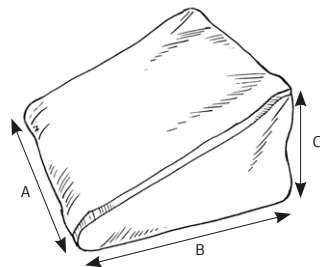- A – Cushion width
- B – Cushion length
- C – Cushion height


BODYMAP I HEAD SUPPORT

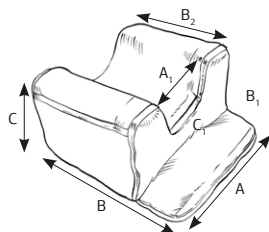- A – Total head support width
- A₁ – Inner head support width
- B – Total depth
- B₁ – Depth
- B₂ – Depth
- C – Total height
- C₁ – Height


BODYMAP O STABILIZING ROLLER

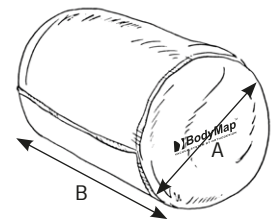- A – The diameter of the cushion
- B – Cushion length


BODYMAP J UPPER LIMB SUPPORT

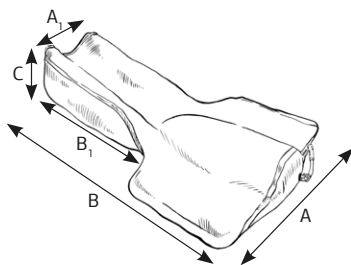- A – Width - front side
- A₁ – Width - rear side
- B – Total cushion length
- B₁ – Cushion length
- C – Height – rear side


BODYMAP P VACUUM LIMB IMMOBILIZER

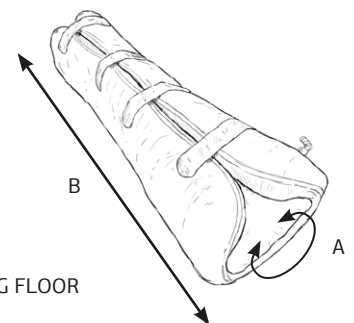- A – Thigh circumference
- B – Leg length


BODYMAP K STABILIZING MATTRESS

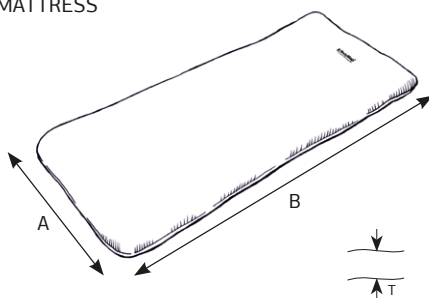- A – Mattress width
- B – Mattress length
- T – Mattress thickness

BODYMAP R SENSORY BEAN BAG FLOOR CUSHION

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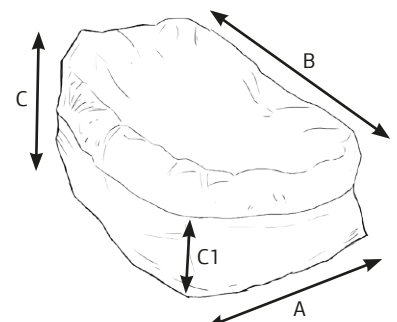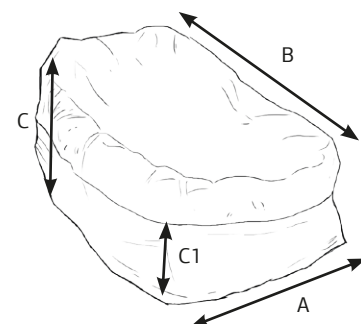
BM products can be used by people with cervical disc disorders. In the lying position, you can use the vacuum head support BM I to offload and stabilize the patient's head, cervical spine and neck. Apart of achieving the anatomic shape, the BM devices can be shaped according to physician or therapist recommendations. Because of innovative construction, in our BM product you can place the head in the required therapeutic position. Also, you can relieve the pressure points (Trigger Points).

BodyMap DW is the second product that is used to stabilize the cervical spine in the lying position. Using our product, you can correct a lot of deformations in the neck area. If you need correction in the sitting position, you can use a various types of our vacuum headrests and neck cushions such as: BodyMap D, BodyMap DX, BodyMap DY, BodyMap DZ.

In car, you should apply comprehensive and practical car seat cushion BodyMap L with a wide range of our headrests and neck cushions.

In the office, we highly recommend our special seat and back cushions with or without pelvic lateral support.

While sleeping, you should use our head, neck, trunk and limbs supports. If it is necessary, you can also use the stabilizing mattresses. They spread the forces over the surface and reduce the pressure in the specific areas. This solution allows to offload the body and prevents against the bedsores.

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CLEANING, MAINTENANCE AND REPAIR

Pillows should be regularly cleaned with the aid of dry brush or vacuum cleaner equipped with attachment made of soft bristles. If stains are big, it is necessary to clean them with running water in a temperature up to 40 °C. It is also possible to use detergent such as washing-up liquid or soap solution. Remember to rinse cleaned place off with running water and dry it with soft towel. It is forbidden to use solvents: toluene or acetone. It is forbidden to use chlorine or its derivatives. Dry far from direct heat. Remember to be careful when you clean seals which protect places that connect pillow's elements. It is necessary to be cautious not to cut or pierce pillow; do not use any sharp tools for cleaning. You can repair damage cushion using "repair kit" which is included free of charge to each product. Kit consist of special neoprene super glue and few patches which you can use to seal the damage surface of the cushion. Pillow should be sent to manufacturer in case of mechanical damages. In order to minimize the risk of bedsores, we recommend using a cotton cover along with each BodyMap cushion if the permanentduration of use is longer than 2 hours. For people using BodyMap cushions permanently for more than 8 hours it is recommended to use it along with thermo elastic VISEMEMO cover. Caution should be taken in subjects to people with known bedsores-prone. In such a case it is strictly recommended to you use BodyMap cushion with aeration-massaging pad for people who use BodyMap cushions non-stop longer than 8 hours!

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Under any circumstances bodymap product cannot be fold, when the air is pump out and the vacuum valve is assambled.

It is forbidden to press or seat on the bodymap cushions when the air is not pumped out (when the air is inside the cushion) and the valve is assambled at the same time. Before shaping spread the polystyrene granules evenly and prevacuum cushion so it still be soft and ready for shaping.

Self adhesive surface is used to fix the cushion on stroller, chair, special shells, etc or to mount the portable elements like optional straps and other accessories.

Please fix the BodyMap cushion on chair, stroller, etc. at first.

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REPAIR SYSTEM / SYSTEM REPERACYJNY

For any ordered set of cushions repairing system is added.
The set includes:

- Tube of neoprene super glue
- A set of patches
- Instruction manual

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DESIGNATION OF PRODUCTS / OZNACZENIE PODUSZEK BODYMAP

QR code - user scans code and then website for BodyMap cushions will be displayed on the screen

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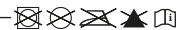
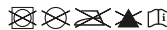
CE hallmark - manufacturer declares that product fulfils
the terms that are included in directives

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ENG
BodyMap System refers to first class medical products which are designed for adults as well as children with cerebral palsy. All these products provide perfect stabilization during seating or lying position. What is more, it is a perfect choice for the disabled due to the fact that it prevents bed sores.
BodyMap System uses vacuum method. This method refers to products which are filled with polystyrene granules thanks to which products can model according to patient's body shape and can create so-called "3D BodyMap". Polystyrene granules move freely inside the device and, more to the point, it is easy to move them in order to shape cushion at your discretion. Between granules the force of friction increases as an result of deflating the air from the device and it keeps required shape of the cushion. To each cushion there is a possibility to use thermosensitive cover with anti bedsores properties.

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
European Article Number (EAN) -
unique code that is given by
manufacturer

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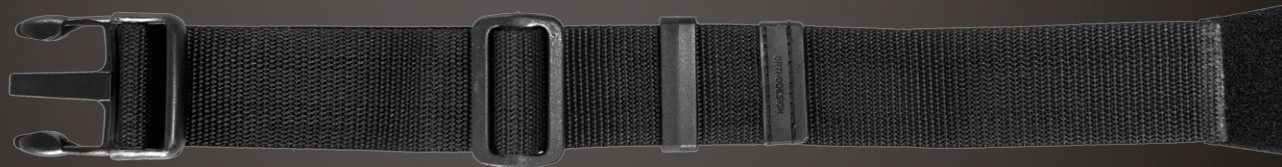NON-ELASTIC STABILIZING BELT – it is non-elastic belt with Velcro tapes and grommet at the end. It makes it possible for a patient to have stabilization and to fasten cushion on chair, rehabilitation stroller etc.

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ELASTIC STABILIZING BELT – it is elastic belt with Velcro tapes and grommet at the end. It makes it possible for a patient to have stabilization and to fasten cushion on chair, rehabilitation stroller etc.

It is possible to use parts listed below together with this product:
Grommets P/4 and P/5 → p. 41
Fastener with rectangular grommet SP/4 and SP/5 → p. 45
Fastener with half-round grommet SO/4 and SO/5 → p. 46

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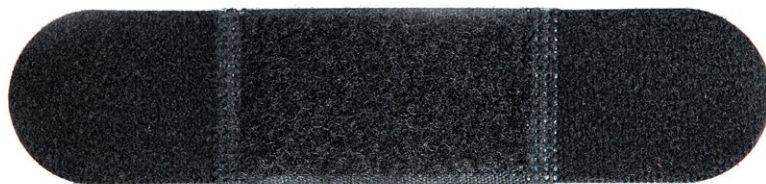
GROMMETS FOR FASTENING THE TAPES – these are grommets with Velcro tapes. They can be mounted on self-adhesive surface of cushion. Grommets are used so as to adjust tapes on cushions. They have 4 different sizes and, moreover, they can be used with various types of tapes. One set is equipped with 4 grommets.

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NON-ELASTIC ONE-SIDED TAPE – it is non-elastic tape that ends with Velcro tapes. The use of Velcro tapes is advantageous because it is easy to fasten non-elastic one-sided tape on self-adhesive surface of cushion single-handedly or with the aid of fasteners. Tape can be used together with front clasp thanks to which pelvic belt is created and stabilization of the patient is possible.

It is possible to use parts listed below together with this product:
Grommets P/4 and P/5 → p. 41
Front clasp ZT/4 and ZT/5 → p. 42
Fastener with rectangular grommet SP/4 and SP/5 → p. 45
Fastened with round grommet SO/4 and SO/5 → p. 46

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ELASTIC ONE-SIDED TAPE – it is elastic tape that ends with Velcro tapes. The use of Velcro tapes is advantageous because it is easy to fasten elastic one-sided tape on self-adhesive surface of cushion single-handedly or with the aid of fasteners. Tape can be used together with front clasp thanks to which pelvic belt is created and stabilization of the patient is possible.

It is possible to use parts listed below together with this product:

Grommets P/4 and P/5 → p. 41

Front clasp ZT/4 and ZT/5 → p. 42

Fastener with rectangular grommet SP/4 and SP/5 → p. 45

Fastener with round grommet SO/4 and SO/5 → p. 46

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FRONT CLASP FOR ONE-SIDED TAPES – this part is equipped with grommets that make it possible to fasten either elastic or non-elastic one-sided tapes on front clasp. Pelvic belt that stabilizes the patient can be fastened to BodyMap cushion with the aid of Velcro tapes and self-adhesive surface of BodyMap cushion.

It is possible to use parts listed below together with this product:

Non-elastic one-sided tape TJN/58/4 and TJN58/5 → p. 41

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STABILIZING BELT WITH CLAMPS – it is adjustable as well as non-elastic tape that is fastened with the aid of clamp. It makes it possible for a patient to have stabilization and to fasten cushion on chair, rehabilitation stroller etc. It is possible to use parts listed below together with this product:

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ONE-SIDED STABILIZING TAPE WITH CLAMP / TAŚMA JEDNOSTRONNA Z KLAMRĄ

STABILIZING TAPE WITH CLAMP – it is non-elastic as well as adjustable belt that is equipped with clamp and Velcro tape thanks to which tape can be fastened to self-adhesive surface of cushion. Tape can be used so as to fasten vest on pillow. Furthermore, tape together with front clasp with clamps can create pelvic belt that stabilizes the patient.

It is possible to use parts listed below together with this product:

Grommets P/2,5, P/3, P/4 and P/5 → p. 41

Front clasp with clamps ZPK/2,5, ZPK/3, ZPK/4, ZPK/5 → p. 44

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NON-ELASTIC STABILIZING TAPE WITH CLAMP – it is non-elastic as well as adjustable belt with clasps at the ends. Tape can be used together with front clasp with clasps thanks to which pelvic belt is created and stabilization of the patient is possible. Furthermore, it is possible to fasten cushion on chair, rehabilitation stroller etc. Tape can be also used so as to fasten vests on cushion.

It is possible to use parts listed below together with this product:
Grommets P/2,5, P/3, P/4 and P/5 → p. 41
Front clasp with clasps ZPK/2,5, ZPK/3, ZPK/4, ZPK/5 → p. 44
4-point vest K4/1, K4/2, K4/3 → p. 48
6-point vest K4/1, K4/2, K4/3 → p. 48
H harness KH-1, KH-2, KH-3 → p. 49

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FRONT CLASP WITH CLAMPS – this part is equipped with clasps thanks to which it is possible to use it together with one-sided tape with clamp. Pelvic belt that stabilizes the patient can be fastened to BodyMap cushion with the aid of Velcro tapes and self-adhesive surface of BodyMap cushion.

It is possible to use parts listed below together with this product:
One-sided tape with clamp TJK/63/2,5, TJK/63/3, TJK/63/4, TJK/63/5 → p. 43

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NON-ELASTIC CHEST BELT – it is universal and adjustable belt that is used so as to stabilize the patient. Moreover, it makes it possible to fasten BodyMap cushion on chair, rehabilitation stroller etc. Belt is equipped with fastener made of Velcro tapes thanks to which it can be adjusted at any length. Additionally, belt is secured by means of tape with clamp.

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FASTENERS WITH RECTANGULAR GROMMET – this part is equipped with Velcro tapes as well as rectangular grommet. It can be fastened to any surface on Body Map cushion that is self-adhesive and, moreover, it can be used as bearing for some tapes. The use of fastener is beneficial because it makes it easier to fasten tapes on cushions and it prevents self-adhesive surface from damages that can appear when user fastens and unfastens Velcro tapes frequently.

It is possible to use parts listed below together with this product:

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EN FASTENERS WITH HALF-ROUND GROMMET – this part is equipped with Velcro tapes as well as half-round grommet. It can be fastened to any surface on Body Map cushion that is self-adhesive and, moreover, it can be used as bearing for some tapes. The use of fastener is beneficial because it makes it easier to fasten tapes on cushions and it prevents self-adhesive surface from damages that can appear when user fastens and unfastens Velcro tapes frequently.

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FASTENING TAPE FOR HEADREST – it is non-elastic tape that can be fastened to headrest with the aid of endings with Velcro tapes. Tape is used so as to stabilize patient's head in BodyMap headrest.

It is possible to use parts listed below together with this product:
Fastener with rectangular grommet SP/2,5 and SP/5 → p. 45

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FASTENING BELT FOR HEADREST – it is non-elastic belt that is used so as to fasten BodyMap headrest on stroller's frame, chairs etc. BodyMap headrest has self-adhesive surface thanks to which belt with Velcro tapes can be fastened to cushion and, moreover, headrest can be mounted to frame at any height.

It is possible to use parts listed below together with this product:
Grommets P/2,5 → p. 41

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PELVIC-FEMORAL BELT / PAS BIODROWO-UDOWY

PELVIC-FEMORAL BELT – non-elastic as well as adjustable belt that is equipped with fastening system that perfectly stabilizes patient's hip. It can be used so as to stabilize the patient and to fasten BodyMap cushion on stroller.

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4-POINT VEST / KAMIZELKA 4-PUNKTOWA


It is possible to use parts listed below together with this product:

One-sided tape with clamp TJK/63/2,5, TJK/63/3 → p. 43
 Non-elastic stabilizing belt with clamps TONK/174/2,5
 and TONK/174/3 → p. 44

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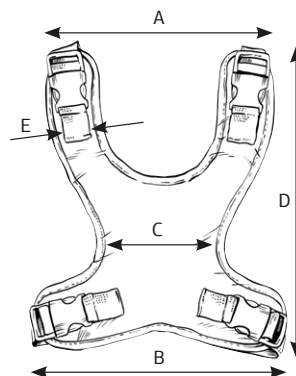6-POINT VEST / KAMIZELKA 6-PUNKTOWA


It is possible to use parts listed below together with this product:

One-sided tape with clamp TJK/63/2,5, TJK/63/3 → p. 43
 Non-elastic stabilizing belt with clamps TONK/174/2,5 and TONK/174/3 → p. 44

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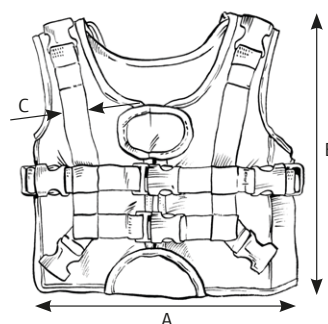H HARNESS / KAMIZELKA H

It is possible to use parts listed below together with this product:

One-sided tape with clamp TJK/63/2,5, TJK/63/3 → p. 43

Non-elastic stabilizing belt with clamp TONK/174/2,5 and TONK/174/3 → p. 44

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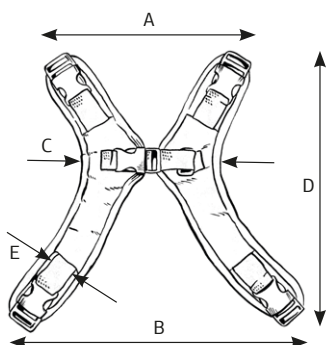
BACKREST FOR ADULTS / OPARCIE DL A DOROSŁYCH

FRAME FOR BODYMAP SYSTEM / STELAŻ DO SYSTEMU BODYMAP

Backrest for adults - it is light as well as handy frame that is created for stabilization. It is used together with BodyMap system. Backrest is perfect for daily activities especially it is recommended during feeding the patients who are bedridden. The product is equipped with Velcro tapes thanks to which it is easy to fasten any BodyMap cushion on it. It is highly beneficial to fasten BodyMap cushion because it gives support and stabilization for trunk, hips, head or limbs (it depends which kind of cushion is chosen). Furthermore, it is also possible to use support as a single product, without any cushions.

As far as its advantages are concerned, it is solid and, moreover, it has a wide range of adjustment for tilt angle and the height of backrest. Caregiver can easily adjust frame to patient so as to make him/her feel comfortable. Upholstery is made of washable material. Moreover, the product is equipped with wide trunk belt that not only stabilizes the patient but also secures him/her in the frame.

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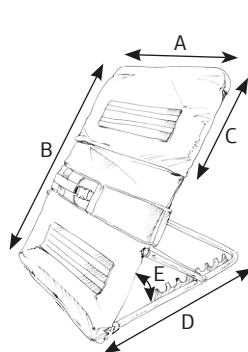
FRAME FOR BODYMAP SYSTEM / STELAŻ DO SYSTEMU BODYMAP

It is universal and durable support with wide range of adjustments. Designed for stabilizing the upper limbs with BodyMap System. Perfect for hemiplegia and hemiparesis to support and relieve upper limbs with paresis or palsy. Thanks to universal grip the support can be mounted on wheelchairs, special needs strollers, etc. This support is designed for use with BodyMap E cushion. We can fix the cushion on the support using velcro tapes available in our offer also (page 35-54 of BodyMap catalogue).

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There is a possibility to order **VisMemo cover** which is equipped with thermoactive foam. This foam works according to two steps:

- 1. First step** – it adjusts to pressure which is put on patient's body. It is first, introductory level of adjustment. It is also first and last level for normal foam.
- 2. Second step** – it is not immediate because it is spread over time. Moreover, cover softens due to patient's body heat. This effect is not the same within whole surface when body has contact with device. For this reason, foam softens in the hottest places, these "red points", where body pressure on device is still too firm. This reaction between cover and thermo-elastic foam makes it possible to distribute pressure points.

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
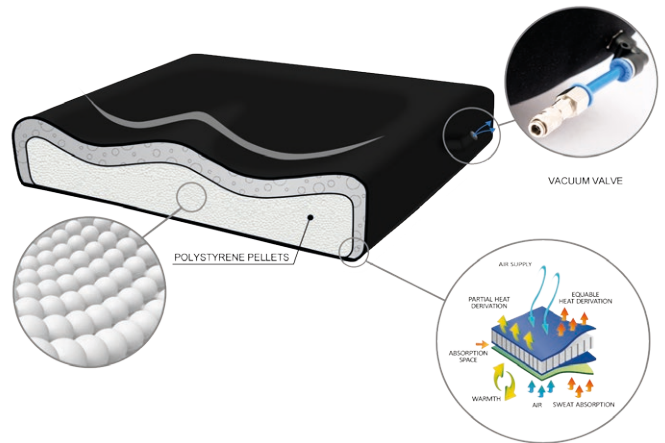
COTTON COVER / POKROWIEC BAWĘLNIANY

BodyMap cover is made of soft cotton in 5 modern colors. The fabric is neutral for skin, air-permeable and absorbs the sweat. Also, the cover prevents against the bedsores and skin lesions. The cover is fitted and it's easy to put it on and off the BodyMap cushion. It may be washed with similar colors in washing-machines in temperature 30°C.

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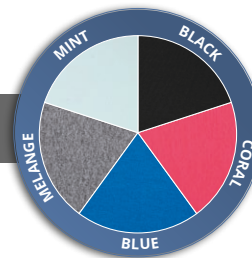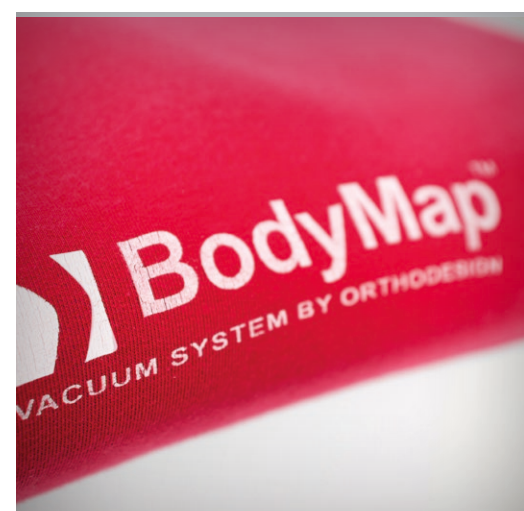
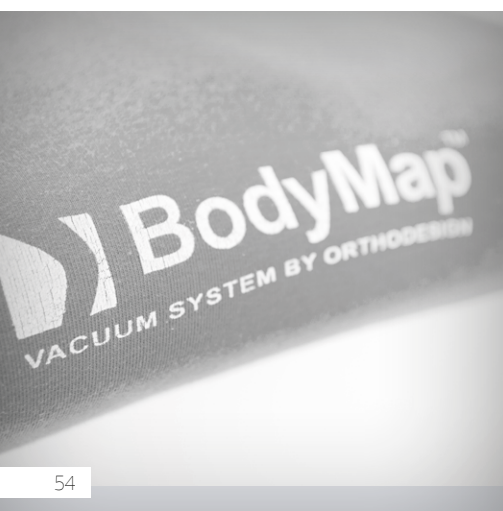
4. Melange



5. Blue

Aerating-massaging pad was created for bed sore prevention for patients that use BodyMap products for a long time. This pad is used mostly in system's seats. Pad can be used only with terry cover if patient is sensitive to abrasion. Aerating-massaging pad makes it possible to lighten as well as ventilate sciatic region. Additionally, when the patient performs movement, his skin is massaged which is really beneficial due to the fact that it improves blood circulation and prevents bedsore development.

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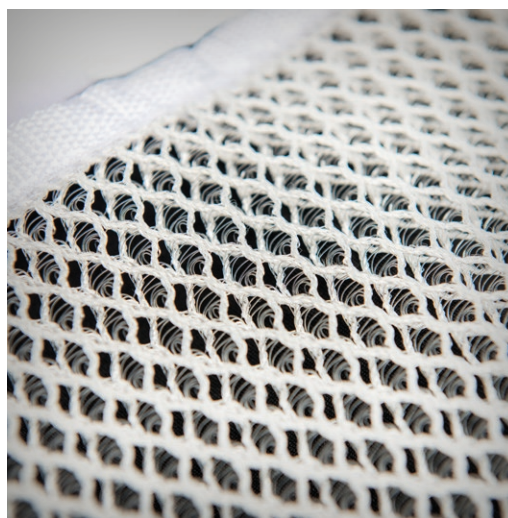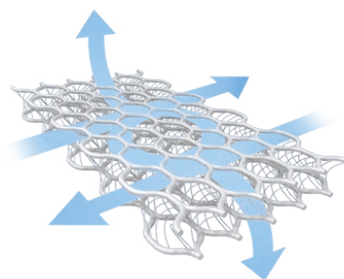
ELECTRIC PUMP / POMPKA ELEKTRYCZNA

Pump is ergonomically shaped and it is equipped with elastic connection pipe for BodyMap cushions. Pipe is easy to carry and pump and, more to the point, it is comfortable to use. Pump is powered with the aid of batteries AA (1,5Vx6) and/or power pack AC (100-240V, 9V, 12,5 W). Pump is efficient, average speed of pumping is 8l/min. The maximal vacuum of pump is about -65 Kpa thanks to which it is possible to pump enough amount of air so as to cushion is hard enough and, moreover, the shape of cushion remains the same for months. Pump is made of solid material, namely ABS.

How to use:

1. Tightly insert the tip of the pump into the cushion's valve

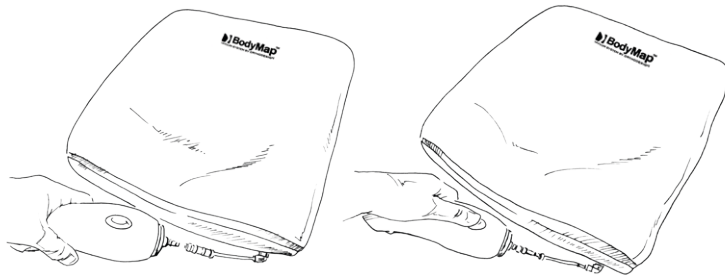2. Press the switch and pump out the air until the desired hardness of the cushion

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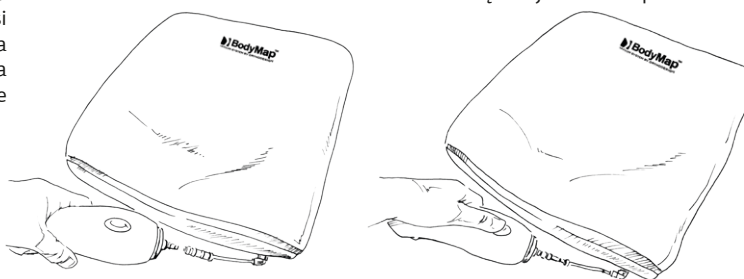
Duży worek

- wymiary 28x48 cm
- 200 g granulatu

As far as additional equipment is concerned, BodyMap system is equipped with bags. They contain granules and they are available in two sizes. Bags are created so as to have control over the amount of granules in BodyMap cushions. It is highly beneficial due to the fact that it is possible to adjust BodyMap cushions to user's needs. It is necessary to take the size of BodyMap cushion into account while choosing the size of bag with granules. BodyMap cushion has to be equipped with service valve in order to pour in or take out granules. Service valve is situated at the back of BodyMap cushion thanks to which it is not very visible and, moreover, it is not an obstacle while using the product. What is more, it is really easy to use it – you just need to unscrew cork and either pour in or take out specific amount of granules.

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3

3. Take out cork which secures funnel from bag with granules. Be careful so as not to spill out its contents.

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2

2. Unscrew carefully valve's cork. **CAUTION! When valve is unscrewed it is forbidden to move pillow violently or press it because then granules may spill out.**

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4

4. Insert tightly bag's funnel into hole from cushion at a depth of 1cm. Pull bag upwards, pour granules in as much as it is needed. If you want to take out some granules from cushion, it is necessary to insert funnel into hole from cushion and then turn cushion in a way that valve is placed down. Next, take out granules as much as it is needed.

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