



Molift Repositioning Sheet

An in-bed lift and positioning sheet





Molift Repositioning Sheet

The Molift Repositioning Sheet is an essential aid for effective care of bed-bound patients. The Repositioning Sheet makes strenuous lifts easier, lessens the risk of injury for the carer and promotes a higher quality of care for patients.

Reduce risks and save time

Repositioning and transfer of users in beds is repeated countless times a day and is associated with injury risks for carers. Molifts Repositioning Sheet reduces the risk of injury and contributes to a higher quality of care by making strenuous lifts and transfers easier.


Comfortable and dignified care

The Molift Repositioning Sheet is designed as an in-bed system for maximum patient comfort. As the sheet is always positioned in the bed, a multitude of care tasks can be done faster and more easily for both carer and patient. Additional demands on a bariatric patients body may adversely affect their comfort and dignity. The Repositioning Sheet can alleviate this.

In addition, the Molift Repositioning Sheet provides support to the user's body, ensuring maximum comfort and a secure feeling when transferring, turning and repositioning.

Facilitates care tasks

The Molift Repositioning Sheet is suitable for bariatric care, turning bed-bound patients and lifting patients higher in the bed. The sheet is also suitable when transferring users from bed to bed and to and from supine surfaces. When treating for example sores and skin issues, the sheet can provide good access to various parts of the body. The Repositioning Sheet is also very useful when changing bed linen.

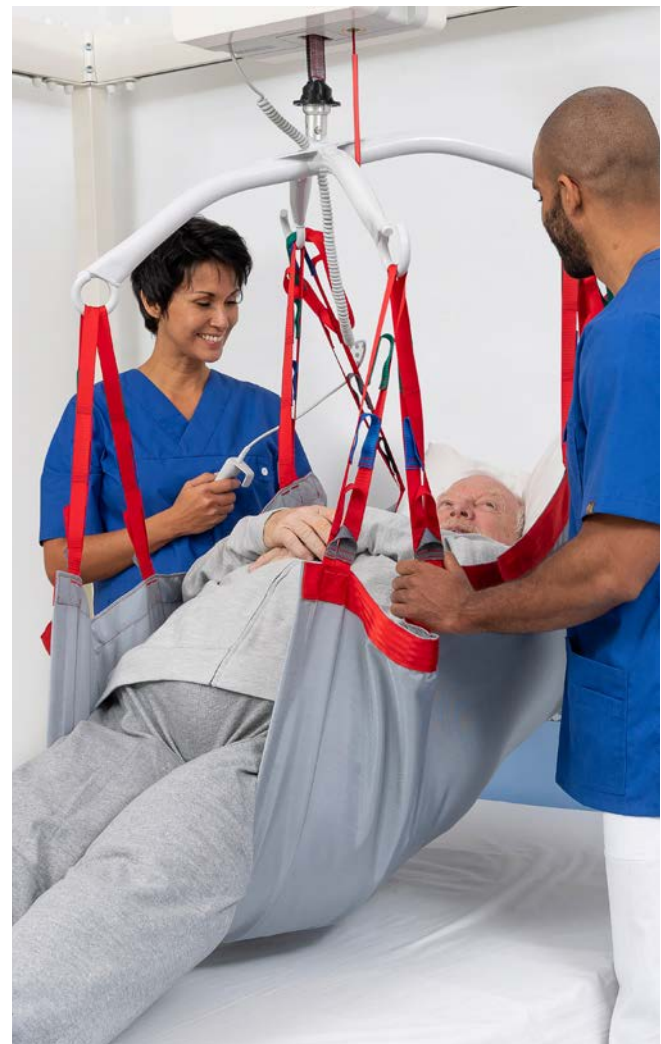

| Product | Packaging | Item no. | Washing instructions |
|--|-------------------------|-----------|--|
| Molift Repositioning Sheet | 1 pc | 1721570 | <p>Machine wash - max. temperature 85 °C/185 °F. Washing at lower temperatures increase the product life span. Tumble drying - low temperature; max. temperature 60 °C/140 °F. Do not iron. No dry cleaning. Do not bleach</p> |
| Molift Repositioning Sheet | 10 pcs | 1721570-C | |
| Material | | | |
| Repositioning sheet: | 100 % Durable polyester | | |
| Lifting straps: | 100 % Polyester | | |
| Specification | | | |
| 8 straps with fixed loops, ¾ length of the bed | | | |
| Length: | 1340 mm (52,76 inch) | | |
| Width: | 1300 mm (51,18 inch) | | |
| | | | |

Features



Time-saving convenience

The Repositioning Sheet is designed to stay fitted on the bed, ready to use at any time. This frees up carers' time for other daily care tasks.



Clear colour coding

Thanks to clear colour coding of straps, carers can easily see and communicate which position/colour the strap should be attached to on each side of the sling bar.



Maximum user comfort

The webbing on the Repositioning Sheet is sewn along the edge of the sheet. This ensures a smooth surface and a comfortable transfer situation, reducing the risk of pressure injury for the patient.



Part of a flexible system

The Repositioning Sheet is part of Molift's wide range of flexible patient handling solutions. These include rail systems, ceiling and mobile hoists, sling bars and slings for all kinds of care situations.



Accessories



Immedia Positioning Wedge

When turning a user to the side, the body can be supported by a pillow, the Positioning Wedge, in the back.

Small, L: 530 mm
Large, L: 800 mm

IM53NS
IM53/80NS



MultiGlide SG nylon sheet

Tubular Nylon sheets can be used to minimize friction under pressure points, e.g. feet when moving higher in bed with the Molift Repositioning sheet.

Tubular 600 mm
Tubular 400 mm

IM0066
IM0040



Etac R82
Unit D4A, Coombswood Business Park East
Coombswood Way, Halesowen,
West Midlands B62 8BH
Tel 0121 561 2222
Email: enquiries@etac.uk.R82.com
Visit: etac.com/uk


Creating Possibilities