




# Partial Hand SOLUTIONS, LLC

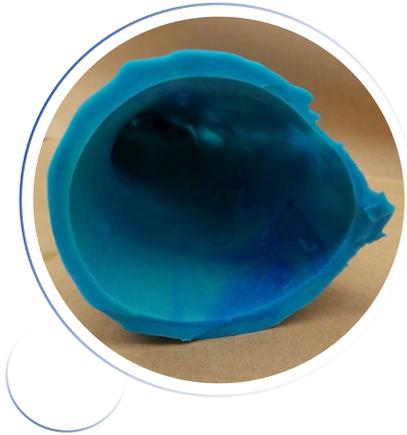

## *Technical Instructions* M-Finger Casting Procedure

### **REQUIRED MATERIALS**

Smooth On Fast Set Kit  
Vaseline  
Gloves  
Tongue Depressors  
Scale





Instruct patient on proper hand positioning prior to casting:


- a. Slight abduction between fingers
- b. extended but relaxed position
- c. Wrist neutral



Measure equal parts of Smooth On casting material, and then blend thoroughly.



Apply casting material to limb, and thoroughly cover limb with the first coating.

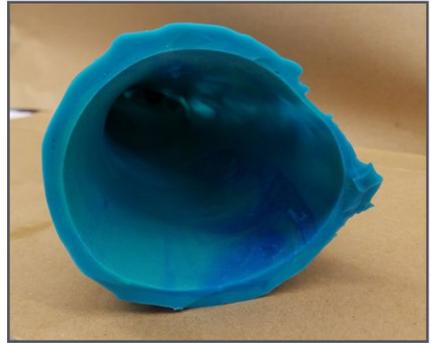

Confirm adequate thickness prior to cast removal.  
(Should be  $\frac{1}{2}$  inch thick)

M-Fingers are distributed by:  
 **college park**  
800-728-7950 | 586-294-7950  
27955 College Park Drive, Warren, MI 48088

## TECHNICAL ASSISTANCE/ EMERGENCY SERVICE 24-7-365

College Park's regular office hours are Monday thru Friday, 8:30 am - 5:30 pm (EST). After hours, an emergency Technical Service number is available by calling College Park's main line.



**Partial Hand**  
**SOLUTIONS, LLC**



**Partial Hand Solutions**  
86 Ladyslipper Lane  
Southington, CT USA 06489  
+1 (860) 538-5532



**EMERGO EUROPE**  
Prinsessegracht 20,  
2514 AP  
The Hague, Netherlands

**MADE IN THE USA**  
667 INS PHS MCP 170914

