

TITAN series


Instructions for use



TITAN series (TITAN Full, TITAN Partial, TITAN Thumb)

1. If using a silicone socket, apply a small amount of water to dampen limb and push on prosthetic socket to limb.
2. To flex finger, apply pressure to back side of fingertip and push to desired location. To extend finger back, pull on fingertip to dislocate pin and reposition to desired location.
3. New users should start with a wearing schedule and ease in to a full day of use.
4. To remove prosthetic socket from limb, pull on socket to remove or consult with your prosthetist or therapist on proper technique.
4. Clean daily with fragrant free anti-bacterial wipes. Consult with your Doctor and prosthetist on preferred cleansing wipe.
5. Prior to patient receiving the device, LOCTITE 242 should be applied to all screws.

[TITAN Full](#)

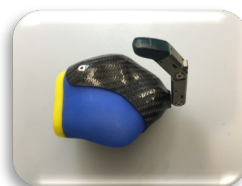

Shown connected to prosthetic socket

[TITAN Full](#)



[TITAN Partial](#)



Shown connected to prosthetic socket

[TITAN Partial](#)



[TITAN Thumb](#)



Shown connected to prosthetic socket

[TITAN Thumb](#)



PARTIAL HAND SOLUTIONS

86 Ladyslipper Lane
Southington, CT, USA 06489
+1 (860) 538-5532

EC

REP

EMERGO EUROPE

Prinsessegracht 20
2514 AP The Hague
The Netherlands

