

K SCAN SPRAY

- K SCAN SPRAY is the most thin anti-light reflection spray for dental CAD/CAM scanning
- Non flammable

Quick and Easy Cleaning

Easily removed by steam cleaner

Best choice for custom abutment scan

Guarantee the best fit of zirconia crown and custom abutment

Reasonable Price

Better price with amount of 200g

Do not spray thickly

No need for thick application
Thin application over several times is more effective than one-time thick application

With K Scan, Reduce the working dust!

Reduced dust by 50% compared to existing product, using special solvents

PRECAUTIONS

Do not store in temperature above 40°C
Use ventilation when used in a closed room
Do not dispose into fire and keep it in a closed place.



EASY SCAN PEN

10ml

- Easy scan pen is for coating the surface of the crown to facilitate CAD/CAM scanning
- Non flammable

HOW TO USE

- Please shake well before use. For the first use, press pen point 5-10 times until pen tip get moisturize by liquid
- If the pen point gets dry during use, gently press the pen tip 1-2 times. Do not shake the pen after pressing the pen point. Liquid could splatter from the pen.
- Close the lid of the pen when it is not in use. Liquid may leak if the lid is open.
- The liquid in the pen will evaporate if the pen tip gets exposed for a long time.



EASY SCAN POWDER

10g

- Easy Scan powder is a powder-type scan coating for the preparation of CAD/CAM scan.
- Non flammable

HOW TO USE

- Brush or gently pat the powder to the object.
- The build up could become broken if patted with strong impact

