


Block Specimen Submission Instructions

Please use these instructions when preparing blocks for FedEx Overnight shipment
Please review the instructions and contents of the kit before preparing the requested tumor samples



qty: 1 - 4"x 6"
biohazard bag

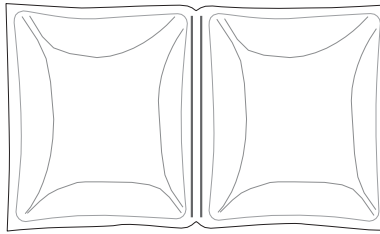

qty: 1
6oz gel cold pack

Specimen submitted must have 30% or more tumor content. Do not submit specimens that are older than 5 years. Blocks that are shipped must be sent with provided chilled cold pack via FedEx Overnight (Monday-Thursday) to preserve the integrity of the specimen. Please avoid sending out on a Friday for temperature control issues.

FFPE Tumor Block Submission Instructions

- It is highly important to submit the correct specimen for the Ignite RPPA Assay testing
 - Improperly prepared blocks will be rejected and potentially delay testing
 - Blocks that are shipped must be sent on a chilled cold pack via FedEx Overnight (Monday – Thursday) to preserve the integrity of the specimen. Please avoid sending on a Friday due to temperature control issues.
 - Ensure there is sufficient tissue. Thin or scant tissue blocks cannot be used for testing
1. Provide a copy of the requisition and pathology report for this specimen along with specimen submission
 2. Place patient specimen block within provided biohazard bag – place the kit, block in the biohazard bag, and chilled cold pack in the FedEx envelope along with a copy of the requisition and pathology report
 3. Send materials via FedEx Overnight
 4. A copy of the air bill should be maintained for your records
 5. Schedule package pick-up with FedEx by calling: 1-800-463-3339

Sample types not currently accepted:

Bone, bone marrow, any type of fluids (i.e., pleural effusions) and any samples with a high blood content/low tumor cell content

Ignite Proteomics, LLC

Attn: Lab Manager

15000 W 6th Ave., Suite 400

Golden, CO 80401

To order additional kits:

Call: 720-935-3916

| Fax: 720-302-2400

| Email: clientservices@igniteproteomics.com