



Application Instructions

Product: Universal Wrist Brace with Thumb Spica
Model: 170-R / 180-L

Constructed of durable, lightweight material Universal sizing Adjustable dorsal stay Removable and malleable aluminum palmer stay for correct fitting.



Size Chart



Order #	Size
170-R	Right Universal
180-L	Left Universal

Sizing: No Sizing Necessary - Universally Sized

HOW TO APPLY:

Open brace and position hand in thumb brace. The palmer stay should line up below the middle crease of the hand. The thumb stay should follow the length of the thumb and arm. Align the lacing dorsal stay with the middle of the knuckles. Wrap the wide thumb strap around the thumb and secure to hook. Secure the thinner strap between the thumb and index finger and attach to hook. Wrap the lacing straps around wrist and attach to hook and loop. Re-adjust if necessary. The knuckles should be free for bending.

CARE INSTRUCTIONS:

Fasten Closures, Hand Wash Cold, Use Mild Soap Only, Air Dry Away From Heat

CAUTION:

Wear as recommended. Always consult a physician when encountering persistent pain, discomfort, inflammation, or swelling. Contact neoprene material with skin may cause irritation. Persons with a known susceptibility to dermatitis or with broken skin should avoid contact with neoprene. Anyone who develops skin irritation from contact with this material should discontinue its use immediately. Should not be worn by persons with phlebitis or circulatory disorders. Not recommended to be worn while sleeping.