

# Instructions for Cleaning Face Shields, Eye Goggles or Safety Glasses

**NOTE:** This does NOT apply to surgical/procedural masks with visors attached as they cannot be re-used.

- Doff other PPE as per usual procedures and perform hand hygiene with an alcohol-based hand rub.
- Open a blue pad and place the pad on clean surface and perform hand hygiene with an alcohol based hand rub.
- Carefully remove the face shield, eye goggles or safety glasses and place on blue pad and perform hand hygiene with an alcohol-based hand rub
- Don gloves and wipe the inside (portion facing face), followed by the outside of the face shield with a clean cloth saturated with neutral detergent solution or a cleaner wipe (e.g. baby wipe).
- Carefully wipe the outside of the face shield with the Accel wipes, to visibly dampen it, and allow one minute of contact time on the blue pad.
- Avoid using Accel wipes on the foam and elastic band of face shields or eye goggles.
- Wipe the outside of the face shield with clean water or alcohol wipe to remove residue.
- Allow face shield to air dry on the blue pad.
- Remove gloves and perform hand hygiene.
- Once face shield is dry, move to a storage container or bag, being careful not to touch inner surface and perform hand hygiene with an alcohol-based hand rub.

